


# Sound Box & Manifold

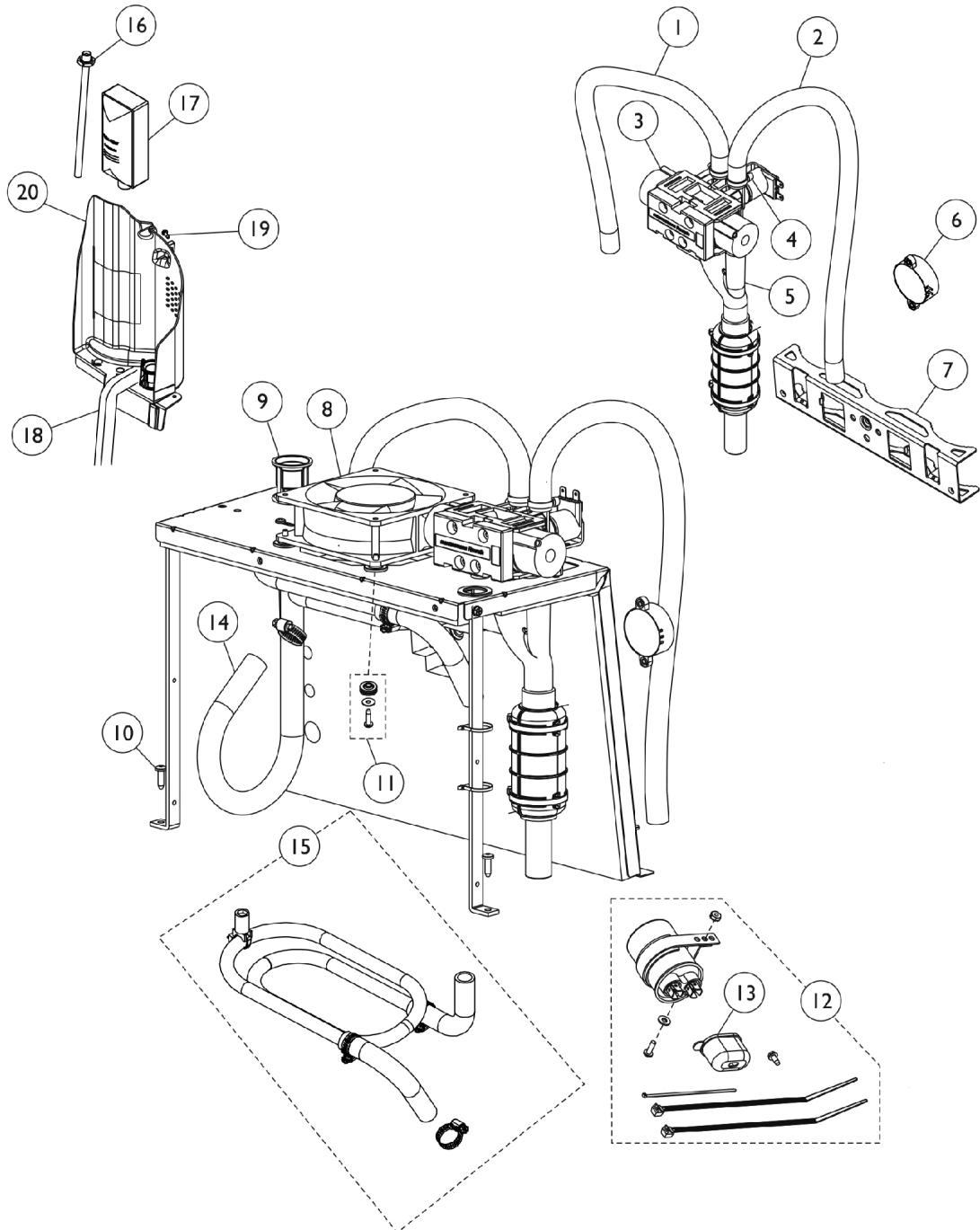

## Sound Box & Manifold

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1108783	Tubing, Braided (1/2 x 23")	1	1
2		1108784	Tubing, Braided (1/2 x 20-1/4")	1	1
3		1101141	Kit, 4-Way Valve w/ Screws	1	1
4		2001682	Tie Wrap (8" L)	1	5
5		1136945-NLA	NLA - Tech Only - Muffler/Y-Tube Assembly	1	1
6		2001765	Harness, Horn/Alarm	1	1
7		1125477-NLA	NLA - Tech Only - Bracket, Soundbox	1	1
8	C	See Note	See Footnote	1	1
9		1106675	Coupling, Inlet Filter	1	1
10		1108810	Screw, Hi-Lo Six Lobe Drive (#10-16 x 5/8")	1	7
11	A	1141964	Kit, Fan Mounting Hardware	1	1
			Grommet, Isolator w/ Spacer	4	1
			Washer (#10)	4	1
12	B	1141835	NLA - Screw, Phillips Pan Head (10-24 x 3/4")	4	1
			Kit, Capacitor 10 MFD	1	1
			NLA - Tech Only - Capacitor, 10 MFD	1	1
			Clamp, Cap	1	1
			Screw, Phillips Pan Head (8-32 x 1/2")	1	1
			Washer, Flat (#8)	1	1
			NLA - Nut, Keps (#6-32)	1	1
			NLA - Boot, Terminal	1	1
			Tie Wrap (3-29/32" L)	1	1
			Screw, Phillips Hex Washer Head (#8-32 x 3/8")	1	1
			Tie Wrap (8" L)	1	2
13		P0875X560-NLA	NLA - Boot, Terminal	1	1
14		1112561	Tube, Compressor	1	1
15		1108817	Heat Exchanger Assembly	1	1
16		2000799	Adapter with Tubing for Humidifier Bottle	1	1
17		1131249	Filter, Inlet Sound Reduced	1	1
18		2000812	Seal, Foam	1	2
19		1128404	Screw, Phillips Hex Washer Head (#8-32 x 3/8")	1	4
20		1115183	Partition Assembly	1	1

NOTE: A - Includes (4) Isolator Grommets, (4) Washers and (4) Screws.  
 B - Includes (1) Capacitor w/mounting Hardware, Tie Wraps and Boot.  
 C - Part number is no longer serviceable in the field.