


# Seat Frame Mounting Hardware & Interface Mtg. Plates (CGR) For TDX SP/SP-GT w/ 22NF Batteries (Before 7/14/11)

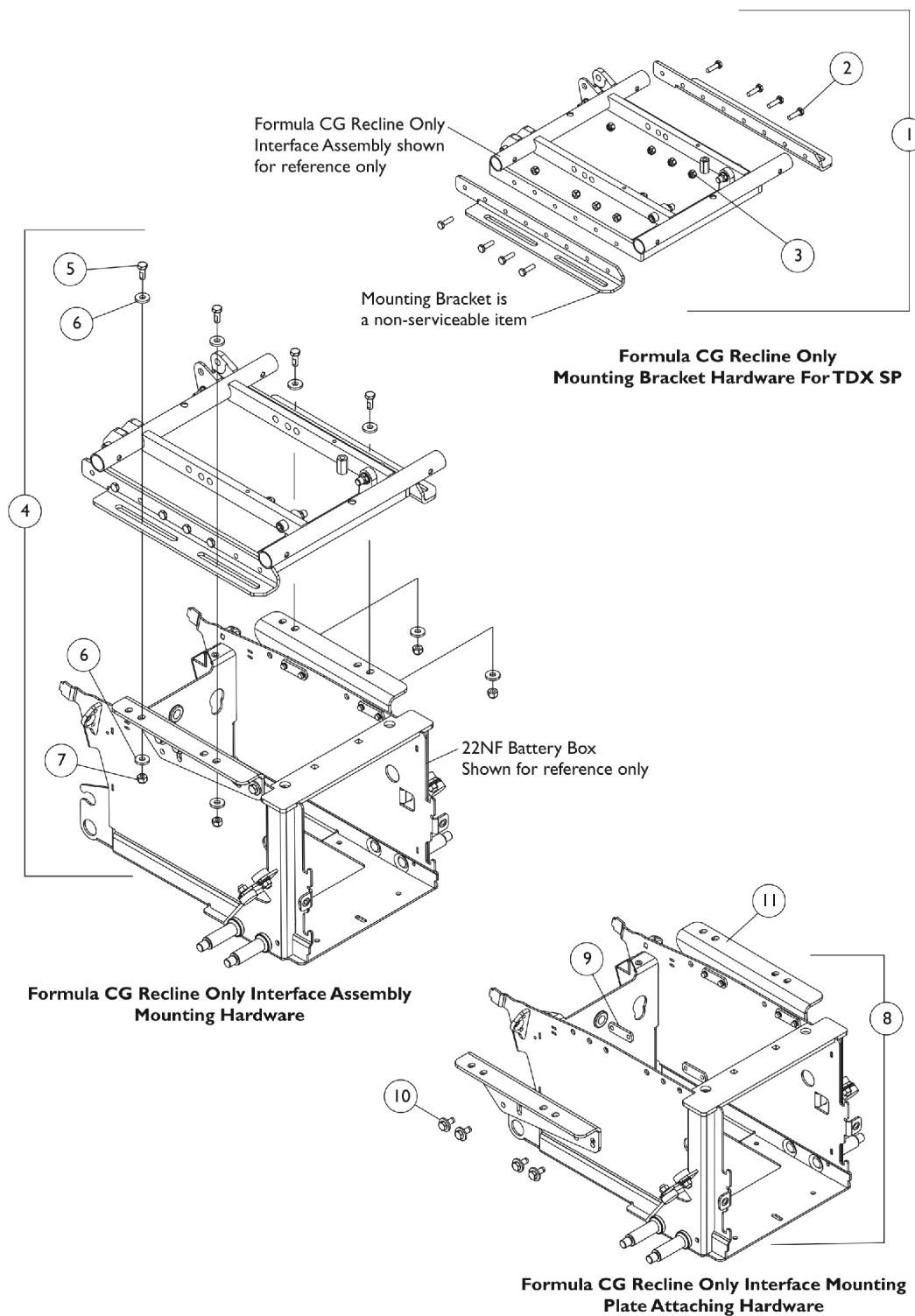

## Seat Frame Mounting Hardware & Interface Mtg. Plates (CGR) For TDX SP/SP-GT w/ 22NF Batteries (Before 7/14/11)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1143803-NLA	NLA - Kit, Seat Mounting Plate Hardware (Recline Only)	1	1
2			Screw, Hex Head (1/4-20 x 7/8")	1	8
3			Locknut (1/4-20)	1	8
4	B	1121699	Kit, Formula CG Tilt, Recline & Tilt/Recline Interface Asbly. Mtg. Hdwr. (For TDX SP w/ 22NF Batteries)	1	1
5			Screw, Hex Head (5/16-18 x 1")	1	4
6			Washer, Flat (21/64 x 53/64 x 7/64")	1	8
7			Locknut, High Profile (5/16-18)	1	4
8	C	1121700	Kit, Interface Mounting Plate Attaching Hardware	1	1
9			Nut, Weld (5/16-18 x 1-3/8" Spacing)	1	4
10			Screw, Hex Head SEMS w/ Patch (5/16-18 x 3/4")	1	8
11	D	1121697-NLA	NLA - Plate, Low Interface Mounting - Right	1	1
11	D	1121698-NLA	NLA - Plate, Low Interface Mounting - Left	1	1
11	E	1148011-NLA	NLA - Plate, Medium Interface Mounting - Right	1	1
11	E	1148012-NLA	NLA - Plate, Medium Interface Mounting - Left	1	1
11	F	1148013-NLA	NLA - Plate, High Interface Mounting - Right	1	1
11	F	1148014-NLA	NLA - Plate, High Interface Mounting - Left	1	1
NOTE: A - Includes items 2 and 3 B - Includes items 5-7 C - Includes items 9 and 10 D - Low Seat-To-Floor = 17.5" at 0 degrees E - Medium Seat-To-Floor = 18.5" at 0 degrees F - High Seat-To-Floor = 19.5" at 0 degrees					