
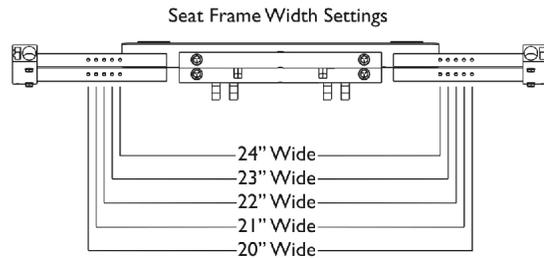


# Seat Frame, Seat Pan and Hardware

(20"-24" W x 16"-22" D) Conv., (20", 22", 24" W x 16"-22" D) Contoura &  
(CGE For 3G Storm)

**NOTE: 3G Storm (CGE) Center Seat Frame uses the Conventional Seat Frame Rails and the 23"-24" seat width settings are not used**

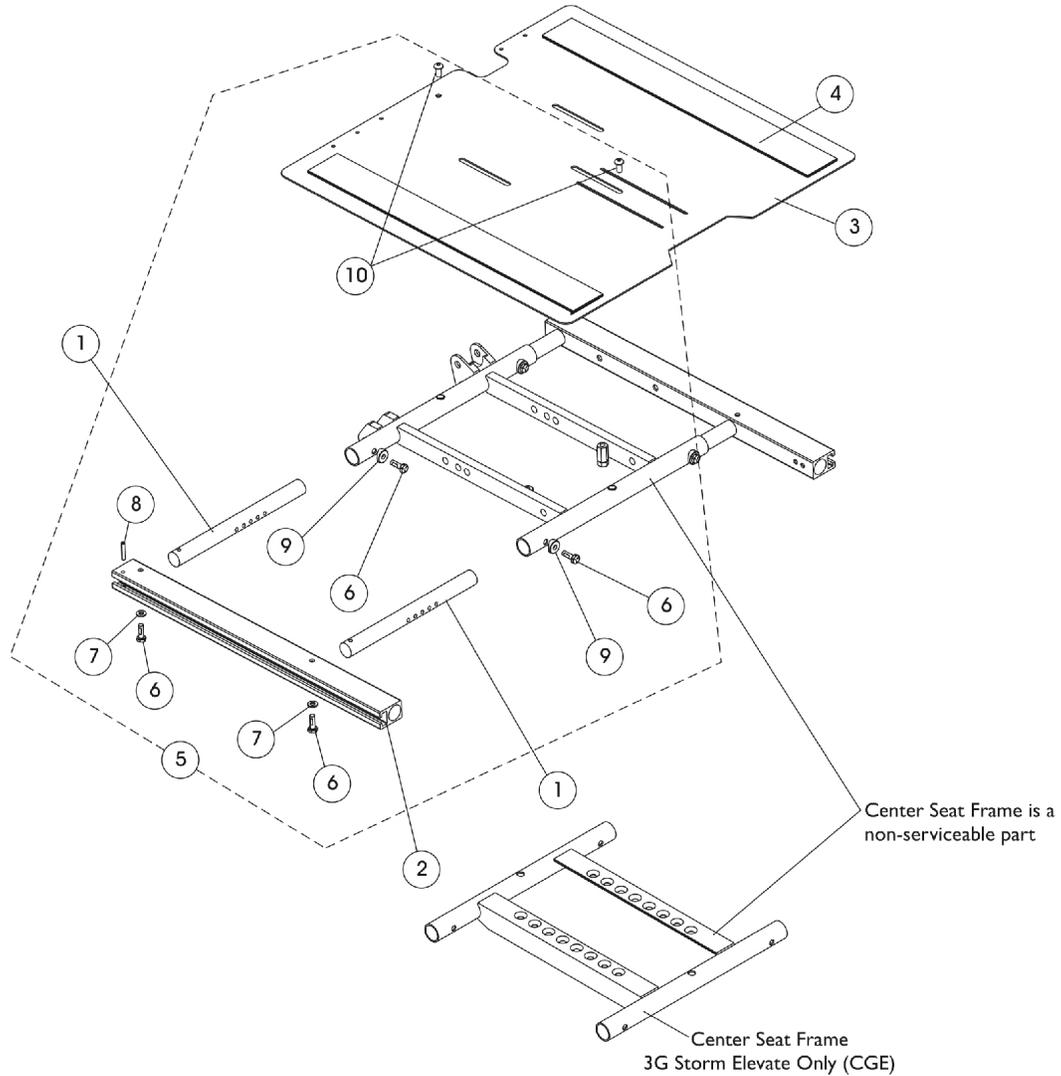

## Seat Frame, Seat Pan and Hardware (20"-24" W x 16"-22" D) Conv., (20", 22", 24" W x 16"-22" D) Contoura & (CGE For 3G Storm)

**NOTE: 3G Storm (CGE) Center Seat Frame uses the Conventional Seat Frame Rails and the 23"-24" seat width settings are not used**

| Item Number | Note | Part Number | Description                                                       | Quantity Per |          |
|-------------|------|-------------|-------------------------------------------------------------------|--------------|----------|
|             |      |             |                                                                   | Package      | Assembly |
| 1           |      | 1120283     | Crossbar, Seat Frame (20" - 24" W)                                | 1            | 4        |
| 2           |      | 1142951     | Rail, Seat Frame (For 12 - 19" D Conventional)                    | 1            | 2        |
| 2           |      | 1142952     | Rail, Seat Frame (For 19 - 22" D Conventional)                    | 1            | 2        |
| 2           | D,E  | 1142953     | Rail, Seat Frame (For 16 - 19" D Contoura)                        | 1            | 2        |
| 2           | D,E  | 1142954     | Rail, Seat Frame (For 19 - 22" D Contoura)                        | 1            | 2        |
| 3           |      | 1143790     | Pan, Seat with Hook and Loop Fasteners (20 - 24" W x 16 - 19" D)  | 1            | 1        |
| 3           |      | 1143791     | Pan, Seat, with Hook and Loop Fasteners (20 - 24" W x 19 - 22" D) | 1            | 1        |
| 4           | A    | 1043082     | Hook Fastener with Adhesive Back (2" W x 18" L)                   | 1            | 2        |
| 4           | A    | 1044580     | Loop Fastener with Adhesive Back (2" W x 18" L)                   | 1            | 2        |
| 4           | B,C  | 1043084     | Hook Fastener with Adhesive back (2" W x 22" L)                   | 1            | 2        |
| 4           | B,C  | 1044584     | Loop Fastener with Adhesive back (2" W x 22" L)                   | 1            | 2        |
| 5           | C    | 1144581     | Kit, Seat Frame Assembly Hardware (12"-24" Wide)                  | 1            | 1        |
| 6           |      |             | Screw, Hex Head w/ Patch (1/4-20 x 3/4")                          | 1            | 8        |
| 7           |      |             | Washer (1/4 x 1/2 x 1/16")                                        | 1            | 4        |
| 8           |      |             | NLA - Pin, Roll (3/16 x 1")                                       | 1            | 2        |
| 9           |      |             | Spacer, Coved (1/4 x 11/16 x 1/2")                                | 1            | 4        |
| 10          |      |             | Screw, Socket Button (1/4-20 x 3/4")                              | 1            | 2        |

NOTE: A - For 16-19" deep seat pans  
 B - For 19-22" deep seat pans  
 C - Includes items 6-10  
 D - When a Contoura Seating System and Power Elevating Legrest (MPCTMT/MPCTMT-L) are ordered "together" Before 7/21/11, shorter length 15" or 18" long Conventional Seat Rails must be ordered. This will prevent the legrest from being positioned to far forward.  
 E - After 7/20/11 part numbers 1142953 and 1142954 are no longer used on-chair and were replaced with part numbers 1142951 and 1142952 for "ALL" Contoura Back models regardless of front rigging selection and Conventional Back models.