






Front Rigging Hangers and Hardware Pin, Sector and Hemi Style (12"-16" Wide) (CGT)

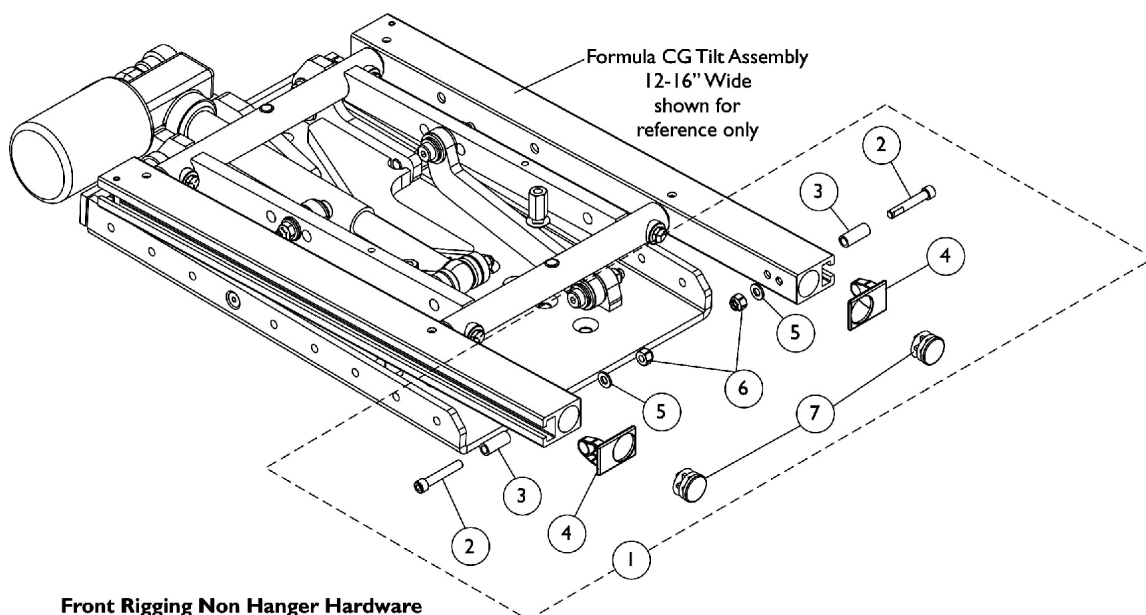

**Front Rigging Non Hanger Hardware
(When Front Rigging Is Excluded From Chair)**

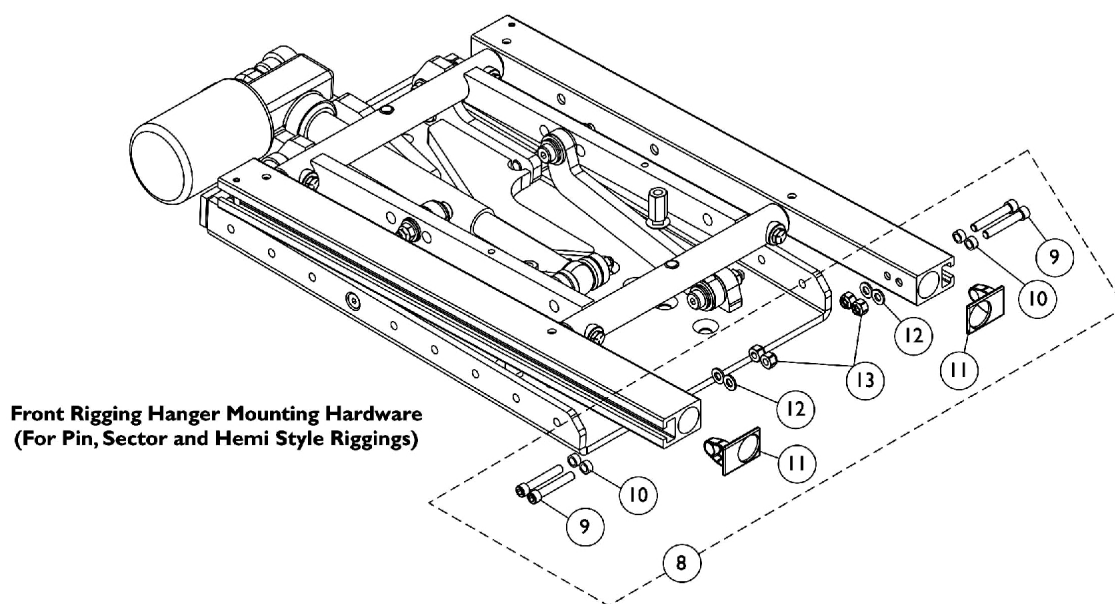

**Front Rigging Hanger Mounting Hardware
(For Pin, Sector and Hemi Style Riggings)**

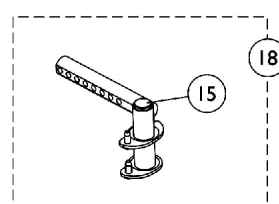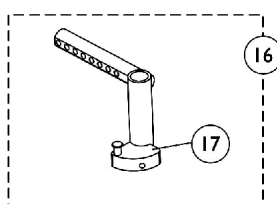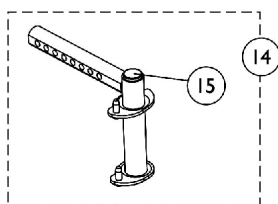

Front Rigging Hangers and Hardware

Pin, Sector and Hemi Style (12"-16" Wide) (CGT)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1126150	Kit, Side Rail Cap Hardware	2	1
2			Screw, Socket Head Cap w/ Patch (1/4-20 x 1-1/2")	1	2
3			Spacer (1/4 x 3/8 x 29/32")	1	2
4			Cap, End, Side Rail, Black	1	2
5			Washer (1/4 x 1/2 x 1/16")	1	2
6			Locknut (1/4-20)	1	2
7			Plug Button, Black (7/8")	1	2
8	B	1136019	Kit, Front Rigging Hanger Mounting Hardware	2	1
9			Screw, Socket Head Cap w/ Patch (1/4-20 x 1-1/2")	1	4
10			NLA - Spacer (1/4 x 3/8 x 3/16")	1	4
11			Cap, End, Side Rail, Black	1	2
12			Washer (1/4 x 1/2 x 1/16")	1	4
13			Locknut (1/4-20)	1	4
14		1143781	Front Rigging Hanger Assembly w/ Plug Button, Pin Style, Adult, Black - Right	1	1
14		1143782	Front Rigging Hanger Assembly w/ Plug Button, Pin Style, Adult, Black - Left	1	1
15		1134426	Package, Plug Button, Black (3/4")	10	1
16	C	1143416	Front Rigging Hanger Assembly w/ Sector Block, Black - Right	1	1
16	C	1143417	Front Rigging Hanger Assembly w/ Sector Block, Black - Left	1	1
17	D	1054571	Kit, Sector Block - Right	1	1
17	D	1054572	Kit, Sector Block - Left	1	1
18	E	1143783	Front Rigging Hanger Assembly w/ Plug Button, Hemi, Black - Right	1	1
18	E	1143784	Front Rigging Hanger Assembly w/ Plug Button, Hemi, Black - Left	1	1
NOTE: A - Includes items 2-7 B - Includes items 9-13 C - Includes Front Rigging Hanger and item 17 D - Includes the Sector Block, Screw and Washer E - Used for TDX SP/SR models only					