


90 WSBT Legreset Tube Support Assembly

Pronto R2 Made After 9/24/00 Only

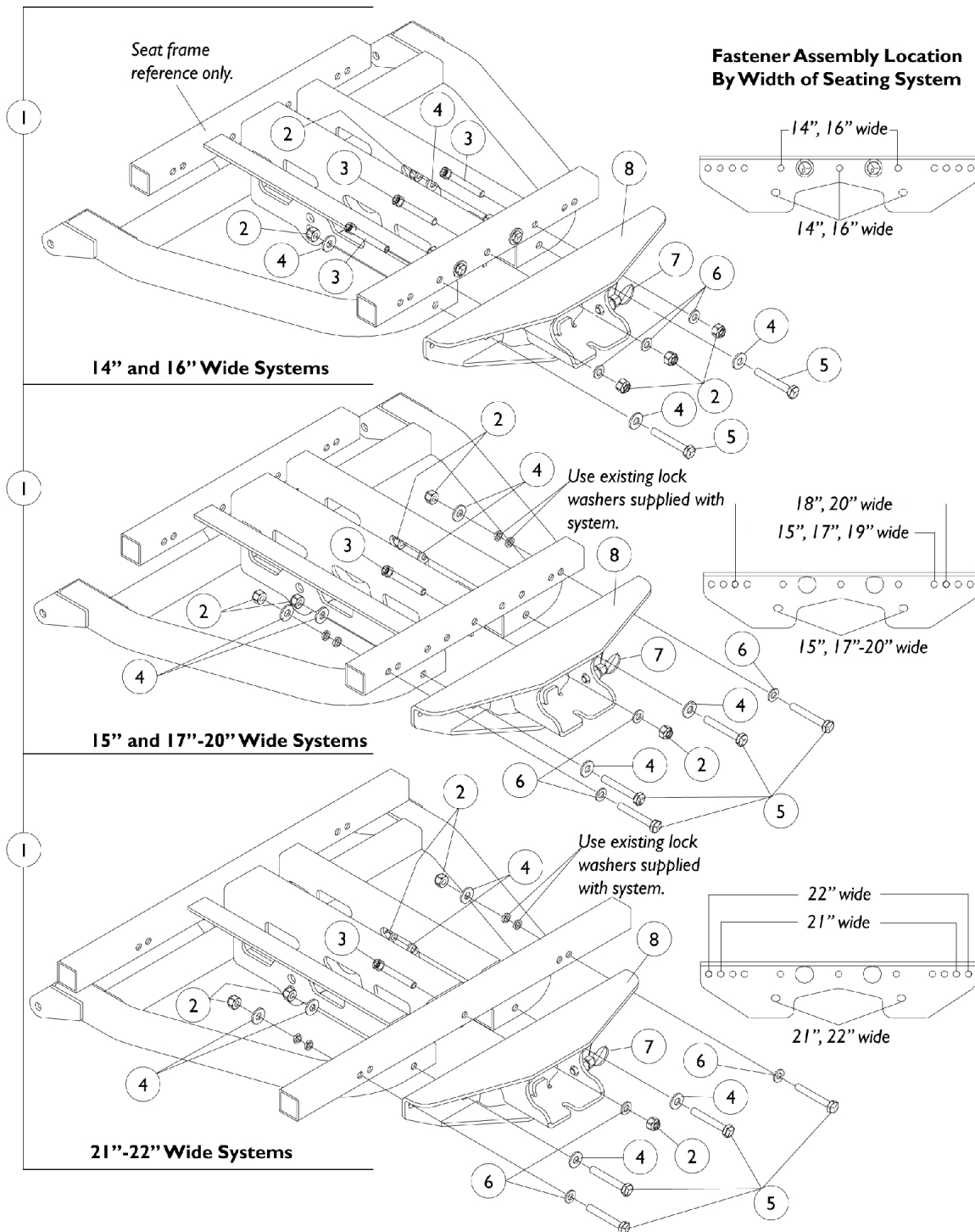

90 WSBT Legreset Tube Support Assembly

Pronto R2 Made After 9/24/00 Only

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1101084-NLA	NLA - Kit, Legrest Tube Support and Hardware, Black (14" and 16" W)	1	1
1		1101085-NLA	NLA - Kit, Legrest Tube Support and Hardware, Black (15" and 17 - 20" W)	1	1
1		1101086-NLA	NLA - Kit, Legrest Tube Support and Hardware, Black (21 - 22" W)	1	1
2		7930333	Locknut (1/4-20)	25	5
3	A	1003424	Screw, Socket Head (1/4-20 x 1-3/4")	1	3
4	B	20105X000-NLA	NLA - Washer, Flat (1/4 x 5/8 x 1/16")	1	4
5	C	1024793-NLA	NLA - Screw, Hex Head (1/4-20 x 1-3/4")	1	2
6		1026907	Washer (1/4 x 1/2 x 1/16")	1	3
7		1099506-NLA	NLA - Pin, Pop	1	1
8		1101083-NLA	NLA - Legrest Support with Pin Assembly, Black	1	1
NOTE: A - 14" and 16" Wide has 3 ea. assembly where as 15", 17"20" and 21" - 22" Wide has 1 ea. assembly. B - 14" and 16" Wide has 4 ea. assembly where as 15", 17"20" and 21" - 22" Wide has 6 ea. assembly. C - 14" and 16" Wide has 2 ea. assembly where as 15", 17"20" and 21" - 22" Wide has 4 ea. assembly.					