

Oxygen Concentrator

**Platinum™
P5NXG1
P5NXGC1**



Yes, you can.

Parts Catalog Usage Guide

The information contained in this document is subject to change without notice. This catalog contains service parts for Form No. 20-098. If you have any questions regarding this catalog, please contact one of our Customer Service Representatives.

For Customer Service Support Call:

1-800-333-6900

For Faxed In Orders Call:

1-800-678-4682 or

1-440-366-0709

For Technical Service Call:

1-800-832-4707

Literature Can Be Found at:

www.invacare.com or

1-800-828-6272

SYMBOL DEFINITION

Throughout this catalog you will notice letters to draw your attention to specific footnotes which are located at the bottom of the page or below specified charts.

Examples:

A,B,C etc. = Special Comments

NOTE: = General information pertaining to that specific page

NLA = No Longer Available

HARDWARE MEASUREMENTS

The measurements for the hardware will be listed as follows:

Screws, Bolts Diameter X Thread Count/Pitch X Length

Washers Inside Diameter X Outside Diameter X Thickness

Spacers Inside Diameter X Outside Diameter X Length

For your convenience, we have supplied an area for you to list your Invacare Representatives.

Name	Phone Numbers


© 2021 Invacare Corporation, Elyria, Ohio. Invacare, the Medallion Design and Yes, you can. are registered trademarks of the Invacare Corporation. Platinum is a trademark of Invacare Corporation. Palnut is a registered trademark of Transtechnology Corporation. Information is subject to change without notification. Palnut is a registered trademark of Transtechnology Corporation. Information is subject to change without notification

TABLE of CONTENTS

Description	Page Number
Accessory Bag	29
Base Assembly, NA	9
Cabinet Shroud w/ Filters and Retractable Handle Assembly	11
Compressor Assembly, Platinum 5NXG	19
Compressor Rebuild Kit - NA	21
Control Panel and P.C. Board Assembly	13
Fans, Compressor	23
Inlet Filter Assembly	27
Internal Assembly (continued) Bleed Tubing, Sieve Beds to Product Tank	17
Internal Assembly, Sieve Beds, 3-Way Valve, Capacitor, Exhaust Muffler, Heat Exchanger ..	15
Packaging and Literature	3
Product Tank Assembly	25
Shells, Inner Component (cont.) Inlet Filter and Access Door Section	7
Shells, Inner Component, Base and Front Section	5

Packaging and Literature

When adjusting or replacing parts, please refer to your Owner's Manual for assistance. The following Owner's Manuals can be found at www.invacare.com/homecare under Literature Search for the following:
60125576 User Manual P5NXG1 - NA, 60127051 User Manual P5NXGC1 French Canadian - CA.

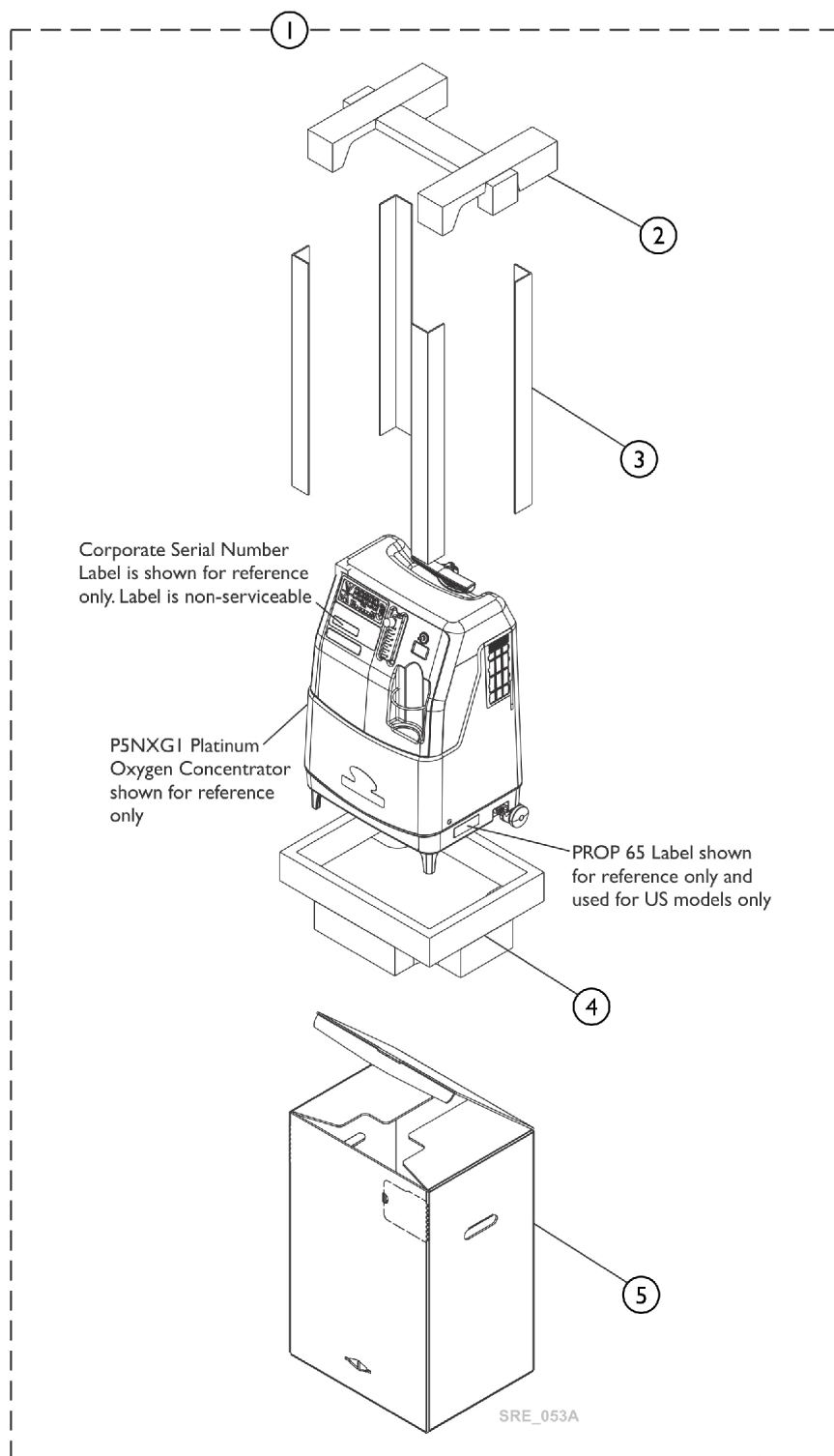


Packaging and Literature

When adjusting or replacing parts, please refer to your Owner's Manual for assistance. The following Owner's Manuals can be found at www.invacare.com/homecare under Literature Search for the following:
60125576 User Manual P5NXG1 - NA, 60127051 User Manual P5NXGC1 French Canadian - CA.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	60127159	Kit, Packaging, Platinum 5NXG	1	1
2			Insert, Packaging Top Foam, Platinum 5NXG	1	1
3			Post, Corner	1	4
4			Insert, Packaging Bottom Foam, Platinum 5NXG	1	1
5			Carton, Packaging Platinum 5NXG	1	1
			Bag, Packaging Platinum 5NXG	1	1
			Sheet, Insert Parts Service	1	1
NOTE: A - Includes items 2-5 and Packaging Bag (Not Shown).					

Shells, Inner Component Base and Front Section

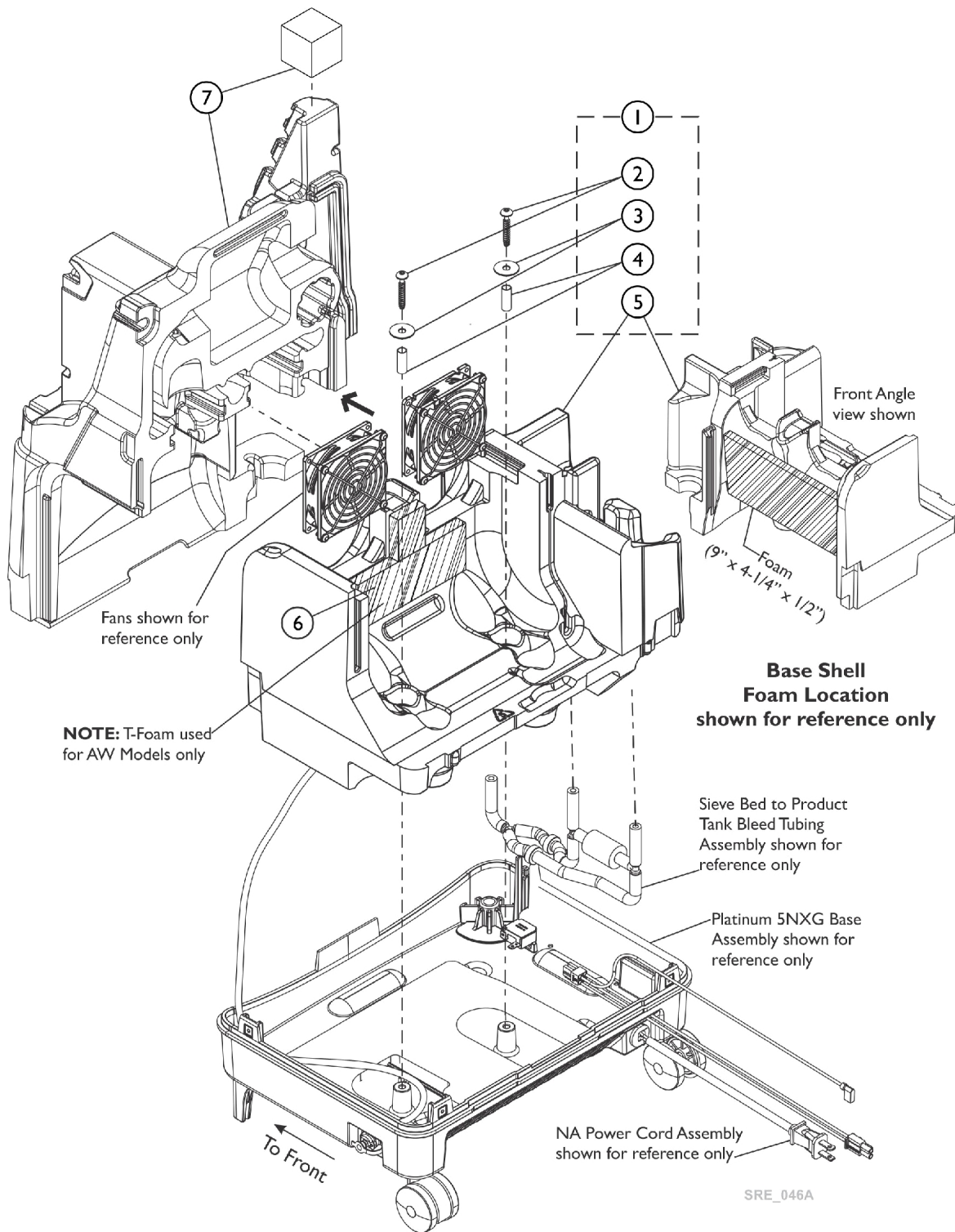


Shells, Inner Component Base and Front Section

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	60127176	Kit, Base Section Shell w/ Hardware, US	1	1
2			Screw, Pan Head TRD FRM (1/4-10 x 2")	1	2
3			Washer, Flat (9/32 x 1 x 3/64")	1	2
4			Spacer, Split Seam (5/16 x 13/32 x 1")	1	2
5			Shell, Base w/ Foam Section	1	1
			Sheet, Insert Parts Service	1	1
7		60127174	Kit, Front Section Shell	1	1
			Shell, Front Section	1	1
			Foam, Support, w/ Self Adhesive (1-1/2 x 1-1/2 x 1-1/2")	1	1
			Sheet, Insert Parts Service	1	1
NOTE: A - Includes items 2-5. For North America (NA) models					

Shells, Inner Component (continued)

Inlet Filter and Access Door Section

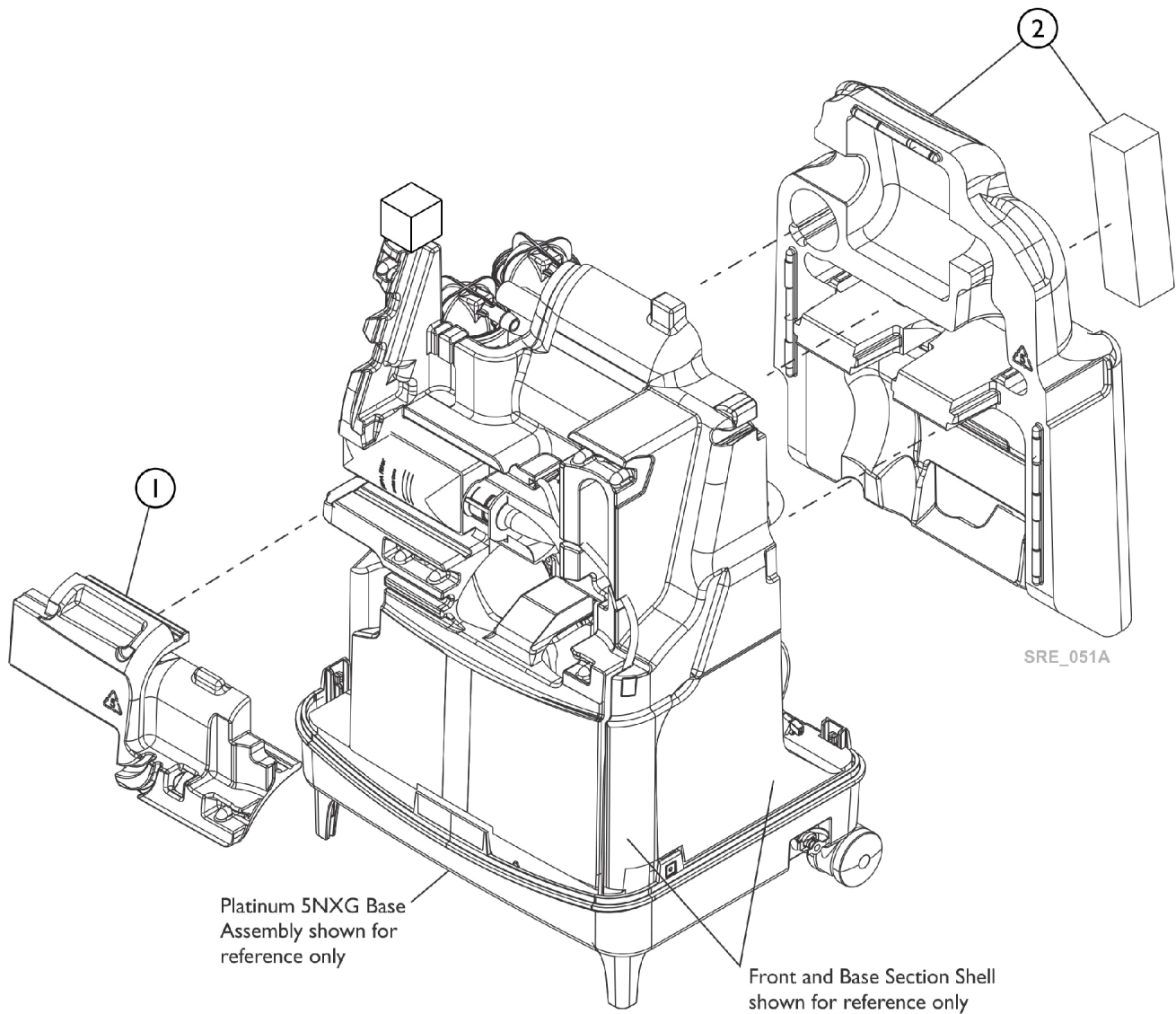


Shells, Inner Component (continued)

Inlet Filter and Access Door Section

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		60127175	Kit, Inlet Filter Shell Cover	1	1
			Cover, Inlet Filter Shell	1	1
			Sheet, Insert Parts Service	1	1
2		60127173	Kit, Service Access Door Shell Assembly	1	1
			Shell, Service Access Door	1	1
			Foam, Wall, Self-Adhesive, P5NXG (6 x 1-1/2 x 1-1/2")	1	1
			Sheet, Insert Parts Service	1	1

Base Assembly, NA

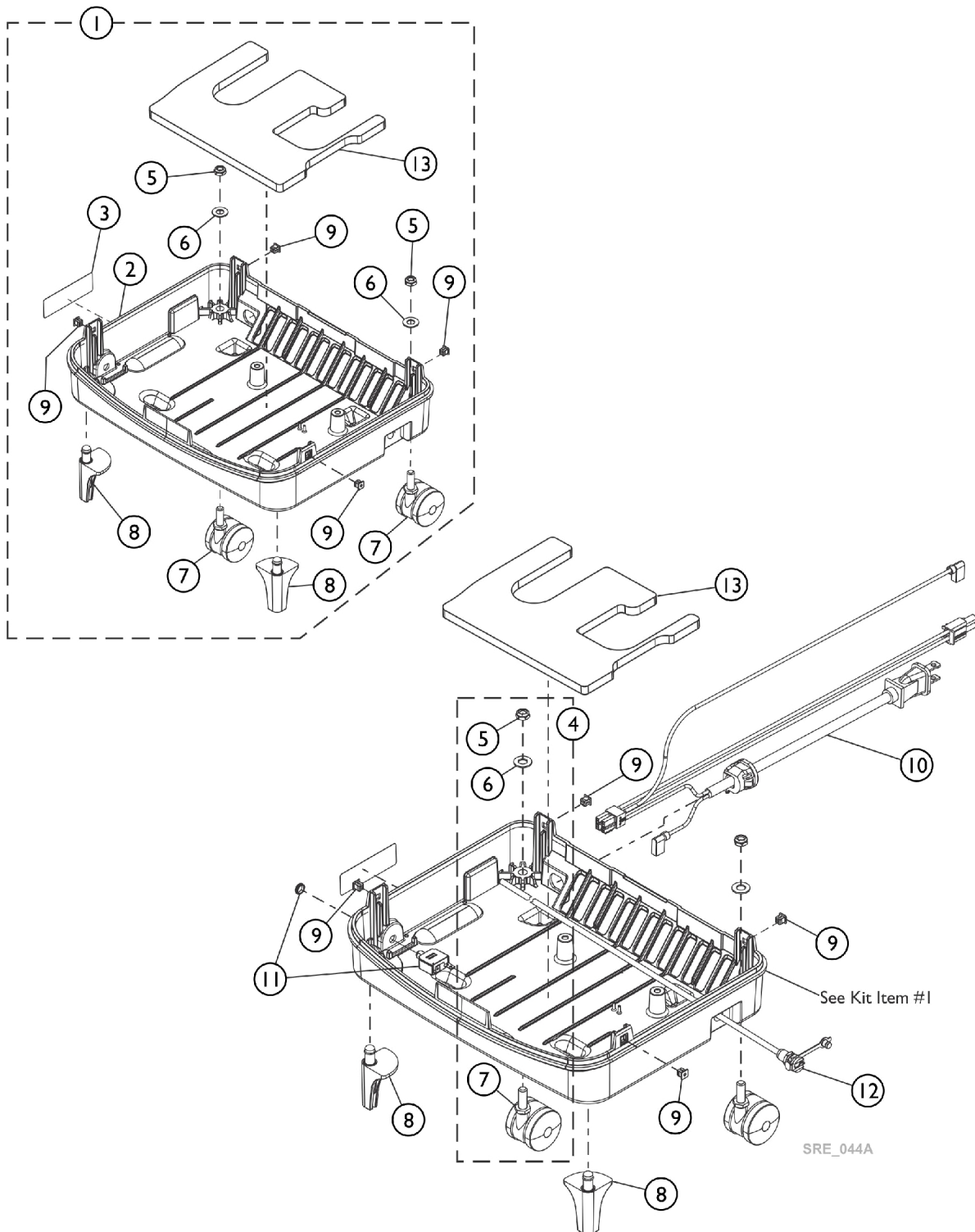


Base Assembly, NA

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	60127140	Kit, Base and Casters Assembly, Platinum 5NXG, NA	1	1
2			Base, Platinum 5NXG 120V AC, NA	1	1
3			Label, US Canada Specification	1	1
5			Locknut, Low Profile (3/8-16)	1	2
6			Washer, Nylon (3/8 x 13/16 x 1/16")	1	2
7			Caster, Swivel, (2") (3/8-16 Threaded Stem)	1	2
8			Peg, Grip Stem	1	2
9			Grommet, Plastic	1	4
13			Foam, Base, Self-Adhesive, P5NXG	1	1
			Sheet, Insert Parts Service	1	1
9		2000360	Grommet, Plastic	1	4
4	B	60127165	Kit, Caster, Swivel, (2") (3/8-16 Threaded Stem) (Pkg. of 2)	2	1
5			Locknut, Low Profile (3/8-16)	1	2
6			Washer, Nylon (3/8 x 13/16 x 1/16")	1	2
7			Caster, Swivel, (2") (3/8-16 Threaded Stem)	1	2
			Sheet, Insert Parts Service	1	1
8		60127164	Kit, Grip Stem Peg (Pkg. of 2)	2	1
			Peg, Grip Stem	1	2
			Sheet, Insert Parts Service	1	1
10		60125152	Kit, Power Cord Assembly, NA	1	1
			Power Cord Assembly, Platinum NXG	1	1
			Sheet, Insert Parts Service	1	1
11		60127155	Kit, Circuit Breaker, 6 AMP Bent Leg, NA	1	1
			Breaker, Circuit 6 AMP Bent Leg, NA	1	1
			Sheet, Insert Parts Service	1	1
12		60127142	Kit, Home Fill Port Connection, Platinum 5NXG	1	1
			Homefill Port Assembly, Platinum 5NXG	1	1
			Sheet, Insert Parts Service	1	1

NOTE: A - Includes items 2, 3, 5-9 and 13
B - Includes items 5-7


Cabinet Shroud w/ Filters and Retractable Handle Assembly

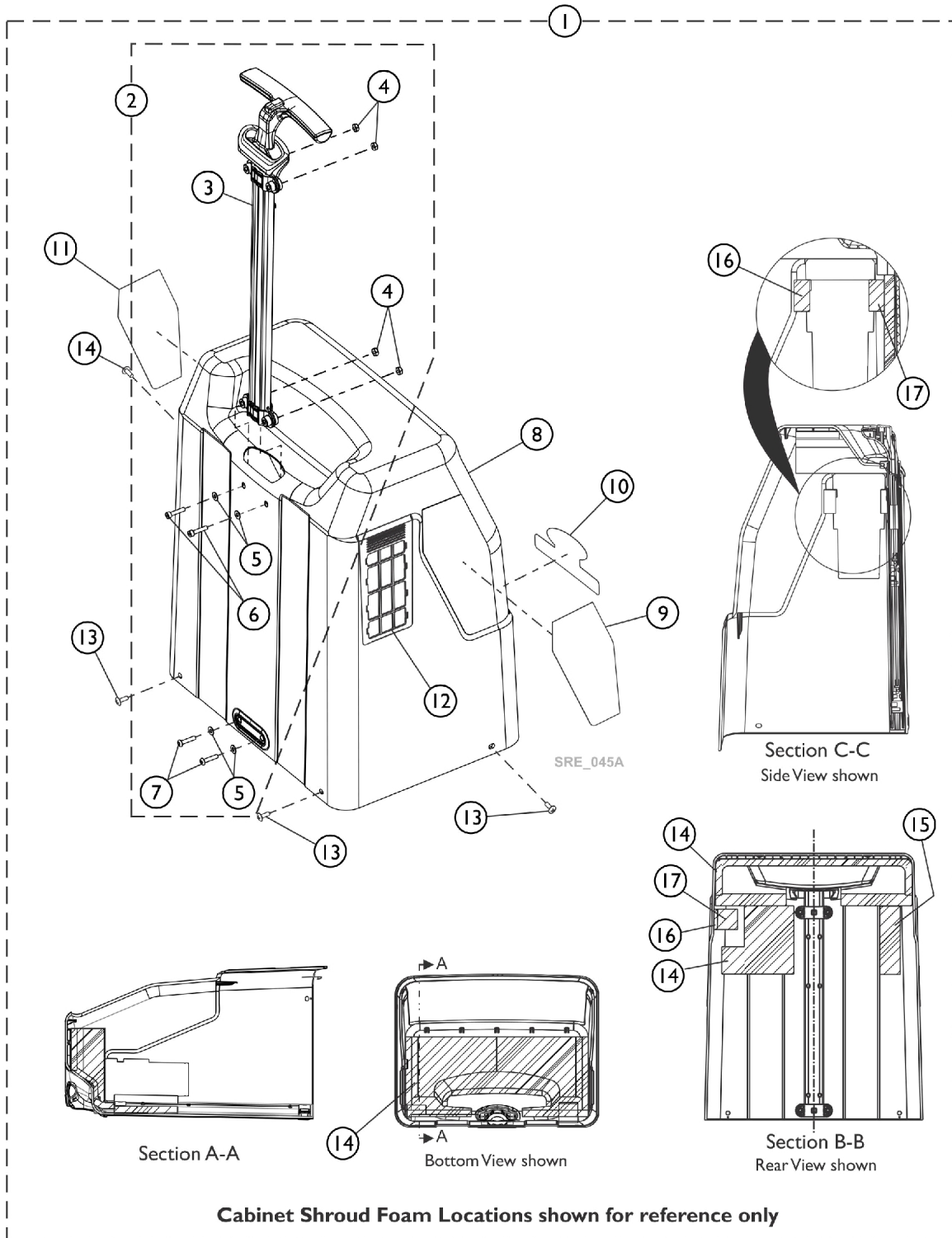

Cabinet Shroud w/ Filters and Retractable Handle Assembly

Item Number	Note	Part Number	Description	Quantity Per			
				Package	Assembly		
1	A	60127160	Kit, Cabinet Shroud and Retractable Handle Assembly, Platinum 5NXG	1	1		
2			Retractable Handle Assembly	1	1		
8			Shroud, Cabinet, Platinum 5NXG	1	1		
9			Label, QSG Platinum NXG	1	1		
10			Label, Brand LG Platinum NXG	1	1		
11			Label, Troubleshooting, Platinum NXG	1	1		
12			Gross Particle Filter Assembly, Platinum 5NXG	1	2		
13			Screw, Phillips Truss Head Self Tap (#8-32 x 5/8")	1	4		
14			Foam, Cabinet Shroud, Platinum NXG	1	1		
15			Foam, Wall, Self-Adhesive, P5NXG (6 x 1-1/2 x 1-1/2")	1	1		
16			Foam, Side	1	1		
17			Foam, Back	1	1		
			Sheet, Insert Parts Service	1	1		
2			B	60127162	Kit, Retractable Handle Assembly	1	1
3					Retractable Handle Assembly	1	1
4					Locknut (10-32)	1	4
5					Washer (#10 x 7/16 x 1/32")	1	4
6	Screw, Socket Button Head Cap (10-32 x 7/8")	1			2		
7	Screw, Socket Head Cap (10-32 x 7/8")	1			2		
	Sheet, Insert Parts Service	1			1		
12		60127161	Kit, Gross Particle Filter Assembly, Platinum 5NXG (Pkg. of 2)	2	1		
			Gross Particle Filter Assembly, Platinum 5NXG	1	2		
			Sheet, Insert Parts Service	1	1		
NOTE: A - Includes items 2, 8-17. B - Includes items 3-7.							

Control Panel and P.C. Board Assembly

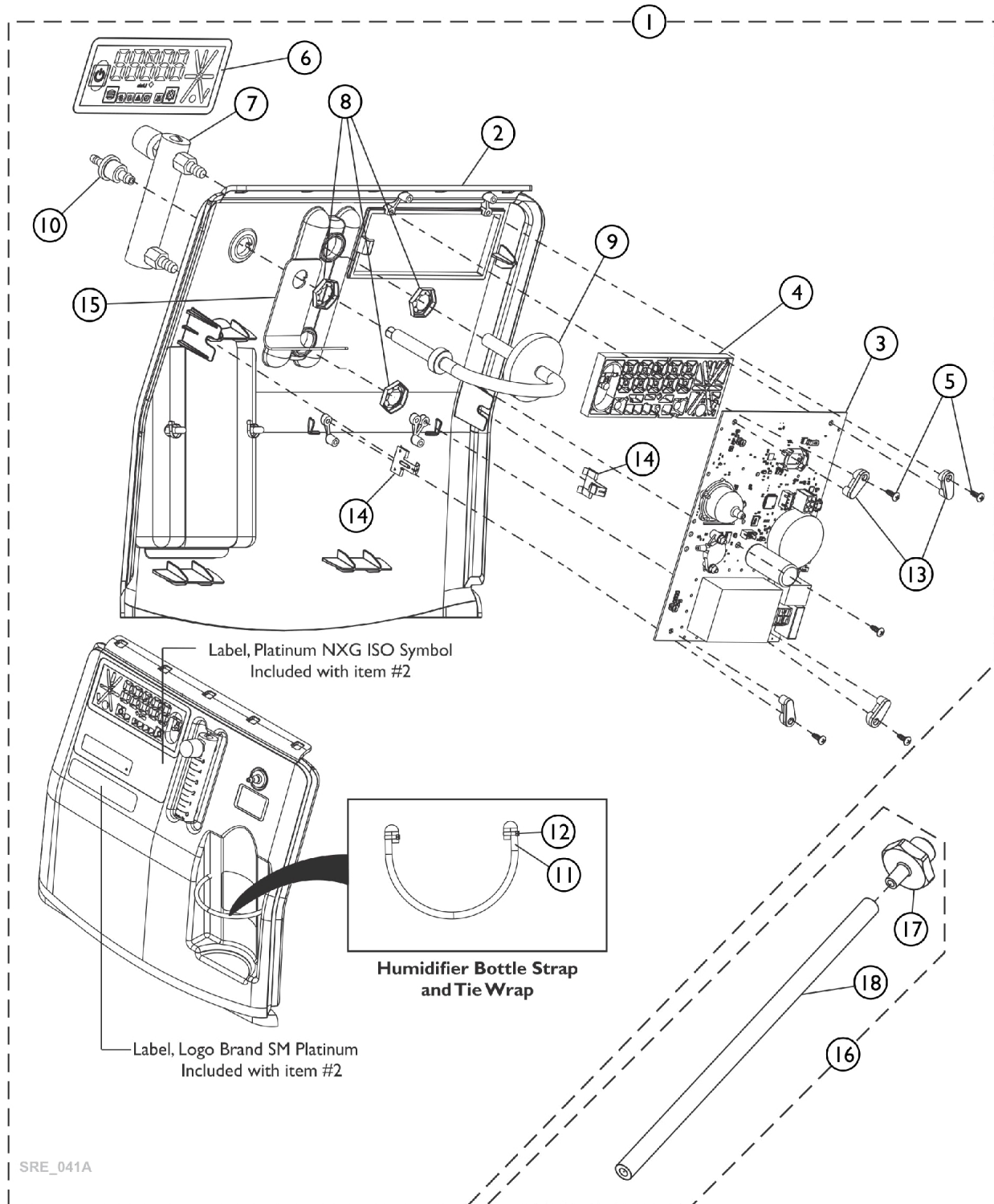


Control Panel and P.C. Board Assembly

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	60127166	Kit, Control Panel Assembly, Platinum 5NXG	1	1
2			Control Panel w/ Labels, Platinum 5NXG	1	1
3			PCB Assembly, Platinum 5NXG	1	1
4			Frame, LED Array, Platinum 5NXG	1	1
5			Screw, Phillips Round Head (#6-19 x 3/8")	1	5
6			Label, LED Overlay, Platinum 5NXG	1	1
7			Flow Meter Assembly, Platinum 5NXG	1	1
8			Palnut (9/16-18)	1	3
9			Filter Assembly, Patient Outlet	1	1
10			Connector, Barbed, Brass (1/4")	1	1
11			Strap, Humidifier Bottle, Platinum 5NXG	1	1
12			Tie Wrap (3-29/32" L)	1	2
13			Clip, PCB Mount NXG	1	4
14			Clip, Retainer, Panel Mount	1	2
15			Bracket, Support	1	1
			Sheet, Insert Parts Service	1	1
3	B	60127136	Kit, PCB Assembly, Platinum 5NXG	1	1
			PCB Assembly, Platinum 5NXG	1	1
			Sheet, Insert Parts Service	1	1
			Kit, Flowmeter w/ Palnuts	1	1
7			Flow Meter Assembly, Platinum 5NXG	1	1
8	C	60127167	Palnut (9/16-18)	1	2
			Sheet, Insert Parts Service	1	1
			Kit, Patient Outlet Port w/ Palnut (Pkg. of 10)	10	1
8			Palnut (9/16-18)	1	1
10			Connector, Barbed, Brass (1/4")	1	1
	D,E	60127144	Sheet, Insert Parts Service	1	1
9			Kit, Patient Outlet Filter Assembly	1	1
			Filter Assembly, Patient Outlet	1	1
			Sheet, Insert Parts Service	1	1
			Kit, Patient HEPA Outlet Filter	1	1
	F	60127143	Filter, Patient HEPA Outlet	1	1
			Sheet, Insert Parts Service	1	1
16			Kit, Adapter w/ Tubing, Humidifier Bottle (Pkg. of 10)	10	1
17			Adapter, Humidifier Bottle	1	1
18			Tube, Humidifier Bottle Adaptor (3/16 x 3/8 x 8-3/4")	1	1
			Sheet, Insert Parts Service	1	1
NOTE: A - Includes items 2-15 B - Includes items 7-8 C - Includes items 8 and 10 D - Not Shown E - Used with item 9 Patient Outlet Filter Assembly. F - Includes 10 ea. of items 17 and 18.					

Internal Assembly

Sieve Beds, 3-Way Valve, Capacitor, Exhaust Muffler, Heat Exchanger

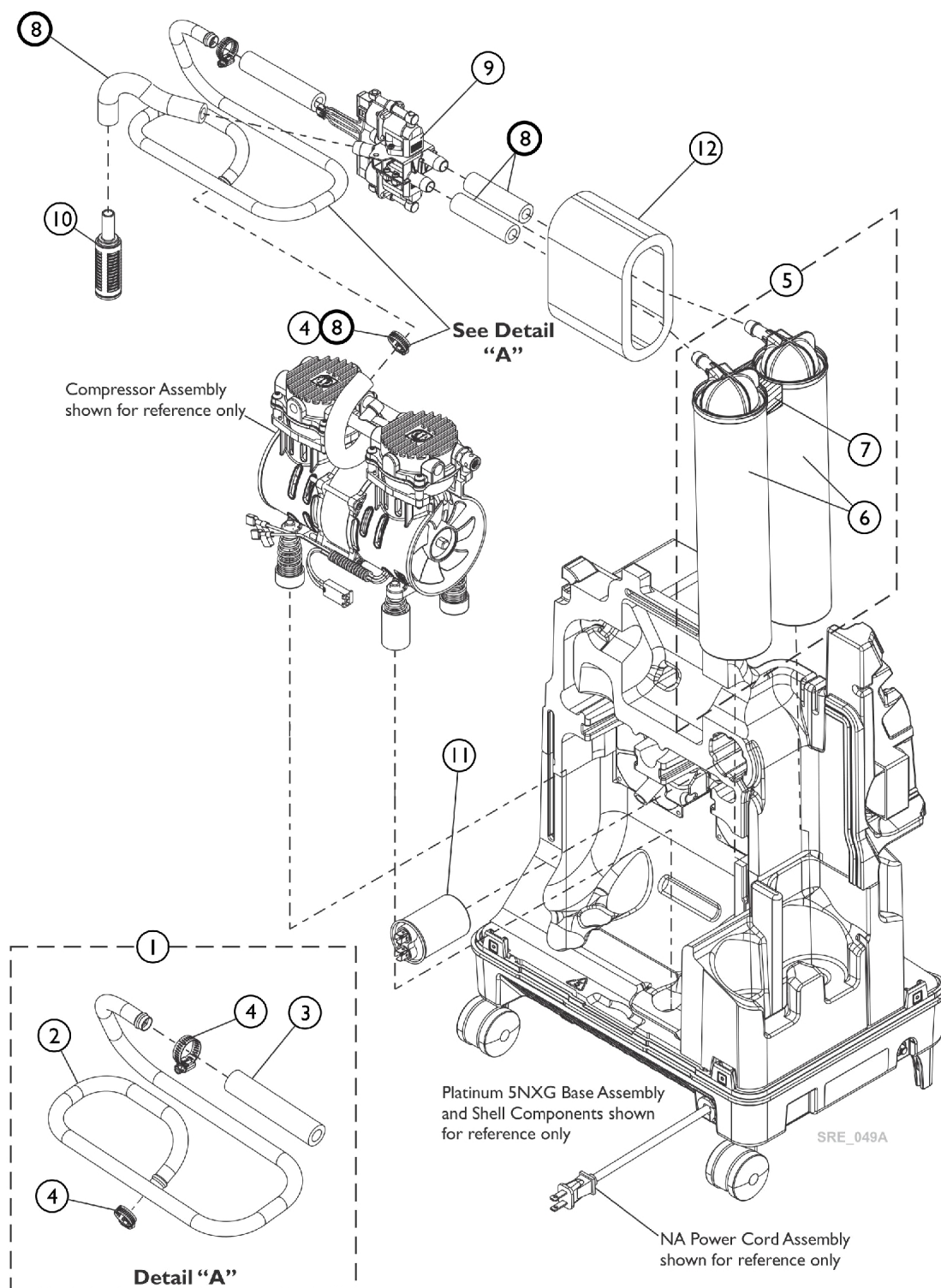


Internal Assembly

Sieve Beds, 3-Way Valve, Capacitor, Exhaust Muffler, Heat Exchanger

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	60127139	Kit, Heat Exchanger Assembly, Platinum 5NXG	1	1
2			Heat Exchanger, Platinum 5NXG	1	1
3			Tube, Silicone (7/16 x 3-29/32" Long)	1	1
4			Clamp, Hose	1	2
			Sheet, Insert Parts Service	1	1
5	B	60127137	Kit, Sieve Bed Assembly (2 ea.)	2	1
6			Sieve Bed Assembly	1	2
7			Foam, Sieve Bed	1	1
			Plug, Sieve Bed Silicone	1	2
			Cap, Sieve Bed Silicone	1	2
			Sheet, Insert Parts Service	1	1
8		60127150	Kit, 3-Way Valve Tubing	1	1
			Tube, Silicone (7/16 x 3" Long)	1	2
			Tube, Valve to Muffler	1	1
			Clamp, Hose	1	1
			Sheet, Insert Parts Service	1	1
9		60127149	Kit, 3-Way Valve Assembly, SMC	1	1
			3-Way Valve Assembly, SMC	1	1
			Sheet, Insert Parts Service	1	1
10		60127151	Kit, Exhaust Muffler	1	1
			Exhaust Muffler	1	1
			Sheet, Insert Parts Service	1	1
11	C	60127145	Kit, Capacitor, 22 MFD, NA	1	1
			Capacitor, 22MFD, NA	1	1
			Sheet, Insert Parts Service	1	1
12		60127497	Kit, Around Valve Foam, 5NXG	1	1
			Foam, Around Valve, 5NXG	1	1
			Sheet, Insert Parts Service	1	1
NOTE: A - Includes items 2-4. B - Includes items 6,7, Sieve Bed Silicone Plugs and Caps (2ea.) (not shown) C - For North America (NA) models.					

Internal Assembly

Bleed Tubing, Sieve Beds to Product Tank

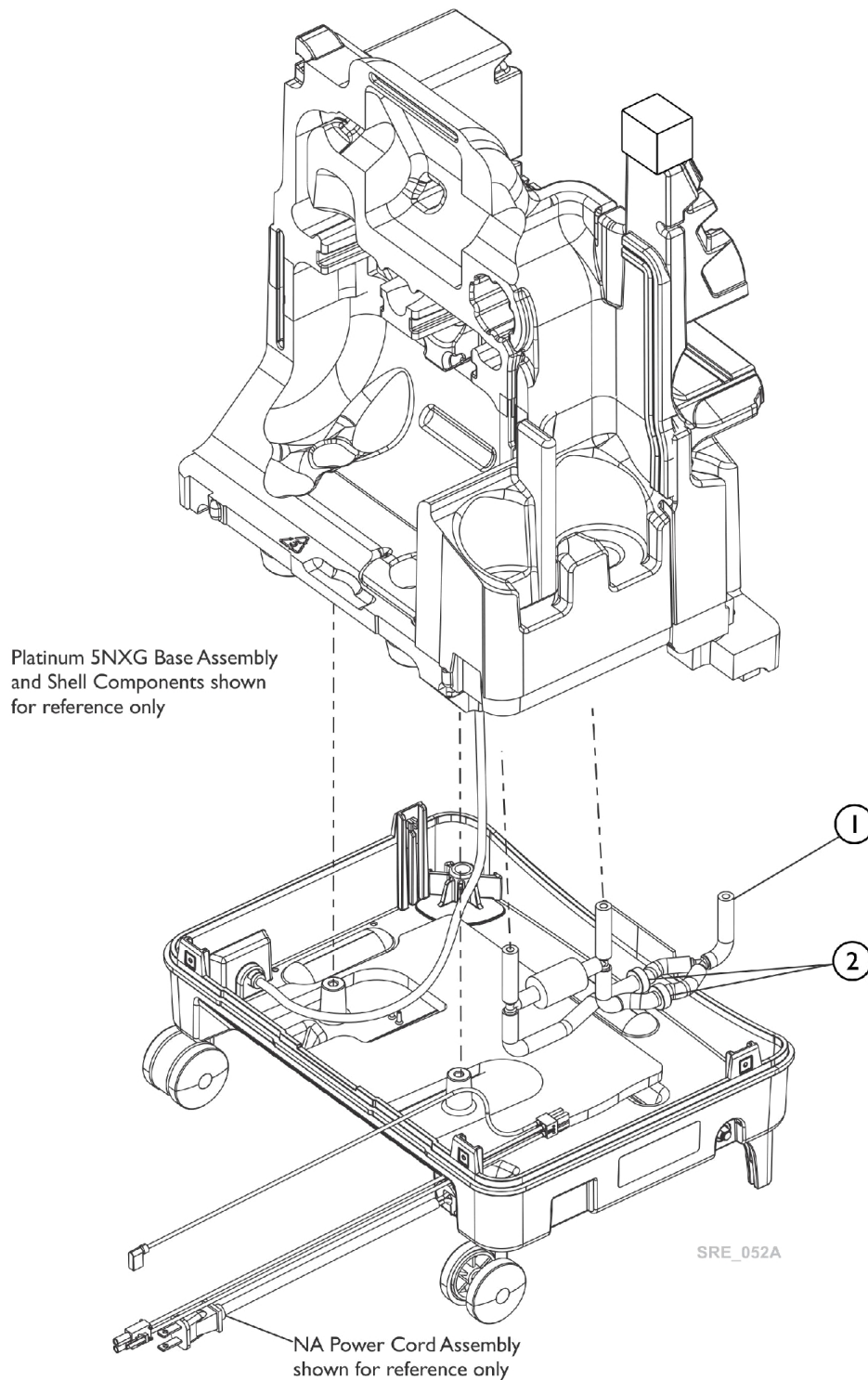


Internal Assembly



Bleed Tubing, Sieve Beds to Product Tank

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	60127138	Kit, Bleed Tubing, Sieve Beds to Product Tank	1	1
			Bleed Tubing, Sieve Beds to Product Tank	1	1
			Sheet, Insert Parts Service	1	1
2		2000436	Check Valve, Barb (5/16")	1	2
NOTE: A - Item #2 is included with kit 60127138. All other individual components are non-saleable.					

Compressor Assembly, Platinum 5NXG

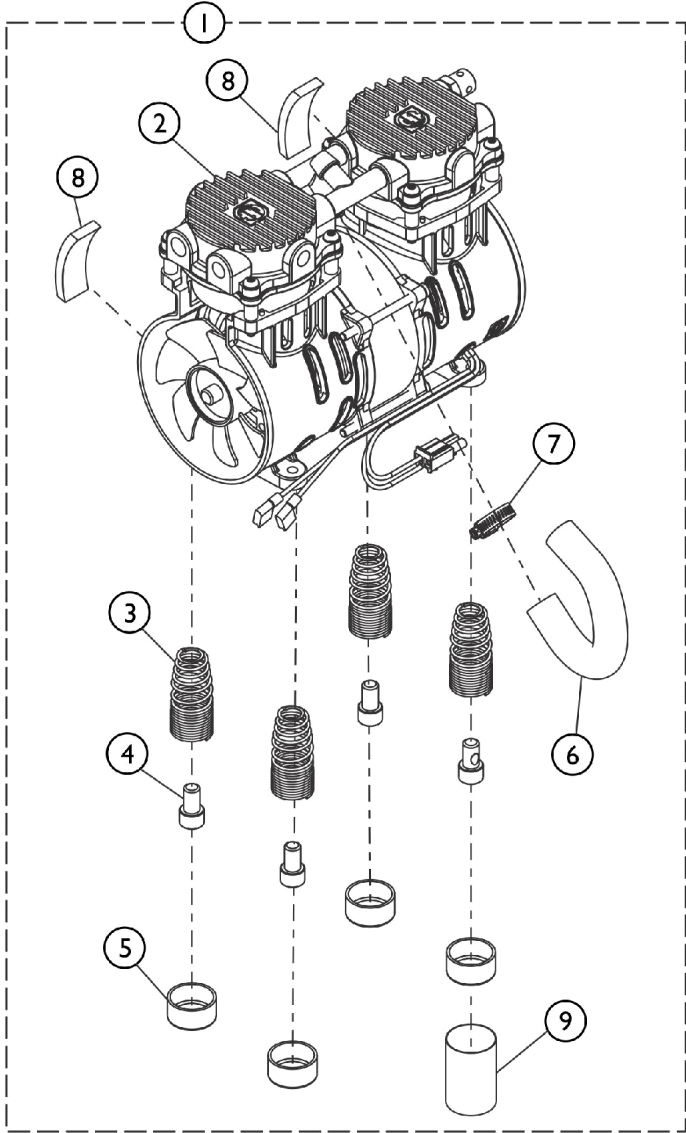

SRE_042A

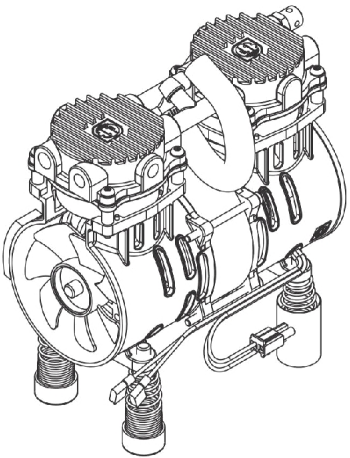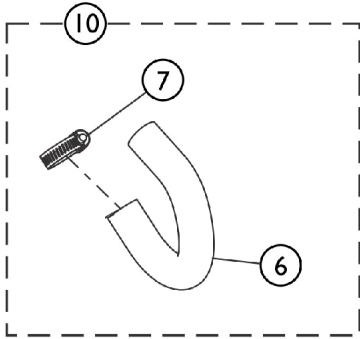


Assembled view shown
for reference only

Compressor Assembly, Platinum 5NXG

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	60127170	Kit, Compressor Assembly, Platinum 5NXG 120V AC, NA	1	1
2			Compressor - Platinum 5NXG 120V AC, NA	1	1
3			Spring, Compression, Platinum 5NXG	1	4
4			Screw, Socket Head Cap w/ Patch (3/8-16 x 5/8")	1	4
5			Cup, Compressor Spring, Platinum 5NXG	1	4
6			Tube, Molded Compressor	1	1
7			Clamp, Hose	1	1
8			Foam, Compressor	1	2
9			Insulator, Compressor Spring	1	1
	B	60127172	Sheet, Insert Parts Service	1	1
10			Kit, Tube, Heat Exchanger to Compressor	1	1
6			Tube, Molded Compressor	1	1
7			Clamp, Hose	1	1
			Sheet, Insert Parts Service	1	1

NOTE: A - Includes items 2-9. For North America (NA) models
 B - Includes items 6 and 7.

Compressor Rebuild Kit - NA Models

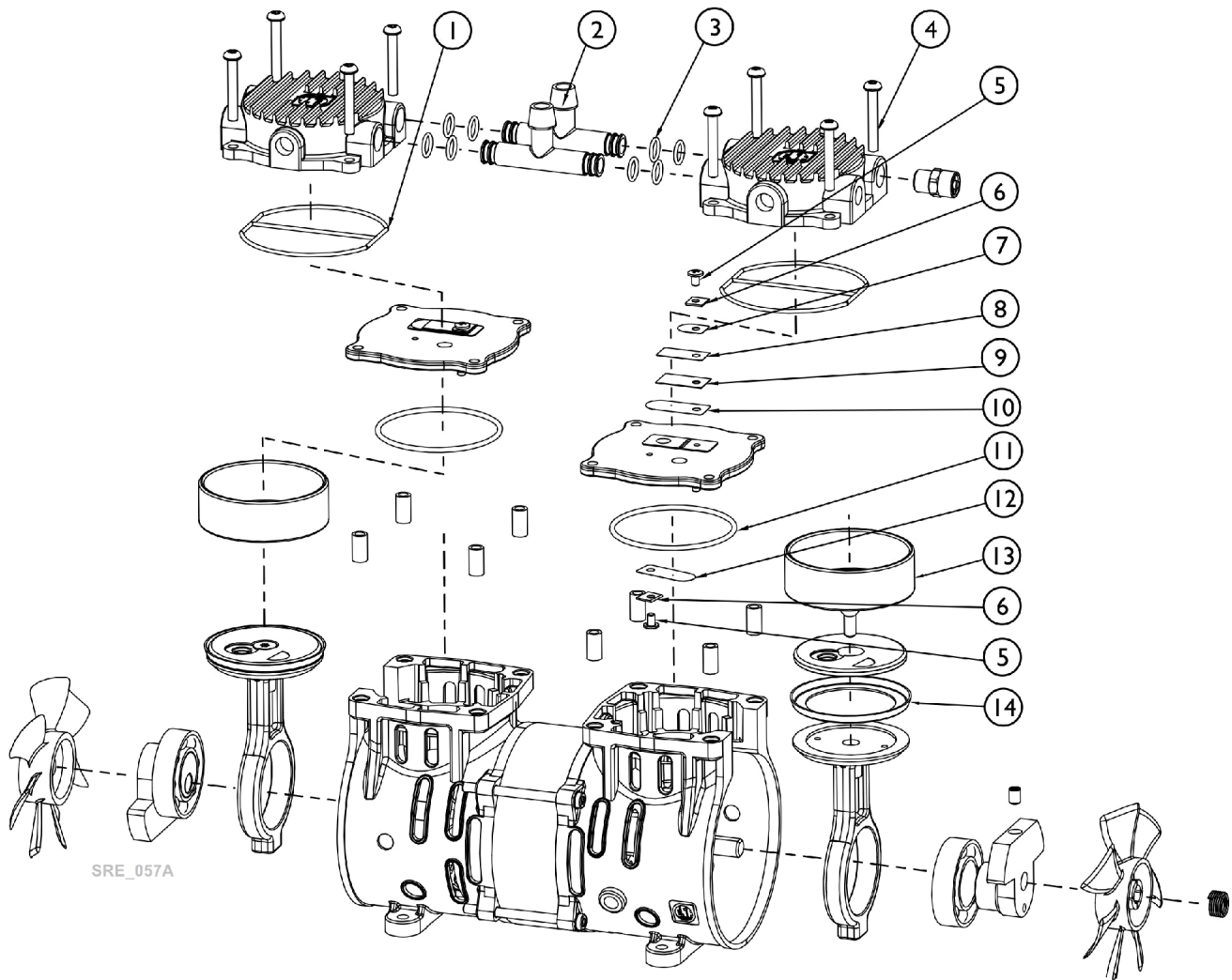


Compressor Rebuild Kit - NA Models

NOTE: Individual Kit Components are Non-Saleable. Description of components provided for reference only.

Item Number	Description	Quantity	P5NXG North America (NA) Model
1-14 See Note(s) A	Kit, Compressor Rebuild, P5NXG	1	60127500
1.	Head Seal, O-Ring	2	X
2.	Tube	2	X
3.	Tube, O-Ring	8	X
4.	Screw, Head	8	X
5.	Screw, Valve Flapper	4	X
6.	Washer, Square	4	X
7.	Valve Keeper Strip, Short	2	X
8.	Valve Keeper Strip, Long	2	X
9.	Valve Keeper Strip, Plastic	2	X
10.	Valve Flapper, Top	2	X
11.	Valve Plate, O-Ring Round	2	X
12.	Valve Flapper, Bottom	2	X
13.	Cylinder	2	X
14.	Piston Cup Seal	2	X
NOTE: A - Kit includes items 1-14.			

Fans, Compressor



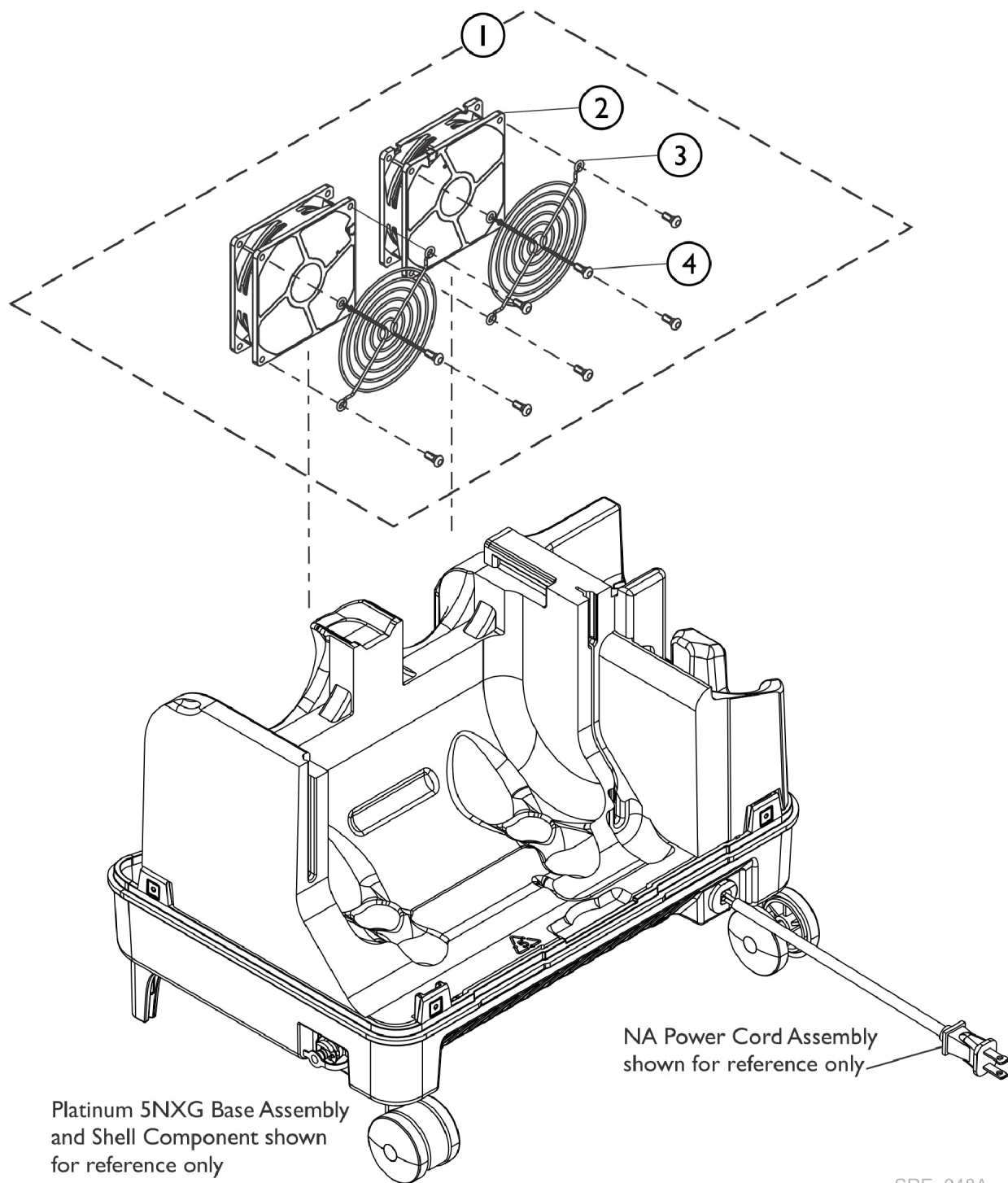


SRE_048A

Fans, Compressor

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	60127169	Kit, Dual Compressor Fans, 12VDC	1	1
2			Fan, Compressor, 12VDC	1	2
3			Guard, Fan	1	2
4			Grommet, Fan Guard	1	8
			Sheet, Insert Parts Service	1	1
NOTE: A - Includes items 2-4					


Product Tank Assembly

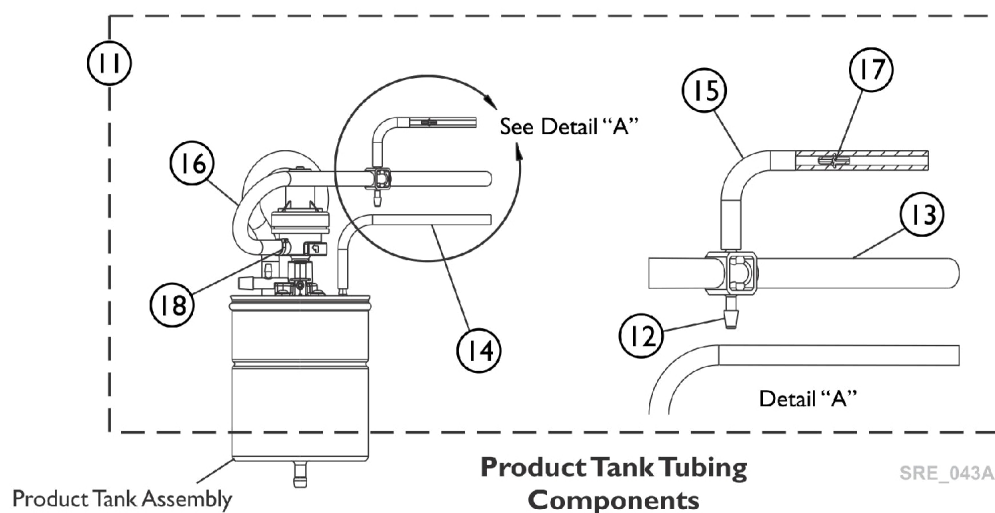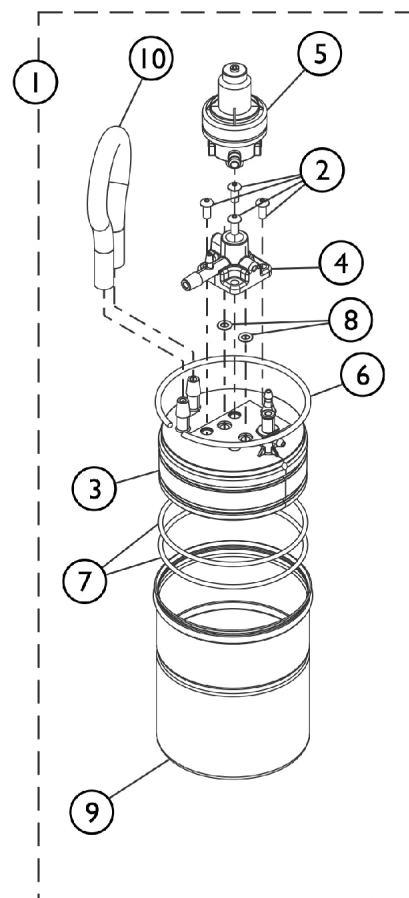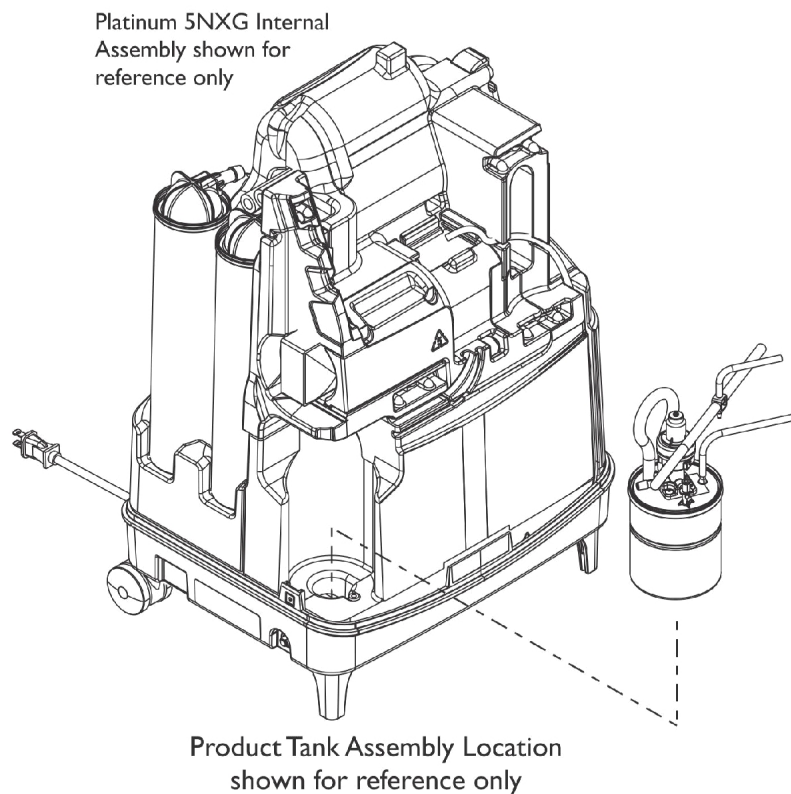

Product Tank Assembly

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	60127134	Kit, Product Tank Assembly	1	1
2			Screw, Socket Button Head Cap, Regulator (8-32 x 3/8")	1	4
3			Cap, Product Tank w/ Aux. (1/8" Outlet)	1	1
4			Adaptor, Regulator	1	1
5			Regulator (5L)	1	1
6			Ring, Snap (1/8" Dia.)	1	1
7			O-Ring	1	2
8			NLA - O-Ring (11/64" I.D.)	1	2
9			Can, Product Tank	1	1
10			Tube, Silicone (Cut 5-1/2" x 3/16" ID)	1	1
	B	60127135	Sheet, Insert Parts Service	1	1
11			Kit, Product Tank Tubing	1	1
12			Fitting, Cross (1/4 x 1/8 x 1/4 x 1/8")	1	1
13			Tube, Silicone (Cut 3-39/64" x 3/16" ID)	1	1
14			Tube, Silicone (Cut 17" x 1/8" ID)	1	1
15			Tube, Silicone (Cut 7" x 1/8" ID)	1	1
16			Tube, Silicone (Cut 13-1/2" x 3/16" ID)	1	1
17			Restrictor, White (0.005)	1	1
18			Tie Wrap (3-29/32" L)	1	1
			Sheet, Insert Parts Service	1	1
NOTE: A - Includes 2-10. B - Includes 12-18.					


Inlet Filter Assembly

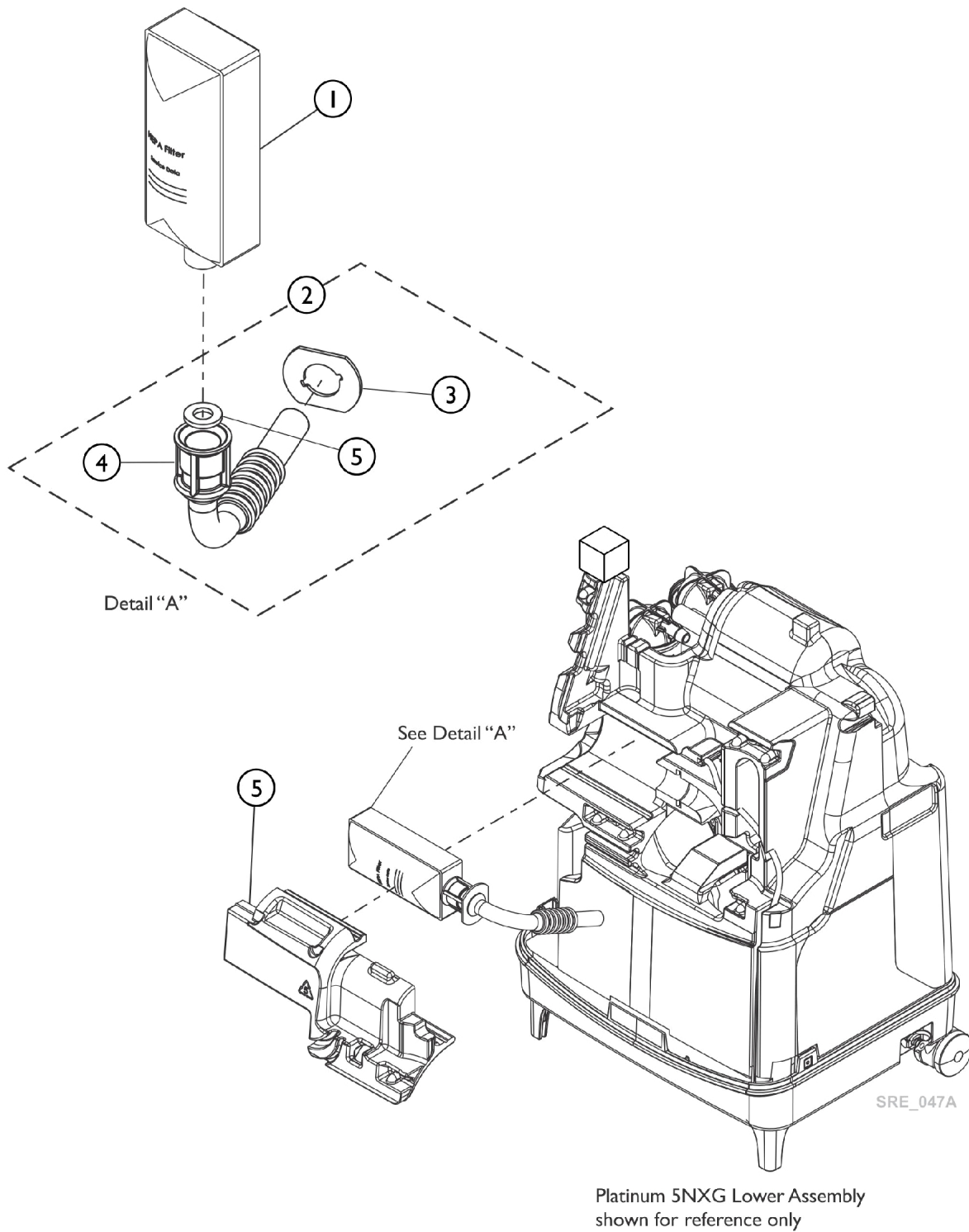

Inlet Filter Assembly

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	60127147	Kit, Inlet Sound Filter, Platinum 5NXG	1	1
			Filter, Inlet Sound Reduced	1	1
			Sheet, Insert Parts Service	1	1
2		60127148	Kit, Inlet Sound Filter Flange and Tubing, Platinum 5NXG	1	1
3			Flange, Inlet Filter, Platinum 5NXG	1	1
4			Tube, Inlet to Compressor Molded, Platinum 5NXG	1	1
5			Washer, Inlet Filter	1	1
			Sheet, Insert Parts Service	1	1
5		60127175	Kit, Inlet Filter Shell Cover	1	1
			Cover, Inlet Filter Shell	1	1
			Sheet, Insert Parts Service	1	1
NOTE: A - Includes items 3-5.					

Accessory Bag

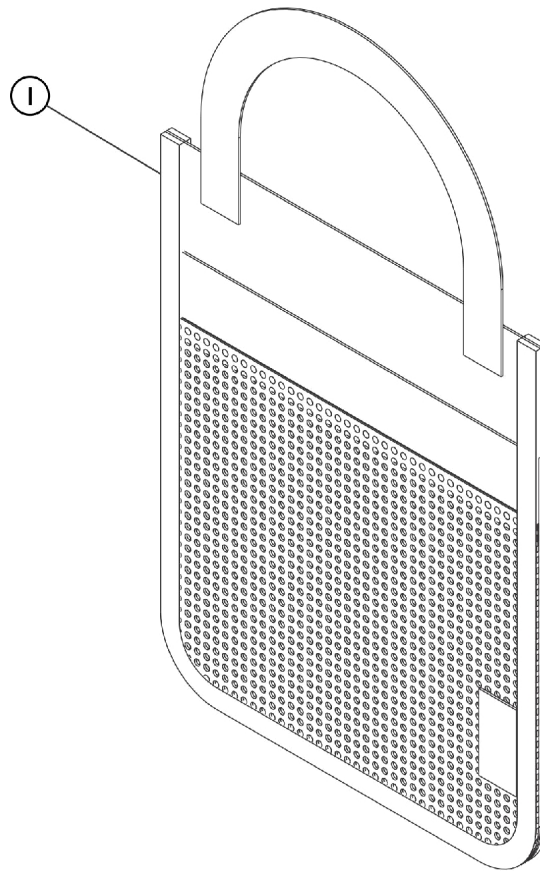

SRE 054A

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	60127483	Kit, Accessory Bag	1	1
			Bag, Accessory	1	1
			Sheet, Accessory Bag Instruction	1	1
NOTE: A - Accessory Bag attaches to the rear side of the concentrator. Instructions provided.					

Invacare Corporation

www.invacare.com



Yes, you can.

One Invacare Way
Elyria, Oh USA
44036-2125
800-333-6900

Technical Support
800-832-4707
Customer Service
800-333-6900

Literature Orders
800-333-6900

© 2021 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified as ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted. Palnut is a registered trademark of Transtechnology Corporation. Information is subject to change without notification.
(Revised 10/27/21)

Form No. 20-098