


Lateral Support Assembly Tarsys

Swingaway Laterals with Cane Clamps

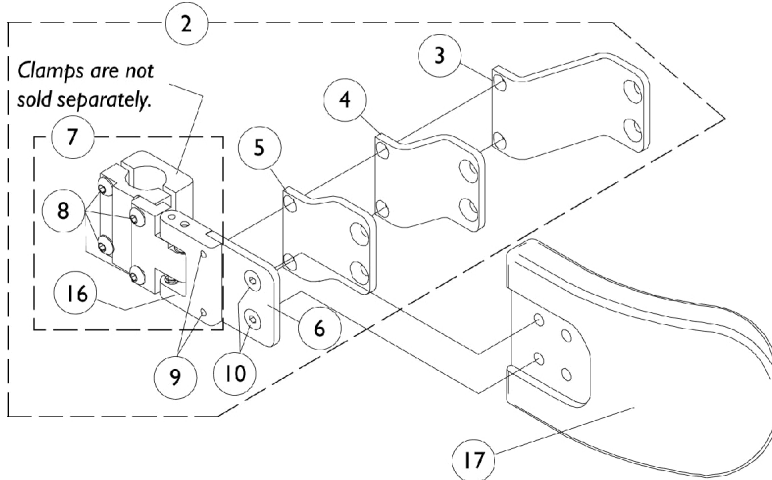


1

Swingaway Laterals with Multi-Bar (Curved Backs)

Curved A.B.S. back, mounting hook and locknut reference only.

NOTE: Block spacers and longer screws are not included with item 1. The block spacers provide clearance between the back cane and curved back for assembling the laterals to the multi-bar.

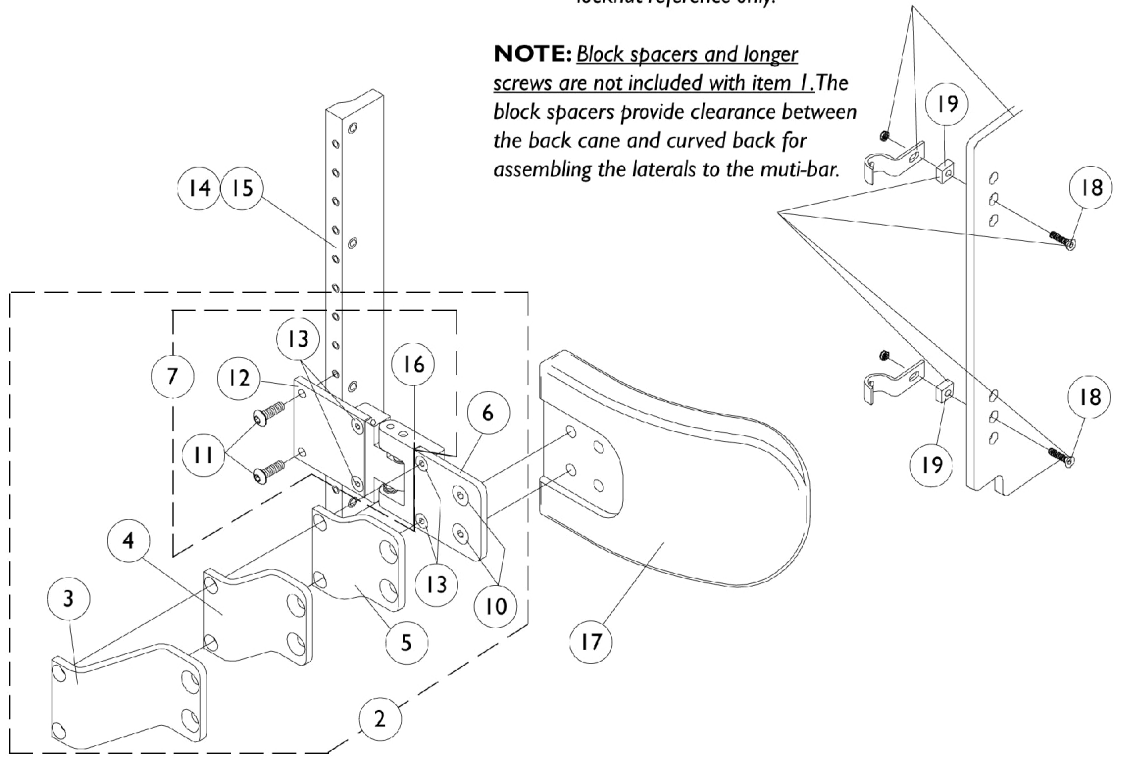

Lateral Support Assembly Tarsys

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1072661	Swingaway Lateral Support Assembly (Tarsys) (Pair) Cane Clamp Hardware with Medium Pads	2	1
1		1072662	Swingaway Lateral Support Assembly (Tarsys) (Pair) Cane Clamp Hardware with Large Pads	2	1
1	A	1072585-NLA	NLA - Swingaway Lateral Support Assembly (Tarsys) (Pair) Multi Bar Hardware with Medium Pads	2	1
1	A	1072586-NLA	NLA - Swingaway Lateral Support Assembly (Tarsys) (Pair) Multi Bar Hardware with Large Pads	2	1
2		1072660-NLA	NLA - Clamp, Cane - Right	1	1
2		1077191-NLA	NLA - Clamp, Cane - Left	1	1
2		1072588-NLA	NLA - Multi-Bar - Right	1	1
2		1072587-NLA	NLA - Multi-Bar - Left	1	1
3		1072592	Bracket, Extension Offset, Black (2")	1	1
4	B	See Note	See Footnote	1	1
5		1077186-NLA	NLA - Bracket, Extension Offset, Black (1/2")	1	1
6		1077185	Bracket, Flush Mount, Black	1	1
7		1077192	Clamp, Cane - Right	1	1
7		1077193	Clamp, Cane - Left	1	1
7		1077194-NLA	NLA - Multi-Bar - Right	1	1
7		1077195-NLA	NLA - Multi-Bar - Left	1	1
8		0051206-NLA	NLA - Screw, Machine, Hex Socket, Button Head (1/4-20 x 1-1/4)	1	4
9		1084411-NLA	NLA - Screw, Socket Flat Head (#10-32 x 1/2")	1	2
10		1102887	Screw, Socket Flat Head (1/4-20 x 1/2")	1	2
11		1072593-NLA	NLA - Screw, Allen Button Head (#10-32 x 5/8")	1	2
12		1077190-NLA	NLA - Bracket, Mounting, Black	1	1
13		1084411-NLA	NLA - Screw, Socket Flat Head (#10-32 x 1/2")	1	4
14		0063522-NLA	NLA - Bar, Multi-Attachment, A.B.S. Curved Back, 3/4" Spacing	1	1
15		0052906-NLA	NLA - Screw, Socket Button Head (1/4-20 x 3/4")	1	2
16		1096489-NLA	NLA - Bracket, Swingaway, Cane Clamp, Black - Right	1	1
16		1096490-NLA	NLA - Bracket, Swingaway, Cane Clamp, Black - Left	1	1
16		1077188	Bracket, Swingaway - Right	1	1
16		1077189-NLA	NLA - Bracket, Swingaway - Left	1	1
17		1072589	Pad, Lateral Support - Medium (6" W x 4" H)	1	1
17		1072590	Pad, Lateral Support - Large (6" W x 5" H)	1	1
18		0050806-NLA	NLA - Screw, Socket Flat Head (1/4-20 x 1-1/2)	1	4
19		0159402	Spacer, Block (1 x 3/4 x 1/2")	1	4

NOTE: A - Items 18 and 19 are not included with Swingaway lateral assemblies.
B - No longer offered as an individual service part.