
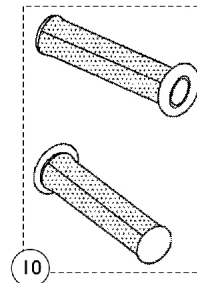
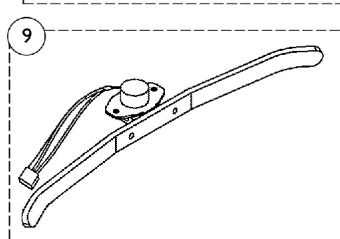
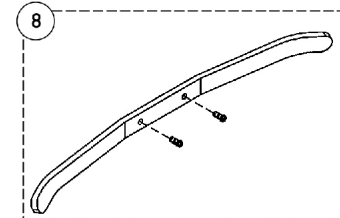
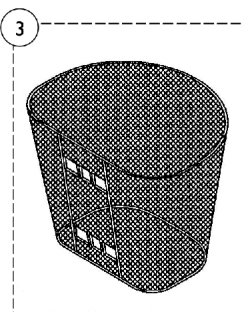
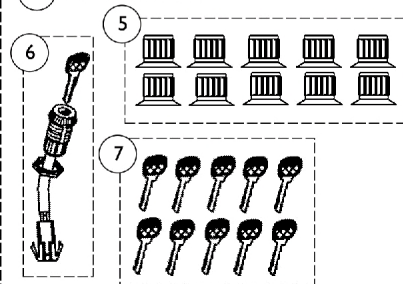
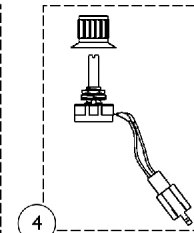
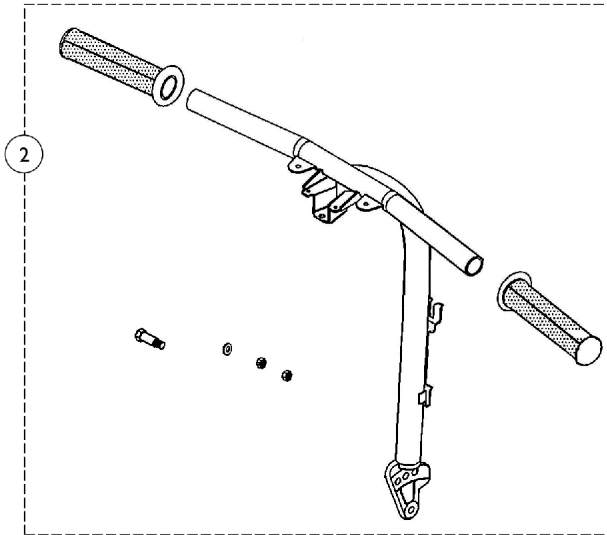


# Tiller Maintenance

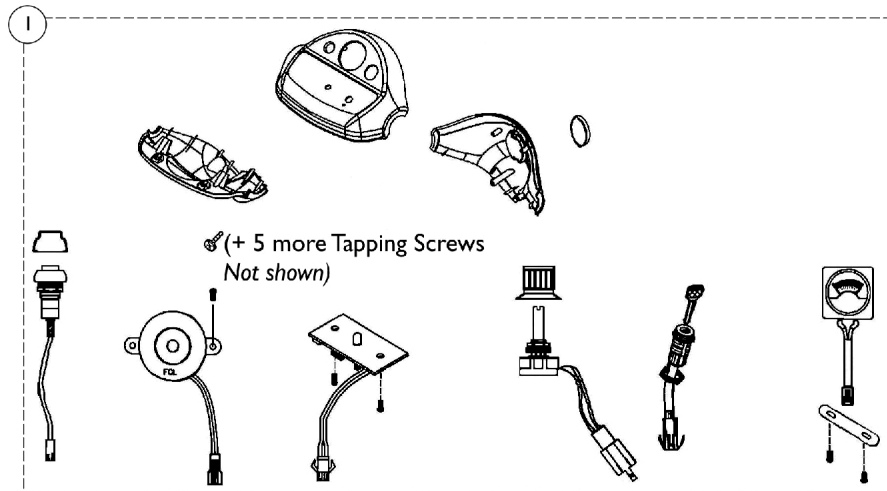

## Tiller Maintenance

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1148838	Kit, Tiller Cover, Red Includes: (1) ea. Upper Steering Cover, (1) ea. Front Steering Cover - Red, (1) ea. Rear Steering Cover - Red, (1) ea. Faceplate Decal, (1) ea. Reflector, (6) ea. Tapping Screws, (1) ea. Volt Meter, (2) ea. Screw, (1) ea. Speed Pot, (1) ea. Speed Control Knob, (1) ea. Ignition Switch w/ Key, (1) ea. Horn speaker, (2) ea. Screw, (1) ea. 2-Pole Switch Complete, (1) ea. Splash Cap Complete, (1) ea. Meter PC Board, and (2) ea. Screws.	1	1
1	A,B	1148839	Kit, Tiller Cover, Blue Includes: (1) ea. Upper Steering Cover, (1) ea. Front Steering Cover - Blue, (1) ea. Rear Steering Cover - Blue, (1) ea. Faceplate Decal, (1) ea. Reflector, (6) ea. Tapping Screws, (1) ea. Volt Meter, (2) ea. Screw, (1) ea. Speed Pot, (1) ea. Speed Control Knob, (1) ea. Ignition Switch w/ Key, (1) ea. Horn speaker, (2) ea. Screw, (1) ea. 2-Pole Switch Complete, (1) ea. Splash Cap Complete, (1) ea. Meter PC Board, and (2) ea. Screws.	1	1

# Tiller Maintenance

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	C	1169666	Kit, Tiller Cover, Onyx Includes: (1) ea. Upper Steering Cover, (1) ea. Front Steering Cover - Onyx, (1) ea. Rear Steering Cover - Onyx, (1) ea. Faceplate Decal, (1) ea. Reflector, (6) ea. Tapping Screws, (1) ea. Volt Meter, (2) ea. Screw, (1) ea. Speed Pot, (1) ea. Speed Control Knob, (1) ea. Ignition Switch w/ Key, (1) ea. Horn speaker, (2) ea. Screw, (1) ea. 2-Pole Switch Complete, (1) ea. Splash Cap Complete, (1) ea. Meter PC Board, and (2) ea. Screws.	1	1
2		1148841	Kit, Steering Bar Includes: (1) ea. Steering Bar, (2) ea. Hand Grips, (1) ea. Plain Washer, (2) ea. Nylon Washer, and (1) ea. Bolt.	1	1
3	D	1148861-NLA	NLA - Basket	1	1
3		1577430	Front Composite Basket	1	1
4	A	1144791	Kit, Speed Pot - Includes: Speed Control Knob (1) and Speed Pot - 30 K Ohms (1)	1	1
5	A	1144792	Package, Speed Control Knob	10	1
6		1144793	Ignition Switch with Key Assembly	1	1
7		1144794	Package, Ignition Key	10	1
8		1144796	Throttle Lever w/ Hardware	1	1
9		1144795	Throttle Pot and Lever Assembly	1	1
10		1144802	Hand Grip (Pair)	2	1

NOTE: A - Due to design change, the knob may be different in appearance but is functionally equivalent to the original knob.  
 B - To be used on scooters built before serial number 10BCT600382.  
 C - To be used on scooter built after serial number 10BCT600381.  
 D - Basket No Longer Available. Order part number 1577430 as a replacement.