
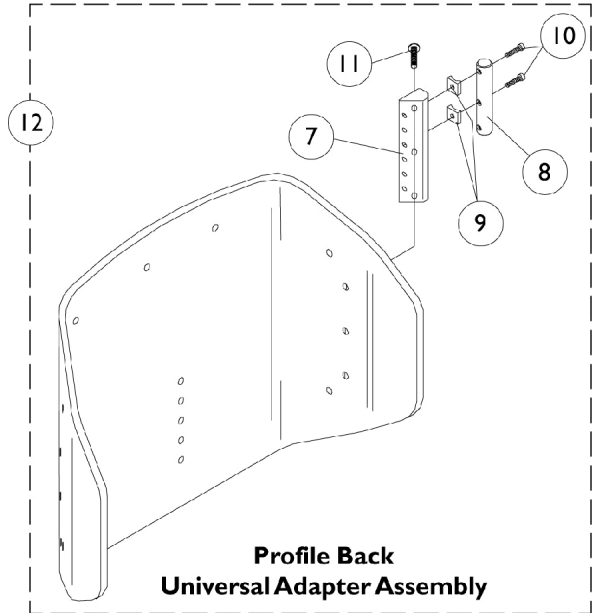


Accessories

Universal Adapters For A.B.S. Curved and Profile Backs

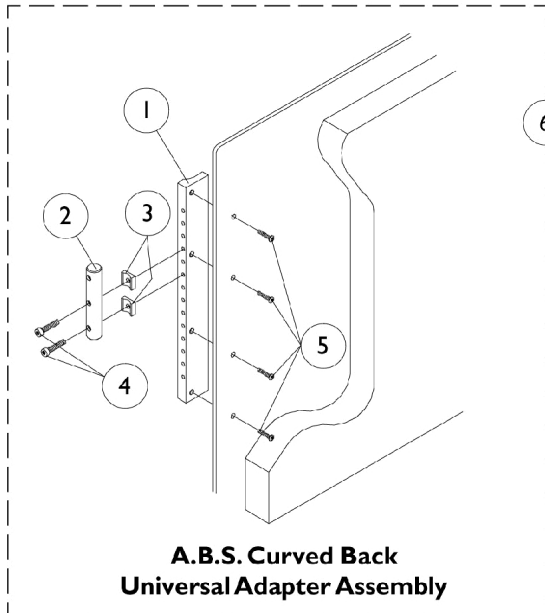

Accessories

Universal Adapters For A.B.S. Curved and Profile Backs

Universal Adapter Assembly., A.B.S. Curved Back

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	0260522-NLA	NLA - Bar, Multi-Attachment, A.B.S. Curved Back, 5/8" Spacing	1	1
1	B	0063522-NLA	NLA - Bar, Multi-Attachment, A.B.S. Curved Back, 3/4" Spacing	1	1
2		0178922	Bar, Universal Adapter Mounting	1	1
3		0049824-NLA	NLA - Spacer, Coved (1/4 x 3/4 x 7/16")	1	2
4		0258406-NLA	NLA - Screw, Socket Head, Cap (#10-32 x 1-1/4")	1	2
5		0052906-NLA	NLA - Screw, Socket Button Head (1/4-20 x 3/4")	1	4
6	C,E	0180400	Universal Adapter Assembly, ABS Curved Back, 5/8" Spacing	1	1
6	D,E	0138200-NLA	NLA - Universal Adapter Assembly, ABS Curved Back, 3/4" Spacing	1	1
	F	1072851	Bar, Multi-Attachment, A.B.S. Curved Back, 5/8" Spacing	1	1
	F	1072850	Bar, Multi-Attachment, A.B.S. Curved Back, 3/4" Spacing	1	1
	F	1084663	Universal Adapter Assembly, ABS Curved Back, 5/8" Spacing	1	1
	F	1084664	Universal Adapter Assembly, ABS Curved Back, 3/4" Spacing	1	1
NOTE: A - 5/8" Spacing (Use 1072851 For 14" Back Heights Only) B - 3/4" Spacing (Use 1072850 For 14" Back Heights Only) C - 5/8" Spacing (Use 1084663 For 14" Back Heights Only) D - 3/4" Spacing (Use 1084664 For 14" Back Heights Only) E - Includes Items 1-5 F - Not Shown					

Accessories

Universal Adapters For A.B.S. Curved and Profile Backs

Universal Adapter Assembly, Profile Back

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
7	C	0260422	Bar, Multi-Attachment	1	1
7	D	0179022	Bar, Multi-Attachment	1	1
8		0178922	Bar, Universal Adapter Mounting	1	1
9		0049824-NLA	NLA - Spacer, Coved (1/4 x 3/4 x 7/16")	1	2
10		0258406-NLA	NLA - Screw, Socket Head, Cap (#10-32 x 1-1/4")	1	2
11		1077793-NLA	NLA - Screw, Socket Head (1/4-20 x 3/4")	1	3
12	B,C	0311300	Universal Adapter Assembly, Profile Back, 5/8" Spacing	1	1
12	B,D	0180300	Universal Adapter Assembly, Profile Back, 3/4" Spacing	1	1
	A	0202400-NLA	NLA - Back Pack, Basic Tilt and Tilt Only	1	1
	A	0020200-NLA	NLA - Kit, Legrest with Screws, Otto Bock (6" W x 9" H)	1	1
	A	0020300-NLA	NLA - Adapter Plate, Otto Bock Legrest Pad	1	1
NOTE: A - Not Shown B - Includes Items 7-11 C - 5/8" spacing D - 3/4" spacing					