


# Seat Frame Assembly

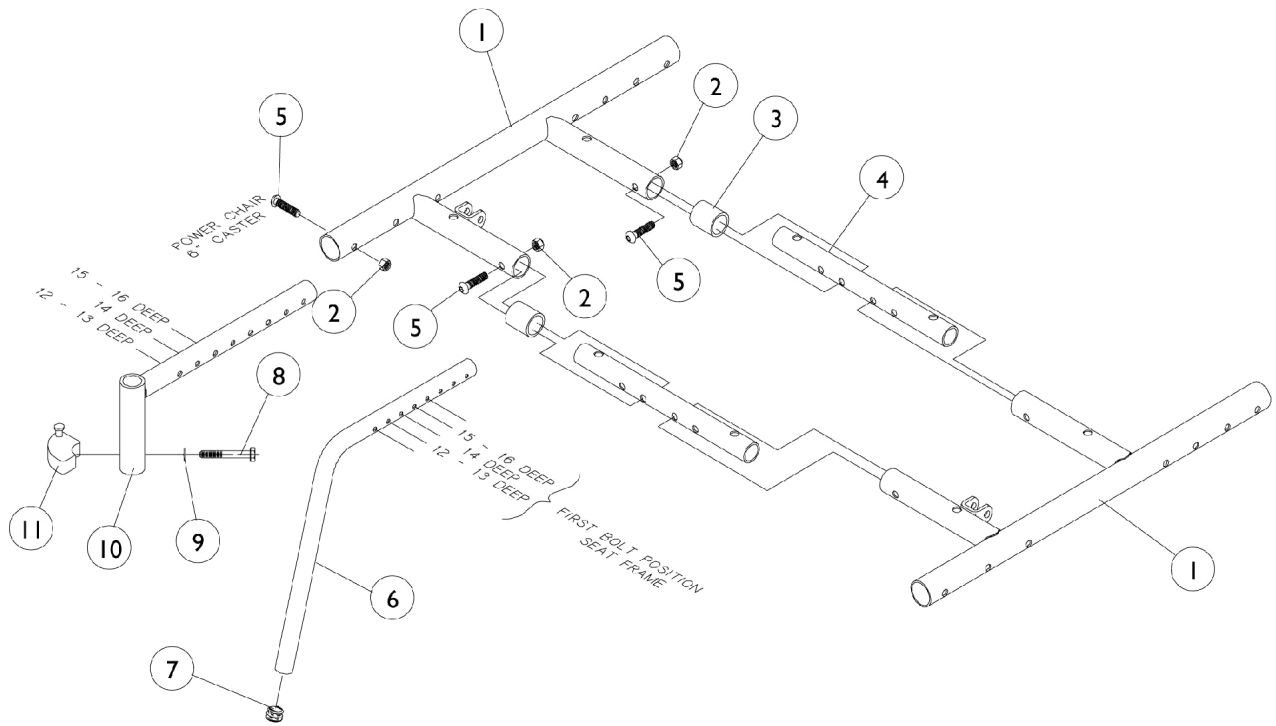

## Seat Frame Assembly

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	8881037653	Tube, Seat Rail, Color	1	2
1	A,C	8881037655	Tube, Seat Rail, Color	1	2
2		7930323-NLA	NLA - Package, Locknut (1/4-20)	25	2
3	B	1037288	Tube, Spacer	1	2
3	C	1037291-NLA	NLA - Tube, Spacer, Black	1	2
4	B	1037276-NLA	NLA - Tube, Slide, Seat Frame, Black	1	2
4	C	1037285-NLA	NLA - Tube, Slide, Seat Frame, Black	1	2
5		1008432	Screw, Socket Button Head (1/4-20 x 1-1/4")	1	2
6		8881038406-NL A	NLA - Tube, Fixed Front Frame	1	2
7		1027095	Plug Button, Black (7/8")	1	2
8		1052957	Screw, Hex Head w/ Patch (1/4-20 x 2")	1	2
9		1026907	Washer (1/4 x 1/2 x 1/16")	1	2
10		8881038407-NL A	NLA - Footrest Mounting Tube w/ Sector Block, Color - Right	1	1
10		8881038408-NL A	NLA - Footrest Mounting Tube w/ Sector Block, Color - Left	1	1
11		1054571	Kit, Sector Block - Right	1	1
11		1054572	Kit, Sector Block - Left	1	1

NOTE: A - Specify frame color when ordering.  
 B - 12-14" Wide Chairs  
 C - 15-16" Wide Chairs