


Back Canes and Stroller Handle Assembly

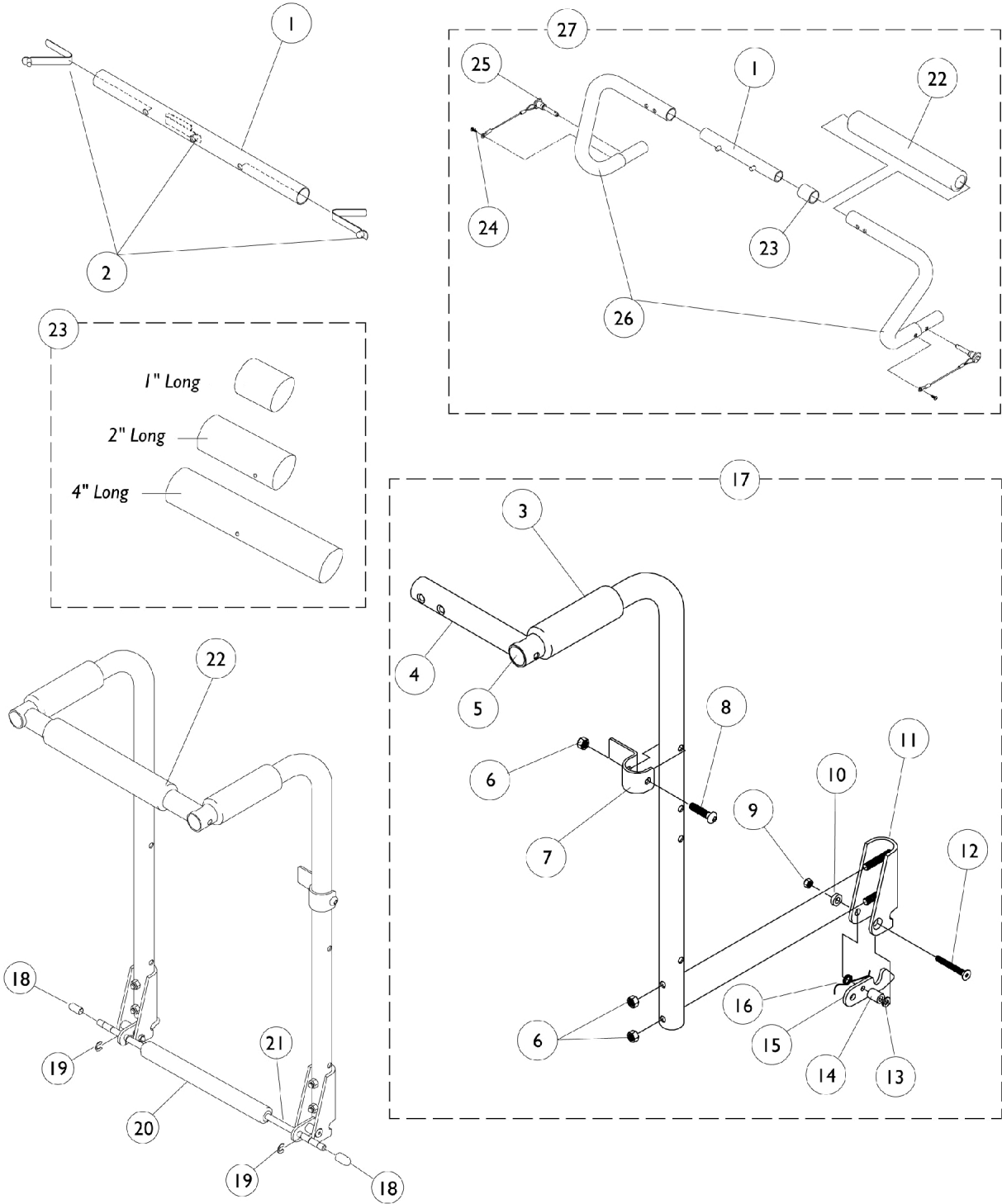

Back Canes and Stroller Handle Assembly

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1038249-NLA	NLA - Tube, Cane Slide, Black	1	1
1	B,F	1038250-NLA	NLA - Tube, Cane Slide, Black	1	1
2		40188X009	Button, Single Snap	1	1
3		1016298	Grip, Push Handle	1	2
4		1029959-NLA	NLA - Cane, Back, Black - Right	1	1
4		1029960-NLA	NLA - Cane, Back, Black - Left	1	1
5		1027095	Plug Button, Black (7/8")	1	1
6		7930323-NLA	NLA - Package, Locknut (1/4-20)	25	3
7		1025331-NLA	NLA - Bracket, Back Mounting, Black	1	1
8		1008432	Screw, Socket Button Head (1/4-20 x 1-1/4")	1	1
9		1026031-NLA	NLA - Nut, Hex with Nylon Cap (#10-32)	1	1
10		1026034-NLA	NLA - Washer, Nylon (3/16 x 7/16 x 1/8")	1	1
11		1025334-NLA	NLA - Bracket, Latch, Black - Right	1	1
11		1025335-NLA	NLA - Bracket, Latch, Black - Left	1	1
12		1025343-NLA	NLA - Screw, Flat Head (#10-32 x 1-1/2")	1	1
13		1002102-NLA	NLA - Washer, Wave (3/16 x 7/16 x 1/16")	1	2
14		1020363-NLA	NLA - Spacer (1/4 x 1/2 x 1/2")	1	1
15		1025337-NLA	NLA - Latch	1	1
16		1025325-NLA	NLA - Spring, Torsion - Right	1	1
16		1025326-NLA	NLA - Spring, Torsion - Left	1	1
17		1029923-NLA	NLA - Back Cane Assembly - Right	1	1
17		1029924-NLA	NLA - Back Cane Assembly - Left	1	1
18		1001634-NLA	NLA - Cap, End, Protective, Black	1	2
19		40314X001-NLA	NLA - Ring, Retaining	1	2
20		1025350-NLA	NLA - Grip, Foam, Bottom	1	1
21	A	1025341-NLA	NLA - Rod, Release	1	1
21	B	1029922-NLA	NLA - Rod, Release	1	1
22		1016254	Grip, Push Handle	1	1
23	C	1025359-NLA	NLA - Tube, Spacer, Black (1")	1	1
23	D	1025360-NLA	NLA - Tube, Spacer, Black (2")	1	1
23	E	1029930-NLA	NLA - Tube, Spacer, Black (4")	1	1
24		1014386-NLA	NLA - Screw, Phillips Pan Head (#8-32 x 5/8")	1	2
25		1045470	Pin, Quick Release with Lanyard	1	2
26		1038243	Handle, Stroller, Black - Right	1	1
26		1038244	Handle, Stroller, Black - Left	1	1
27	C	1038252-NLA	NLA - Stroller Handle Assembly, Black (12 - 13" W)	1	1
27	D	1038253-NLA	NLA - Stroller Handle Assembly, Black (14" W)	1	1
27	E	1038255	Stroller Handle Assembly, Black (15 - 16" W)	1	1

NOTE: A - 12-14" Wide Chairs
 B - 15-16" Wide Chairs
 C - 12"-13" Wide Chairs
 D - 14" Wide Chairs. No longer available as a service part, must order Frame Width Growth Kit (1040142) as its replacement.
 E - 15"-16" Wide Chairs
 F - No longer available as a service part, must order Frame Width Growth Kit (1040142) as its replacement.