


Seat Frame and Back Cane Hardware

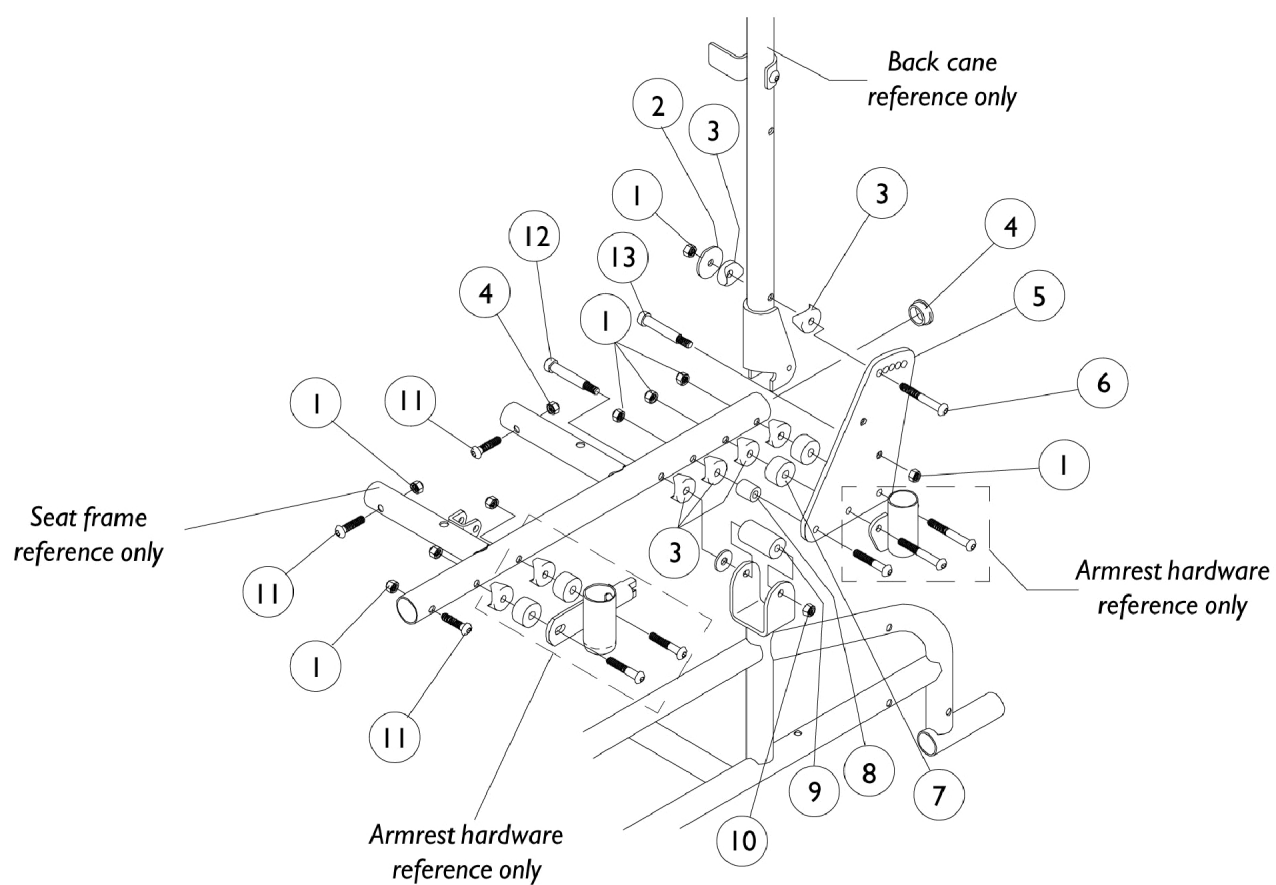

Seat Frame and Back Cane Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		7930323-NLA	NLA - Package, Locknut (1/4-20)	25	20
2		1026060-NLA	NLA - Washer, Flat (9/32 x 1 x 5/64)	1	2
3		40157X019-NLA	NLA - Spacer, Coved (1/4 x 7/8 x 7/16")	1	4
4		1027095	Plug Button, Black (7/8")	1	2
5		1027142	Plate, Back Mounting, Right	1	1
5		1027143-NLA	NLA - Plate, Back Mounting, Left	1	1
6		7930301-NLA	NLA - Package, Button Head Cap Screw (1/4-20 x 1-3/4")	25	4
7	A	1036040-NLA	NLA - Spacer (9/32 x 7/8 x 1/2")	1	4
8	A	1020363-NLA	NLA - Spacer (1/4 x 1/2 x 1/2")	1	2
9		1036394-NLA	NLA - Spacer (1/4 x 3/4 x 1-7/16")	1	2
10	B	7930333	Locknut (1/4-20)	25	2
10	C	1026035-NLA	NLA - Locknut (#10-24)	1	2
11		1008432	Screw, Socket Button Head (1/4-20 x 1-1/4")	1	6
12	B	1036041-NLA	NLA - Screw, Socket Head (1/4-20 x 3-1/4")	1	2
12	C	1030957-NLA	NLA - Screw, Shoulder Hex Socket Head (1/4 x 1-3/4", #10-24)	1	2
13		1025351-NLA	NLA - Screw, Shoulder Hex Socket Head (5/16" x 1-1/4", 1/4-20)	1	2
NOTE: A - 14" Wide B - 12 to 14" Wide C - 15 to 16" Wide					