
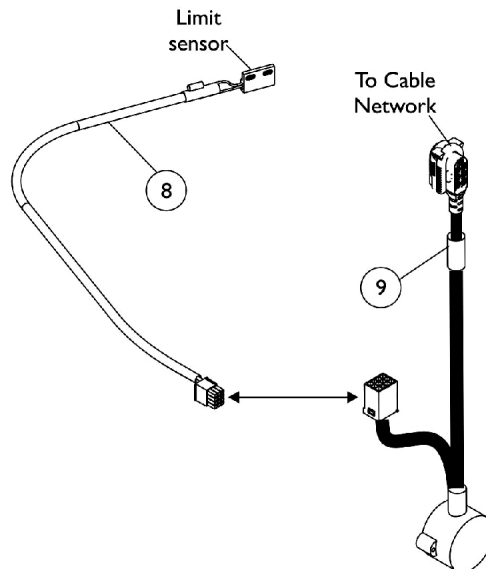
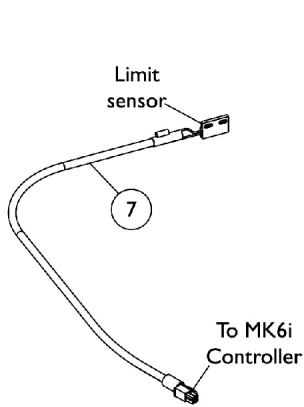
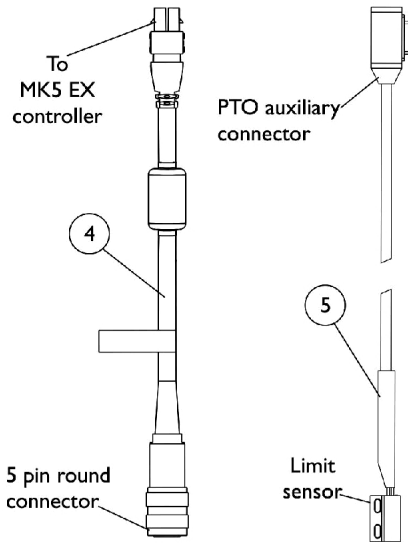
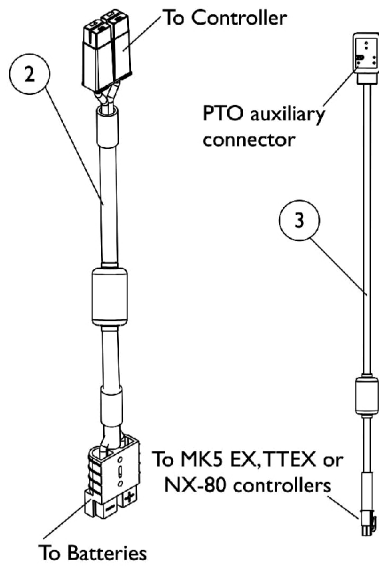
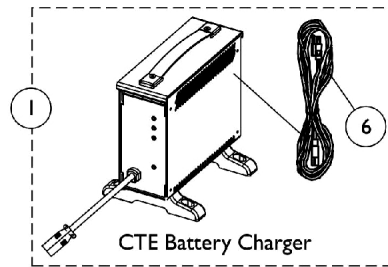


Battery Charger and Cables

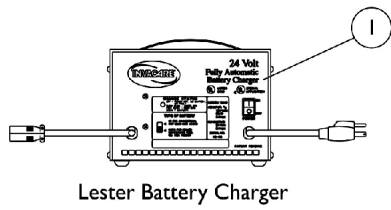

Battery Charger and Cables

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	J	1053161-NLA	NLA - Charger, Battery, 8 Amp, 115vac, Off Board Dual Mode Round Plug, Lester	1	1
1	E	1123249	Charger, Battery, 8 Amp. 115vac (Off Board) Single Mode Round Plug w/ AC Power Line Cord	1	1
1	E	1123249E-NLA	NLA - WARR RPLMT - Charger, Battery, 8 Amp, 115VAC (Off Board) Single Mode Round Plug w/ AC Power Line Cord	1	1
2	A	1116381	Cable, Battery Extension, Black Connector	1	1
3	B	1119418	Cable, Auxiliary Power	1	1
4	C	1118231-NLA	NLA - Cable, MK5 EX Controller Option	1	1
5	D	1092913-NLA	NLA - Cable, Drive Lockout Sensor	1	1
6	I	1089627-NLA	NLA - Cord, AC Power, 115VAC, 2 Prong Off-Board (11' 6" L)	1	1
6		1121086	Cord, AC Power, 115VAC, 3 Prong On-Board (6' 6-1/2" L)	1	1
7	F	1146573-NLA	NLA - Cable, Drive Lockout Sensor, MK6i	1	1
8	G	1144926	Cable, Drive Lockout Sensor, MK6i	1	1
9	H	1142231	Cable, GB TT Interface	1	1

NOTE:

- A - Used with MK5 EX Controller
- B - Used with MK5 EX, TT EX or MX-80 controllers for Powered Seating Systems.
- C - The 5 pin connector attaches to MKIV joystick options, MK5 S.A.C., MK5 T.A.C. or MK5 T.R.C.M. controllers.
- D - Cable sensor is used when a manual recliner is present
- E - CTE charger introduced and standard on-chair 4/1/04. Used on Gel Cell or AGM (Absorbed Glass Mat) batteries. Serial number starts with 04A00000001. For wet cell batteries refer to the Lester charger.
- F - Sensor Cable used for chairs manufactured with ADJRECL Manual Recline Seat Frame, MK6i Electronics and 4-Pole Motors
- G - Sensor Cable used with item 9 Interface Cable for chairs manufactured with ADJRECL Manual Recline Seat Frame, MK6i Electronics and GB TT Motors
- H - Interface Cable used with item 8 Sensor Cable for chairs manufactured with ADJRECL Manual Recline Seat Frame, MK6i Electronics and GB TT Motors
- I - No longer available as a service part. Please use 1121086 as its replacement.
- J - Use Battery Charger part number 1123249 as a replacement.