
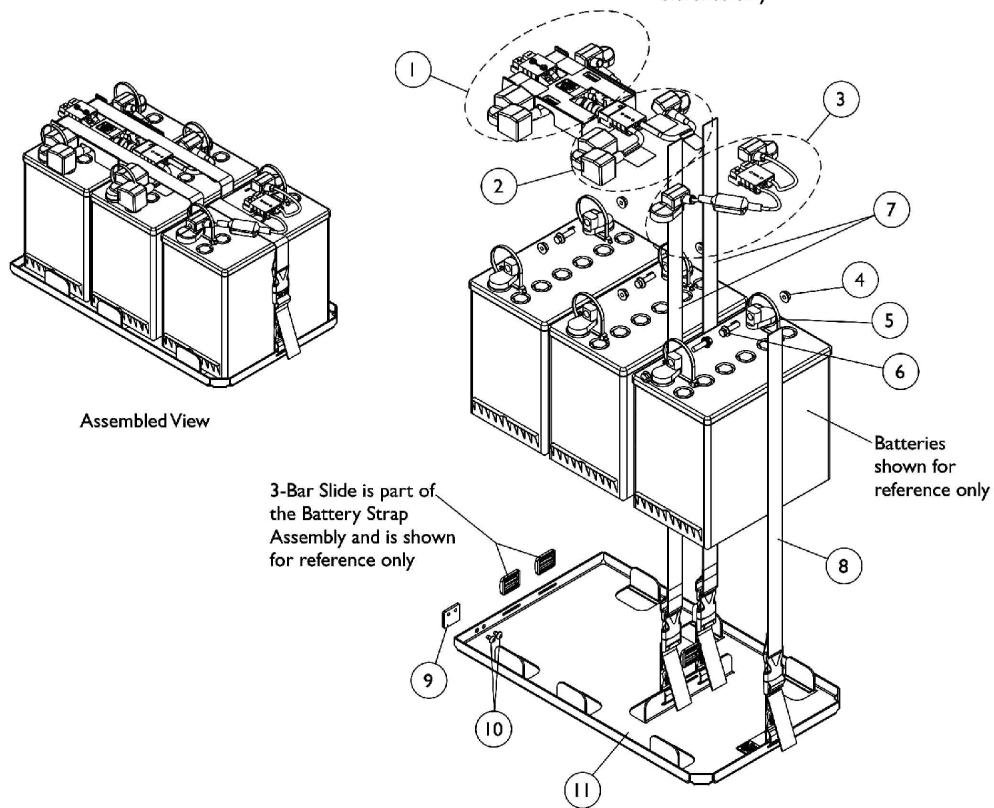


Battery Tray Assembly and Wire Harnesses For Vent Tray (3) 22NF Batteries

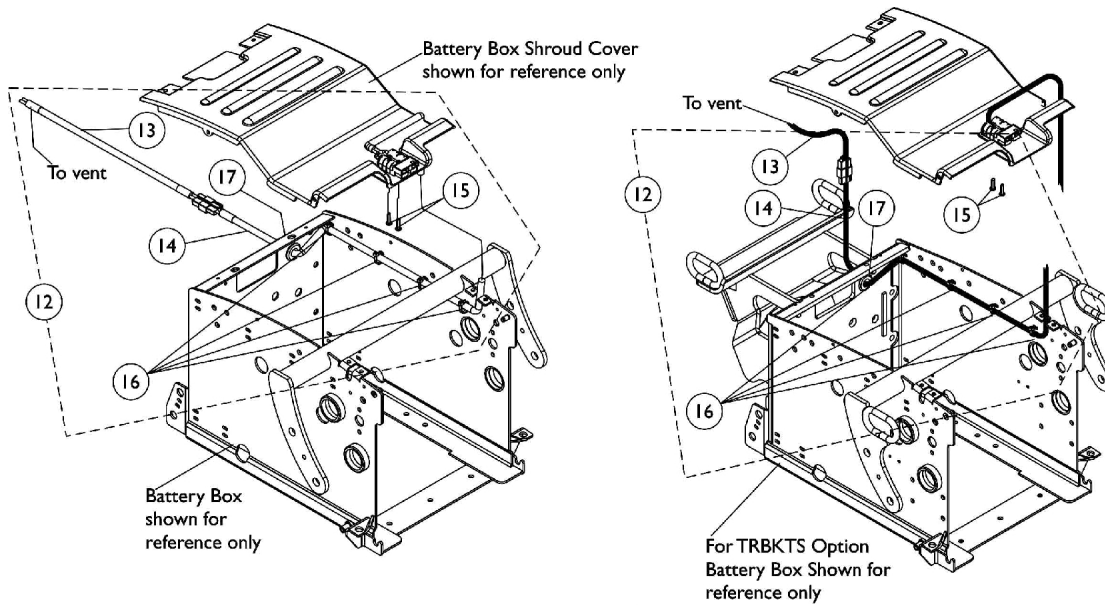

Battery Tray Assembly and Wire Harnesses For Vent Tray (3) 22NF Batteries

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	1120363	Battery Wire Harness with Terminal Hardware, 22NF - Rear	1	1
2	A,B	1120362	Battery Wire Harness with Terminal Hardware, 22NF - Front	1	1
3	A,B	1121143	Harness, Battery Wire with Terminal Hardware, 22NF Vent	1	1
4		16721X000	Locknut (1/4-20)	1	6
5		1095502-NLA	NLA - Package, Tie Wrap (11-1/2" L)	10	1
6		1096799-NLA	NLA - Screw, Hex Flange Whiz Lock (1/4-20 x 7/8")	1	4
7		1121124-NLA	NLA - Battery Strap Assembly, 22NF and GP24	1	2
8		1121125-NLA	NLA - Battery Strap Assembly For Vent, 22NF	1	1
9		1117186-NLA	NLA - Bracket, Battery Tray Stop, Black	1	1
10		90017X000-NLA	NLA - Screw, Phillips Pan Head (#10-32 x 5/16")	1	2
11	C	1121142-NLA	NLA - Battery Tray Assembly, TDX 22NF For Vent	1	1
12	D	1121029-NLA	NLA - Kit, Vent Battery Harness Assembly with Hardware	1	1
13			Harness, Vent Battery	1	1
14			NLA - Harness, Vent Battery Extension	1	1
15			Screw, Phillips Pan Head (#8-16 x 5/8")	1	2
16			Tie Wrap (5-5/8" L)	1	4
17			NLA - Grommet, Rubber	1	1
12	D	1139531-NLA	NLA - Kit, Vent Battery Harness Assembly with Hardware For TRBKTS Option	1	1
13			Harness, Vent Battery	1	1
14			Harness, Vent Battery Extension, TRBKTS Option	1	1
15			Screw, Phillips Pan Head (#8-16 x 5/8")	1	2
16			Tie Wrap (5-5/8" L)	1	4
17			NLA - Grommet, Rubber For TRBKTS Option	1	1
16		1095503-NLA	NLA - Package, Tie Wrap (5-5/8" L)	10	1
17		1055747-NLA	NLA - Grommet, Rubber	1	1
17		1137236-NLA	NLA - Grommet, Rubber For TRBKTS Option	1	1
	E	1144611	Strip, Self Adhesive Wear (2-1/2' L x 1/2" W)	1	1

NOTE: A - Individual components are not available for the battery wire harnesses
B - Includes items 4-6
C - Includes items 7-10
D - Includes items 13-17
E - Not Shown. The Self Adhesive Wear Strip is attached to the bottom side of the Battery Tray on both sides to allow removal from the Battery Box easier. One strip will accommodate both sides. Cut strip to desired length.