


Orbit Seat Option

Seat Rail, Lower Seat Plate Assembly

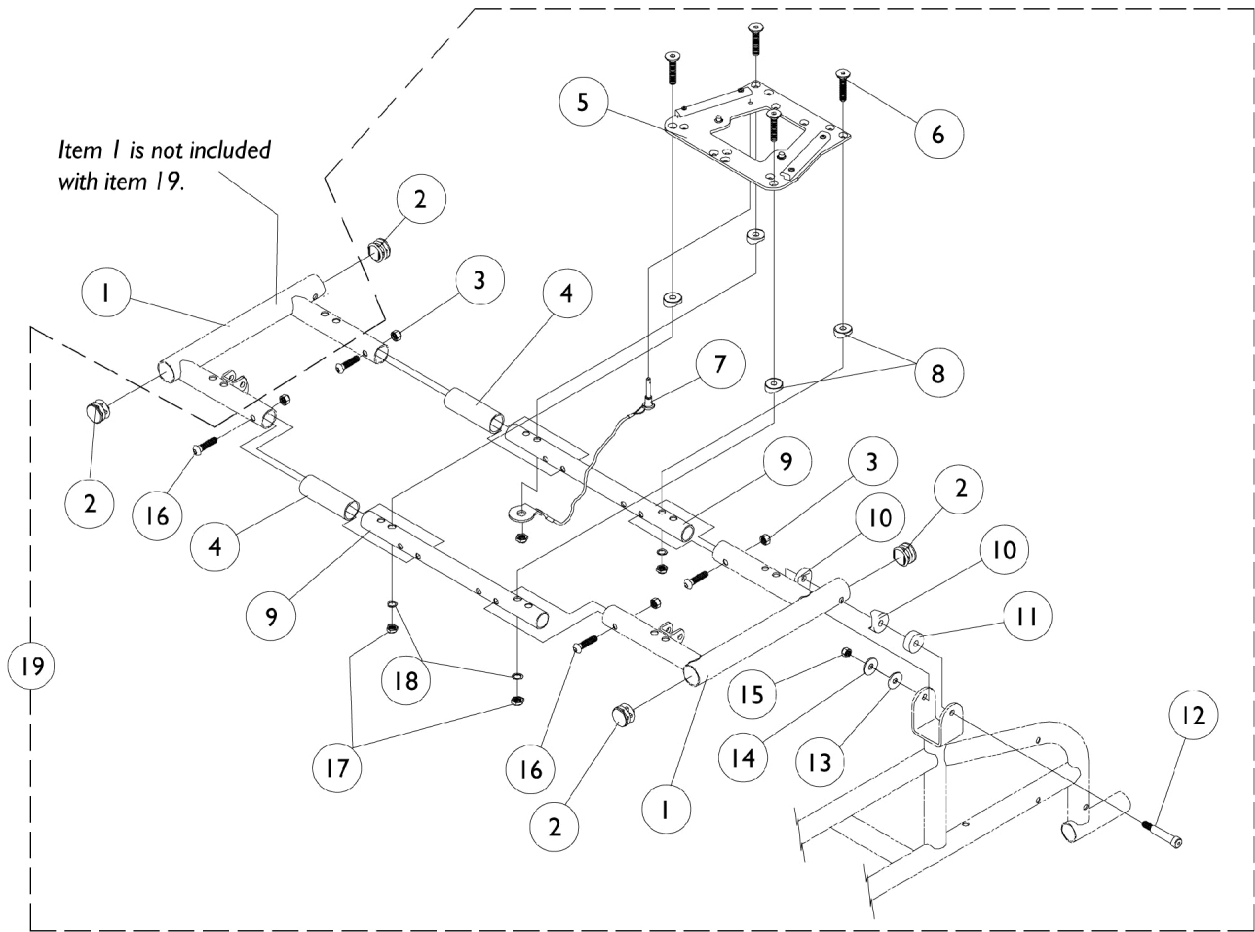

Orbit Seat Option

Seat Rail, Lower Seat Plate Assembly

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	8881083060-NL A	NLA - Frame, Seat, Rail, Color	1	2
2		1029501	Plug Button, Black (1")	1	4
3		7930323-NLA	NLA - Package, Locknut (1/4-20)	25	4
4		1037291-NLA	NLA - Tube, Spacer, Black	1	2
5		1083130-NLA	NLA - Plate, Lower Seat	1	1
6		1055332-NLA	NLA - Screw, Socket Flat Head (1/4 -28 x 1-3/4")	1	4
7		1085036-NLA	NLA - Pin and Lanyard Assembly	1	1
8		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	4
9		1037285-NLA	NLA - Tube, Slide, Seat Frame, Black	1	2
10		40157X020	Spacer, Coved (1/4 x 1 x 9/16")	1	4
11		1036434-NLA	NLA - Spacer (5/16 x 7/8 x 3/16")	1	2
12		1030957-NLA	NLA - Screw, Shoulder Hex Socket Head (1/4 x 1-3/4", #10-24)	1	2
13		1026033	Washer, Nylon (1/4 x 1/2 x 1/16")	1	2
14		1026034-NLA	NLA - Washer, Nylon (3/16 x 7/16 x 1/8")	1	2
15		1026035-NLA	NLA - Locknut (#10-24)	1	2
16		1008432	Screw, Socket Button Head (1/4-20 x 1-1/4")	1	4
17		7930345	Locknut (1/4-28)	25	4
18		1002104-NLA	NLA - Washer, Flat (1/4 x 1/2 x 1/16")	1	4
19	B,C	1083064-NLA	NLA - Hardware, Seat Mounting	1	1

NOTE: A - Specify frame color when ordering
B - When converting to an Orbit seat, item 1 must be ordered as well.
C - Includes items 2-18