


Drive Wheels and Mounting Hardware (10" x 3")

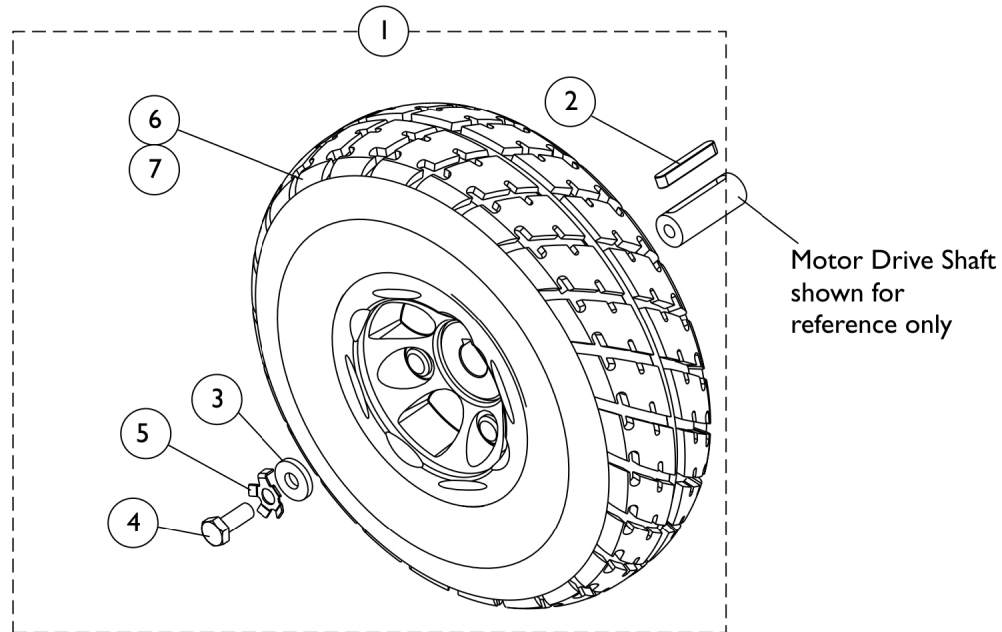

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1102339	Kit, Drive Wheel with Mounting Hardware - Flat Free (10 x 3")	1	1
2			Key (6mm x 6mm x 40mm)	1	1
3			Washer, Flat (21/64 x 53/64 x 7/64")	1	1
4			Screw (M8 x 1.25 x 20mm)	1	1
5			Kit, Locking Tab Washer w/ Instruction Sheet (Pair)	1	1
7			Drive Wheel Assembly, Flat Free Foam, Grey Tire/Black Rim (10" x 3")	1	1
5	B	1064179	Kit, Locking Tab Washer w/ Instruction Sheet (Pair)	1	1
6		1102342	Tire w/ Foam Fill Insert, Grey (10" x 3")	1	1

NOTE: A - Includes items 2-5 and 7. Part #1105043 Drive Wheel Assembly will already include the Tire w/ Foam Fill Insert.
 B - Includes 2 ea. Locking Tab Washers and Instruction Sheet