


# Orbit Seat Option

## Seat Frame Assembly and Back Canes

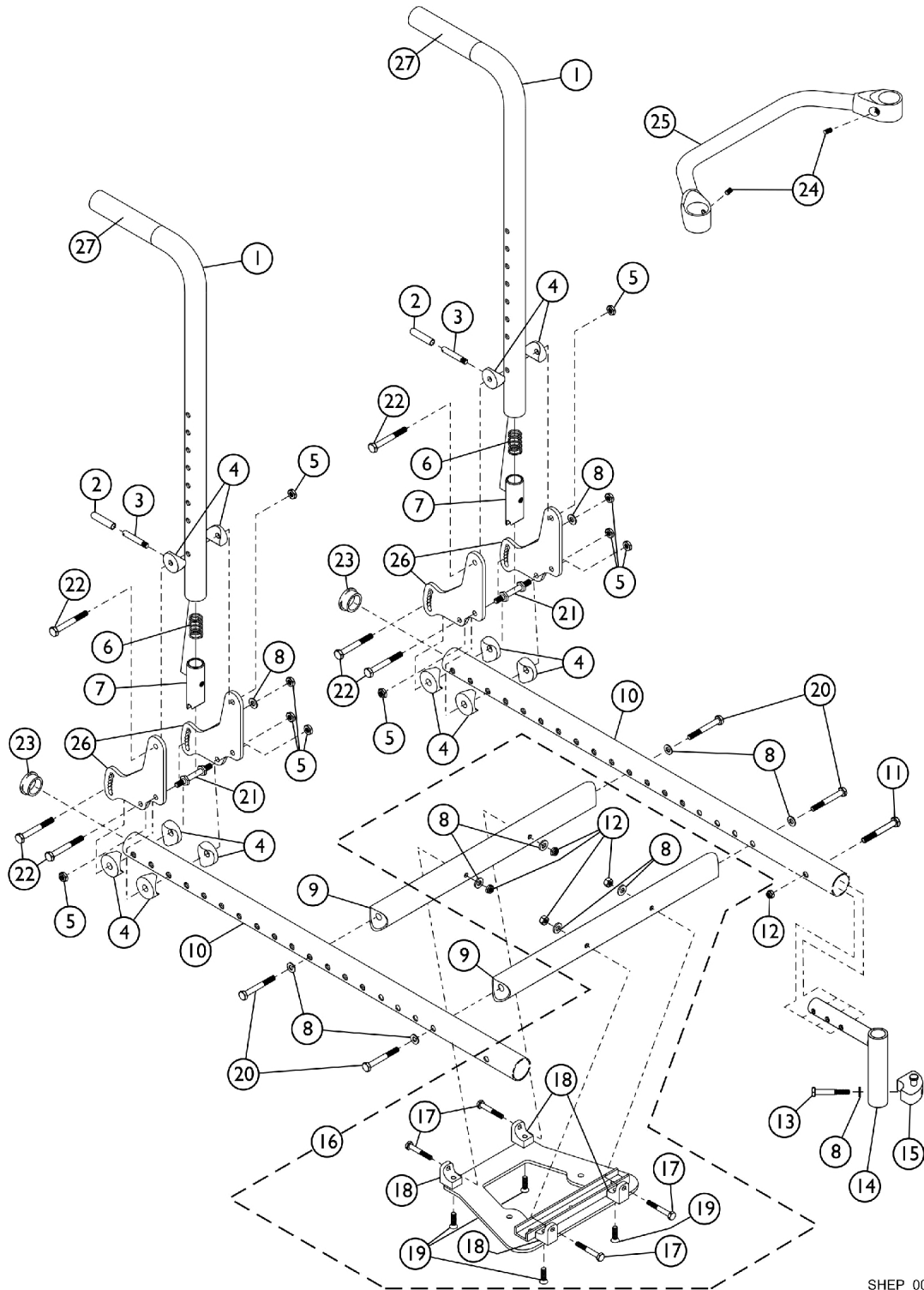

## Orbit Seat Option

### Seat Frame Assembly and Back Canes

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1059265	Cane, Back, Fixed Height, Black - Right/Left (18" H)	1	2
2		1059271	Tip, Plastic, Black	1	2
3		1058545	Lever, Actuator	1	2
4		40157X020	Spacer, Coved (1/4 x 1 x 9/16")	1	12
5		1065362	Locknut (1/4-28)	1	10
6		1011511	Spring, Compression (5/8 x 1 x 3/64")	1	2
7		1058232	Plug, Latch	1	2
8		1026907	Washer (1/4 x 1/2 x 1/16")	1	12
9		See Chart	Item #9 Crossmember	1	2
10		1082508	Rail, Seat, Black - Right	1	1
10		1082509	Rail, Seat, Black - Left	1	1
11		1074640	Screw, Allen Head (1/4-28 x 1-1/4")	1	2
12		1025765	Locknut (1/4-28)	1	6
13		1052957	Screw, Hex Head w/ Patch (1/4-20 x 2")	1	2
14		1086389	Tube, Support, Front Rigging Hanger, Black - Right	1	1
14		1086390	Tube, Support, Front Rigging Hanger, Black - Left	1	1
15		1054571	Kit, Sector Block - Right	1	1
15		1054572	Kit, Sector Block - Left	1	1
16	B	1073711	Upper Seat Plate Assembly and Mounting Hardware	1	1
17		1055347-NLA	NLA - Screw, Hex Head (1/4-28 x 1-1/2")	1	4
18		1071982	Bracket, L, Black	1	4
19		1074191-NLA	NLA - Screw, Socket Flat Head (1/4-28 x 1/2")	1	4
20		1055348	Screw, Hex Head (1/4-28 x 1-3/4")	1	4
21		1059127	Pin, Locking	1	2
22		1055349	Screw, Hex Head (1/4-28 x 2")	1	6
23		1027095	Plug Button, Black (7/8")	1	2
24		1045495-NLA	NLA - Screw, Set Socket (1/4-20 x 1/4")	1	2
25		See Chart	Item #25 Spreader Bar	1	1
26		1059109	Plate, Backrest Mounting	1	4
27		1061484	Grip, Handle, Foam, Black (5")	1	2

NOTE: A - Item #27 Foam Handle Grip is not included  
B - Includes the Upper Seat Plate Assembly and items 8, 12 and 17-19.

**Orbit Seat Option**  
**Seat Frame Assembly and Back Canes**

**Item #9 Crossmember**

10" Wide	11" Wide	12" Wide	13" Wide	14" Wide	15" Wide	16" Wide
1058329-NLA	1058330-NLA	1058331	1058332-NLA	1058333	1058334	1058335

**Item #25 Spreader Bar**

10" Wide	11" Wide	12" Wide	13" Wide	14" Wide	15" Wide	16" Wide
1042506-NLA	1042507-NLA	1042508-NLA	1042509-NLA	1042510	1042511	1042512