


Orbit Seat Option "T" Arm and Hardware

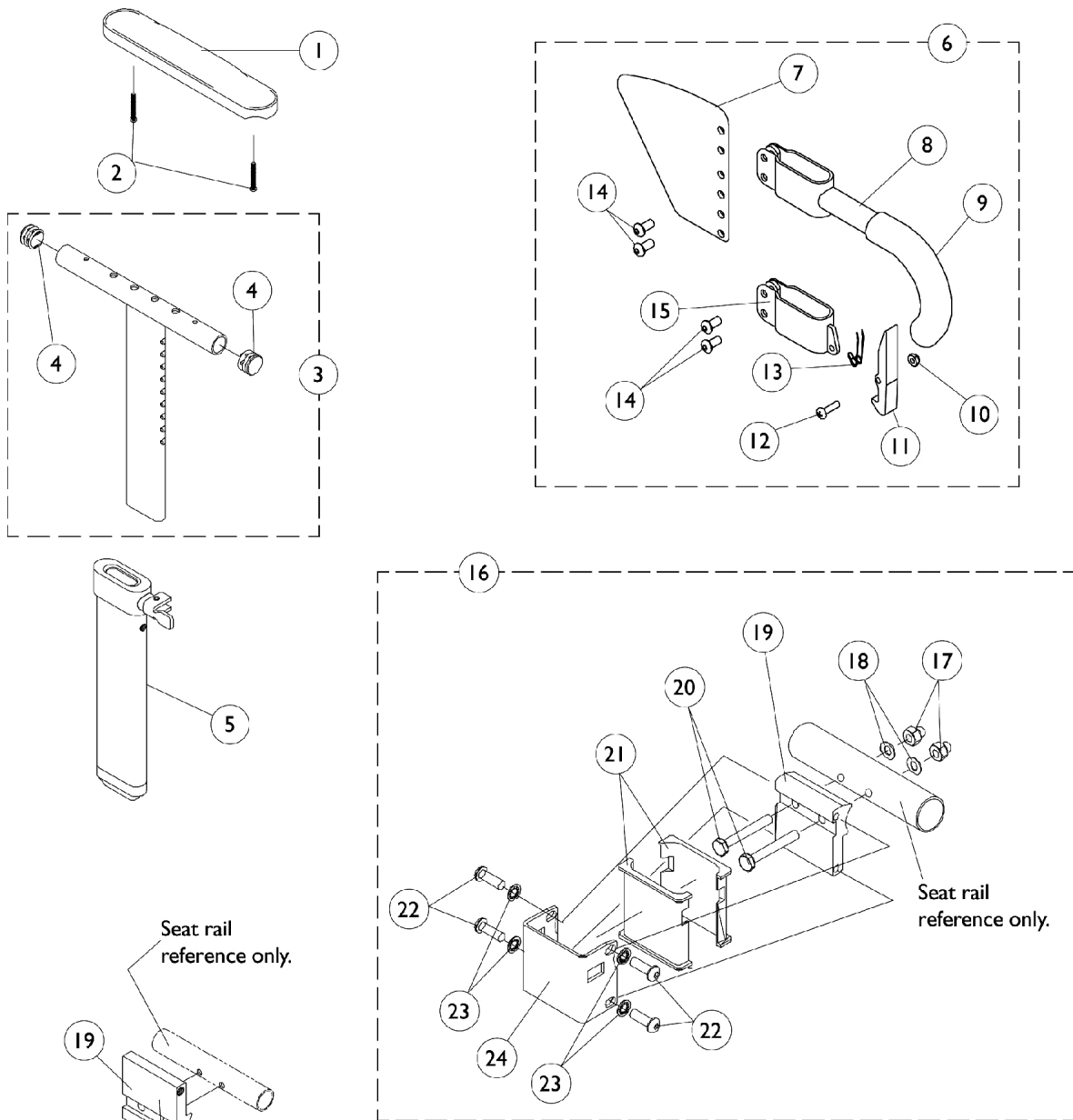

NOTE: All orbit seats that are mounted to a power tiger base will have item 19 assembled upside down to prevent wheellock interference.

Orbit Seat Option "T" Arm and Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	B	8881083213-NL A	NLA - Pad, Arm, Desk Length - Right (9" L x 2" W)	1	1
1	B	8881083214-NL A	NLA - Pad, Arm, Desk Length - Left (9" L x 2" W)	1	1
1	B	8881053570	Kit, Arm Pad w/ Screws, Desk Length (Pair)	2	1
1	B	8881083215-NL A	NLA - Pad, Arm, Full Length - Right (14" L x 2" W)	1	1
1	B	8881083216-NL A	NLA - Pad, Arm, Full Length - Left (14" L x 2" W)	1	1
1	B	8881053571	Kit, Arm Pad w/ Screws, Full Length (Pair)	2	1
2		1055205	Package, Screw Phillips Pan Head (#10-32 x 1-3/8")	2	1
3		1069639	T-Arm Assembly w/ Plug Buttons, Desk Length, Low Range, Black	1	1
3		1069643	T-Arm Assembly w/ Plug Buttons, Full Length, Low Range, Black	1	1
4		1027095	Plug Button, Black (7/8")	1	2
5		1069635	Post, Arm, Low Range	1	1
6	D	1073424	Kit, Clothing Guard, Low Range - Right	1	1
6	D	1073425	Kit, Clothing Guard, Low Range - Left	1	1
7		1069652-NLA	NLA - Guard, Clothing, Low Range	1	1
8		1069634	Tube, Support, Low Range, Black - Right	1	1
8		1073247-NLA	NLA - Tube, Support, Low Range, Black - Left	1	1
9		1028427	Grip, Foam	1	1
10		1025731	Locknut (#10-32)	1	1
11		1076839-NLA	NLA - Lever, Locking	1	1
12		1025716	Screw, Phillips Pan Head (#10-32 x 11/16")	1	1
13		1073196-NLA	NLA - Spring, Torsion	1	1
14		1006773-NLA	NLA - Screw, Socket Button Head (1/4-20 x 1/2")	1	4
15		1075069-NLA	NLA - Clamp, Clothing Guard - Right	1	1
15		1075070-NLA	NLA - Clamp, Clothing Guard - Left	1	1
16	C	1069632	Arm Socket Assembly, Bolt On	1	1
17	A	1085111	Nut, Hex (1/4-28)	1	2
18	A	1026907	Washer (1/4 x 1/2 x 1/16")	1	2
19		1069629-NLA	NLA - Bracket, T-Arm Socket, Threaded, Black	1	1
20	A	1055348	Screw, Hex Head (1/4-28 x 1-3/4")	1	2
21		1069609	Insert, Arm Socket, Black	1	2
22		1055305-NLA	NLA - Screw, Socket Button Head (1/4-28 x 3/4")	1	4
23		1026108	Lockwasher (1/4 x 15/32 x 1/64")	1	4
24		1069620-NLA	NLA - Bracket, Socket, Black	1	1

NOTE: A - Items 17, 18 and 20 must be ordered together for chairs manufactured before 8/13/99 due to a change in hardware.
B - Specify U67 upholstery color when ordering.
C - Includes items 17-24
D - Includes items 7-15