


Solid Back Mounting Hardware

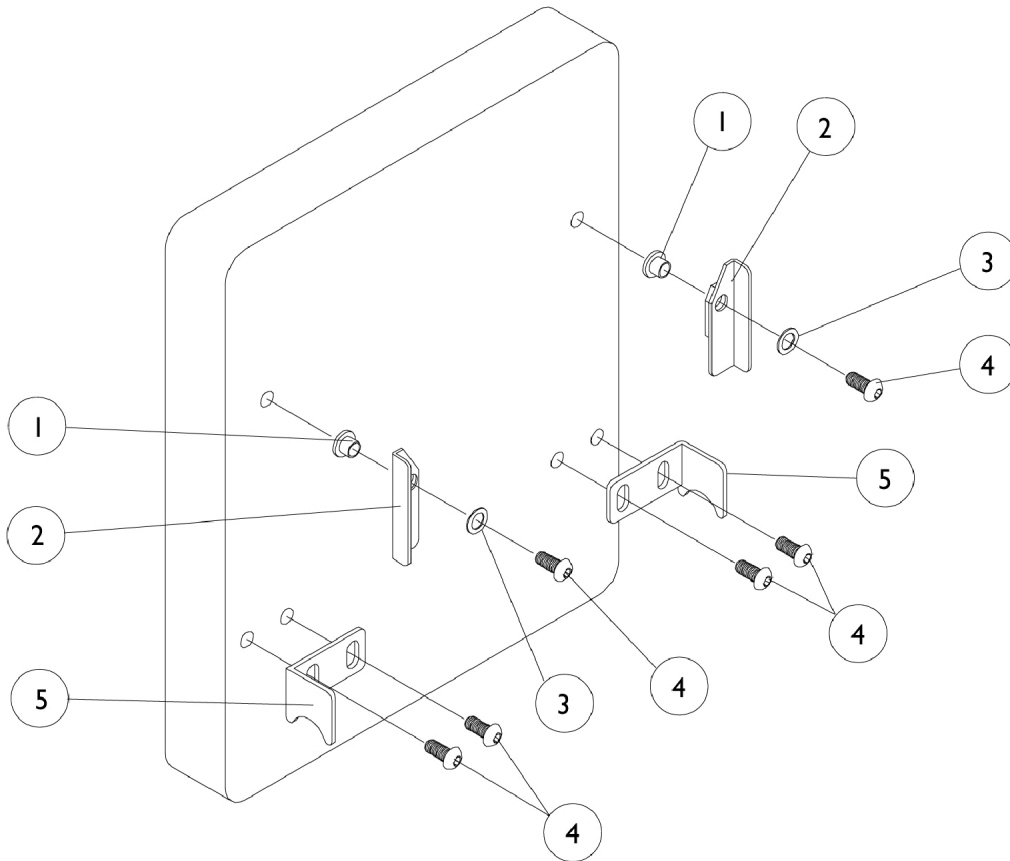

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1026039-NLA	NLA - Bushing, Pivot	1	2
2		1024573-NLA	NLA - Lever, Lock, Black - Right	1	1
2		1024574-NLA	NLA - Lever, Lock, Black - Left	1	1
3		1026040	Washer, Spring (5/16 x 1/2 x 1/16")	1	2
4		1025342-NLA	NLA - Screw, Button Head (1/4-20 x 5/8")	1	6
5		1025332-NLA	NLA - Mount, Lower Back - Right	1	1
5		1025333-NLA	NLA - Mount, Lower Back - Left	1	1