
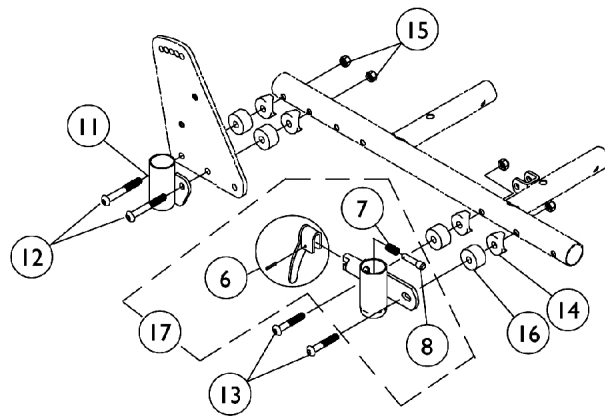
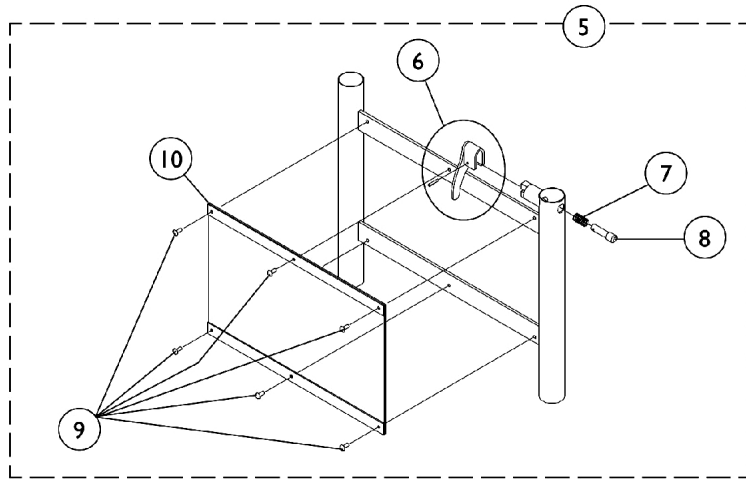


Adjustable Height Arms

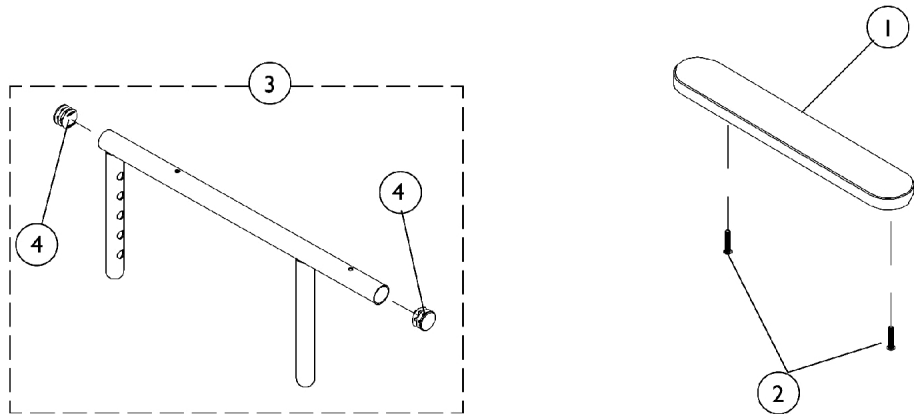

Adjustable Height Arms

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	8881011811-NL A	NLA - Pad, Arm, Vinyl, Desk Length (9" L x 2" W)	1	2
1	A	8881053578	Kit, Arm Pad w/ Screws, Desk Length (9" L x 2" W) (Pair)	2	1
1	A	8881015932-NL A	NLA - Pad, Arm, Vinyl, Full Length - Right (14" L x 2" W)	1	1
1	A	8881015933-NL A	NLA - Pad, Arm, Vinyl, Full Length - Left (14" L x 2" W)	1	1
1	A	8881053577	Kit, Arm Pad w/ Screws, Full Length (14" L x 2" W) (Pair)	2	1
2		7930327-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 1-1/8")	25	2
3		1070859	Arm Tube Assembly w/ Plug Buttons, Desk Length, Black - Right/Left	1	1
3		1041524-NLA	NLA - Tube, Arm, w/ Plug Buttons, Black, Full Length	1	1
4		1027095	Plug Button, Black (7/8")	1	2
5	B	1035892	Adjustable Height Arm Base Assembly - Right	1	1
6			Base, Armrest, Adjustable Height	1	1
7			Spring, Arm Lock	1	1
8			Pin, Arm Lock, Black	1	1
9			Rivet	1	6
10			NLA - Guard, Clothing	1	1
10			Kit, Lock Handle w/Rivets (Pair)	1	1
5	B	1035893	Adjustable Height Arm Base Assembly - Left	1	1
6			Base, Armrest, Adjustable Height	1	1
7			Spring, Arm Lock	1	1
8			Pin, Arm Lock, Black	1	1
9			Rivet	1	6
10			NLA - Guard, Clothing	1	1
10			Kit, Lock Handle w/Rivets (Pair)	1	1
11		1036837-NLA	NLA - Socket, Arm, Rear	1	1
12	C	7930301-NLA	NLA - Package, Button Head Cap Screw (1/4-20 x 1-3/4")	25	2
12	D	1036840	Screw, Socket Button Head (1/4-20X2-1/4")	1	2
13	C	1008495	Screw, Socket Button Head (1/4-20 x 1-1/2")	1	2
13	D	1002108-NLA	NLA - Screw, Socket Button Head (1/4-20 x 2")	1	2
14		40157X020	Spacer, Coved (1/4 x 1 x 9/16")	1	4
15		7930323-NLA	NLA - Package, Locknut (1/4-20)	25	4
16		1036040-NLA	NLA - Spacer (9/32 x 7/8 x 1/2")	1	4
17		1038401-NLA	NLA - Socket, Arm with Lock Handle, Front - Right	1	1
17		1038402-NLA	NLA - Socket, Arm with Lock Handle, Front - Left	1	1

NOTE: A - Specify upholstery color, U67 black vinyl, when ordering
 B - Stainless steel clothing guards were replaced with ABS plastic. Chairs manufactured before 7/30/97 should be ordered in pairs to match.
 C - All Chair Widths Except 14"
 D - For 14" Wide Chairs Only.