


ATL4A Legrest Support Assembly

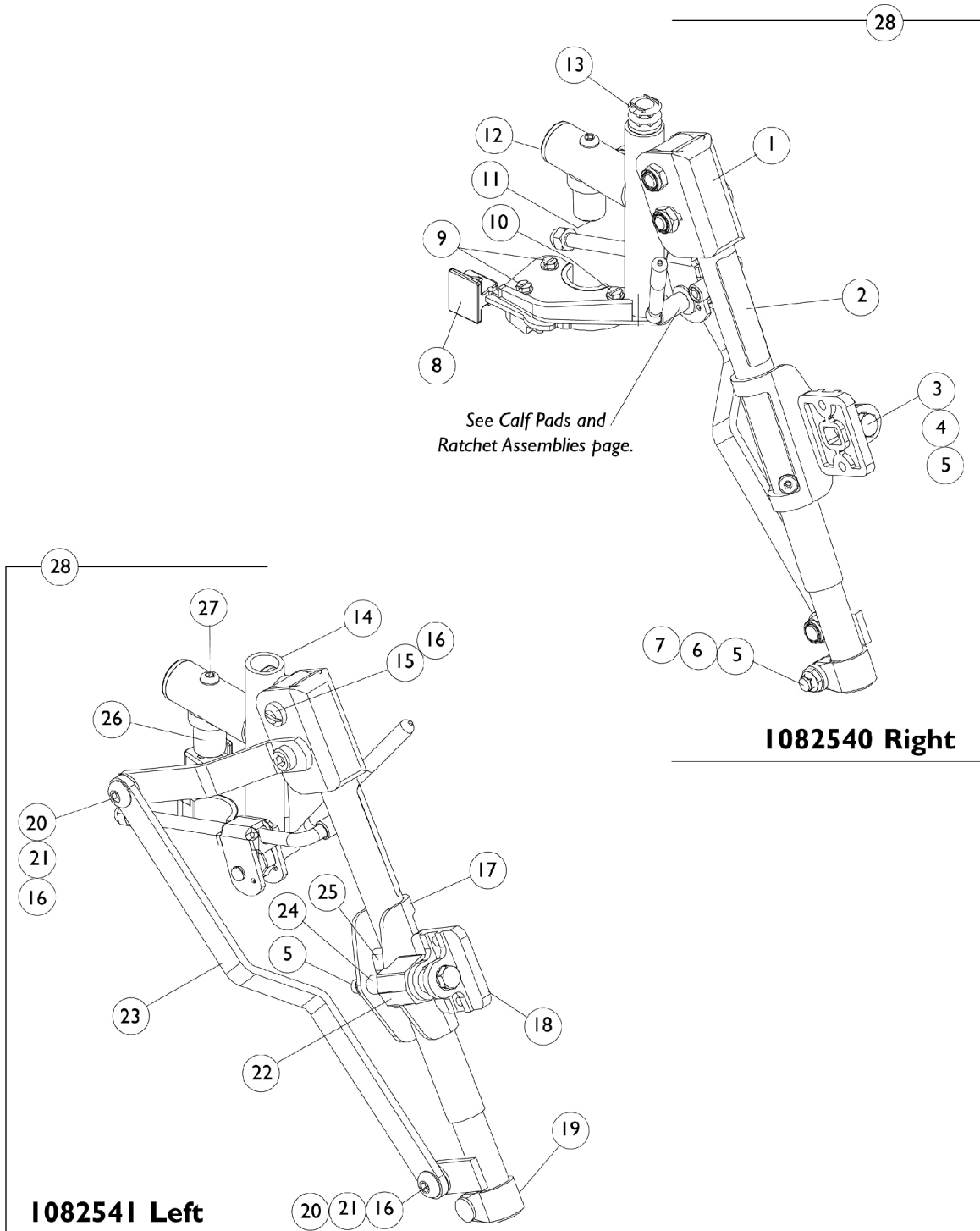

ATL4A Legrest Support Assembly

| Item Number | Note | Part Number | Description | Quantity Per | |
|-------------|------|-------------|---|--------------|----------|
| | | | | Package | Assembly |
| 1 | | 1022877-NLA | NLA - Tube, Support and Drive Link, Chrome - Right | 1 | 1 |
| 1 | | 1022878-NLA | NLA - Tube, Support and Drive Link, Chrome - Left | 1 | 1 |
| 2 | | 1024911-NLA | NLA - Decal, Smartleg | 1 | 1 |
| 3 | | 1014807 | Screw, Hex Head (5/16-24 x 3-1/4") | 1 | 1 |
| 4 | | 47330X003 | Spacer (5/16 x 1/2 x 3/8") | 1 | 1 |
| 5 | | 7930363-NLA | NLA - Package, Locknut (5/16"-24) | 25 | 2 |
| 6 | | 1027377 | Washer (#20) | 1 | 1 |
| 7 | | 11232X009 | Bolt, Lug (5/16-24 x 1-3/8") | 1 | 1 |
| 8 | | 1025454 | Housing, Latch - Right | 1 | 1 |
| 8 | | 1025455 | Housing, Latch - Left | 1 | 1 |
| 9 | | 1025451 | Screw, Slotted Hex Washer Head (#10-32 x 5/8") | 1 | 2 |
| 10 | | 7930314 | Package, Screw, Slotted Hex Washer Head (#10-32 x 1/2") | 25 | 1 |
| 11 | | 7930333 | Locknut (1/4-20) | 1 | 1 |
| 12 | | 1029501 | Plug Button, Black (1") | 1 | 1 |
| 13 | | 1027097 | Plug Button, Black (3/4") | 1 | 1 |
| 14 | | 1082538-NLA | NLA - Tube, Legrest Support, Articulating, Sector, Chrome - Right | 1 | 1 |
| 14 | | 1082539-NLA | NLA - Tube, Legrest Support, Articulating, Sector, Chrome - Left | 1 | 1 |
| 15 | | 1022854-NLA | NLA - Screw, Shoulder Slotted Head (1/2 x 1-1/8", 3/8-16) | 1 | 1 |
| 16 | | 40363X000 | Hex Nut (3/8"-16) | 1 | 3 |
| 17 | B | 1025104 | Bracket, Legrest, Chrome | 1 | 1 |
| 18 | | 04526X019 | Plate, Legrest Pad | 1 | 1 |
| 19 | | 1022848 | Tube, Slide, Articulating, Chrome - Right | 1 | 1 |
| 19 | | 1022849 | Tube, Slide, Articulating, Chrome - Left | 1 | 1 |
| 20 | | 1022866-NLA | NLA - Screw, Socket Button Head (3/8-16 x 3/4") | 1 | 2 |
| 21 | | 1049527 | Washer, Nylon (3/8 x 3/4 x 1/16") | 1 | 2 |
| 22 | | 1025133 | Spacer, Legrest Pad, Black | 1 | 1 |
| 23 | | 1022846-NLA | NLA - Bar, Link, Black | 1 | 1 |
| 24 | | 47328X003 | Bushing, Pivot (5/16 x 7/16 x 7/8") | 1 | 1 |
| 25 | | 48823X008 | Spring, Leaf | 1 | 1 |
| 26 | | 1017148 | Pivot, Footrest | 1 | 1 |
| 27 | | 7930301-NLA | NLA - Package, Button Head Cap Screw (1/4-20 x 1-3/4") | 25 | 1 |
| 28 | A | 1082540 | Articulating Legrest Support Assembly, Sector, Chrome - Right | 1 | 1 |
| 28 | A | 1082541 | Articulating Legrest Support Assembly, Sector, Chrome - Left | 1 | 1 |

NOTE: A - Includes items 1-27
B - Right/Left