


# Controller, Programmer, Shroud Cover & Mounting Hardware - TDX 3, 3 SE, 4 & 5

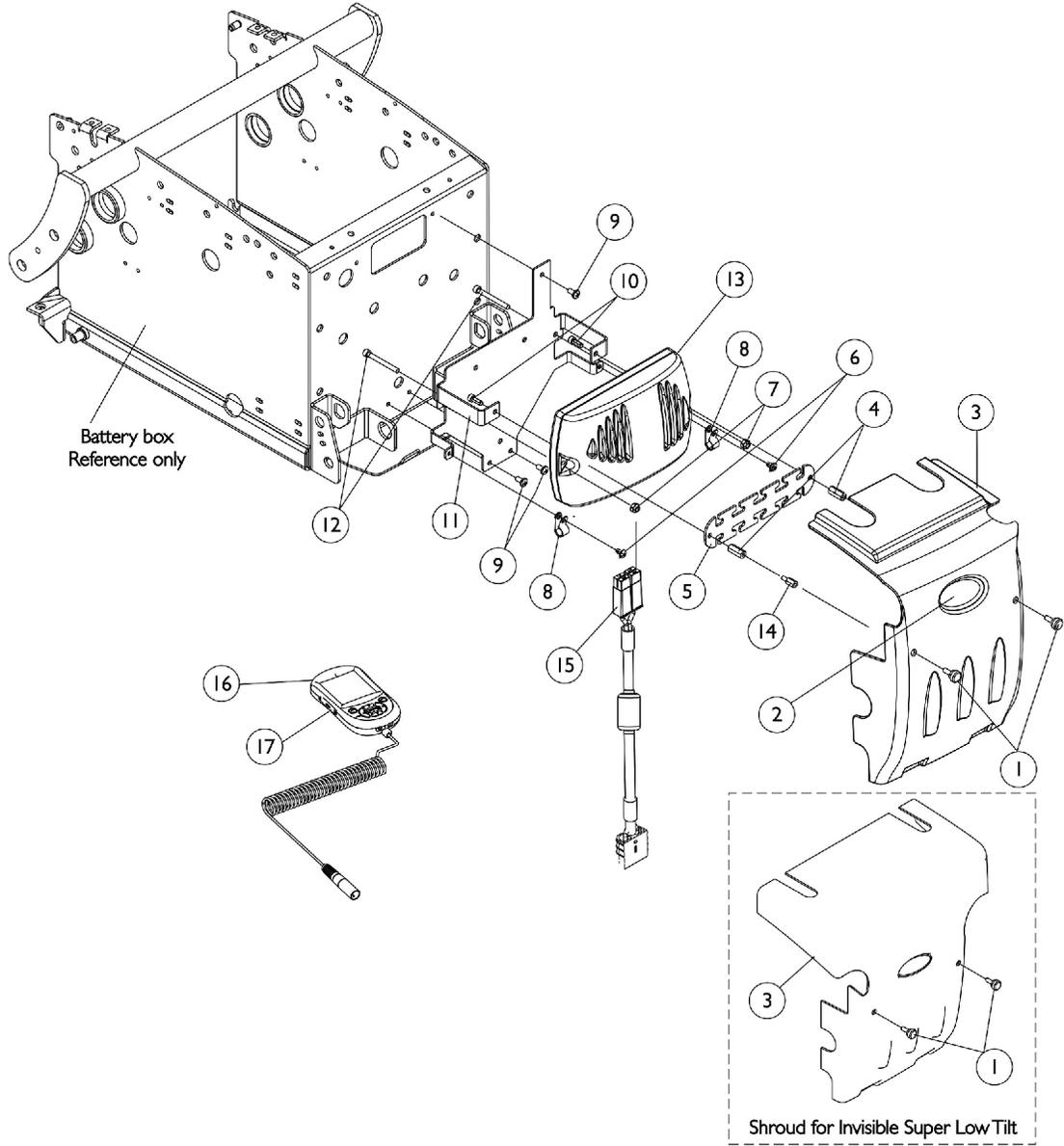

## Controller, Programmer, Shroud Cover & Mounting Hardware - TDX 3, 3 SE, 4 & 5

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1117240	Screw, Thumb (#10-32 x 1/2")	1	2
2		1095489	Decal, Invacare 3-D Logo (2-1/2" L)	1	1
3		1113214-NLA	NLA - Cover with Decal, Controller Shroud	1	1
3	A	1120240-NLA	NLA - Cover with Decal, Controller Shroud	1	1
4		1113342-NLA	NLA - Stand-Off (#10-32 x 3/4")	1	2
5		1117194-NLA	NLA - Strip, Wiring	1	1
6		90017X000-NLA	NLA - Screw, Phillips Pan Head (#10-32 x 5/16")	1	2
7		1056098	Locknut (#8-32)	1	2
8		1079245-NLA	NLA - Clamp, Cable	1	2
9		1022597	Screw, Phillips Pan Head Tap (#10-32 x 3/8")	1	3
10		1113438-NLA	NLA - 3Screw, Socket Head (#10-32 x 1/2")	1	2
11		1115829-NLA	NLA - Bracket, TDX Controller, Black	1	1
12		1127352	Screw, Phillips Pan Head (#8-32 x 1")	1	2
13	C	1136897	Controller, MK690	1	1
13	D,D1,D2	1136898	Controller, MK690 w/ ACC	1	1
13	E	1148360-NLA	NLA - Controller, MK690 w/ ACC	1	1
14	B	1120258-NLA	NLA - Stand-Off (10-32 x 1/2")	1	2
15		60102824	Kit, Cable, Battery Extension	1	1
16	F	1139985	Programmer, MK6i Remote	1	1
16	F	1142215	Kit, MK6i Remote Programmer w/ SD Professional Card & Card Holder (1813M6)	1	1
17		1139997	Cover, SD Card Slot	1	1

NOTE: A - Used on Super Low Tilt system  
 B - Item 14 is increased to a quantity of 4 when an Invisible Super Low Tilt is present  
 C - MK690 Controller used for All Non-Powered Seating except for the Formula TRE Powered Seating Systems. Used for Manual Recline Backs also.  
 D - MK690 w/ ACC used for Formula CG , Formula Invisible Super Low Tilt and Formula PTO Plus Powered Seating Systems.  
 D1 - part #1148360 is no longer required as a replacement for Base Only orders that shipped before 1/19/07,  
 D2 - because the software for part #1136898 was revised 9/12/07 to version 2.61a and now accommodates all Base Only orders that shipped before 1/19/07 as well.  
 E - Used for all Base Only orders. Discontinued 9/12/07, see part #1136898 as a direct replacement.  
 F - The MK6i Programmer Manual (1141471) is not included and is strongly recommended the guide be ordered or downloaded from Literature Fulfillment.