


Controller, Programmer, Shroud Cover & Mounting Hardware - TDX 3, 3 SE, 4 & 5 with TRRO/TRBKTS Option

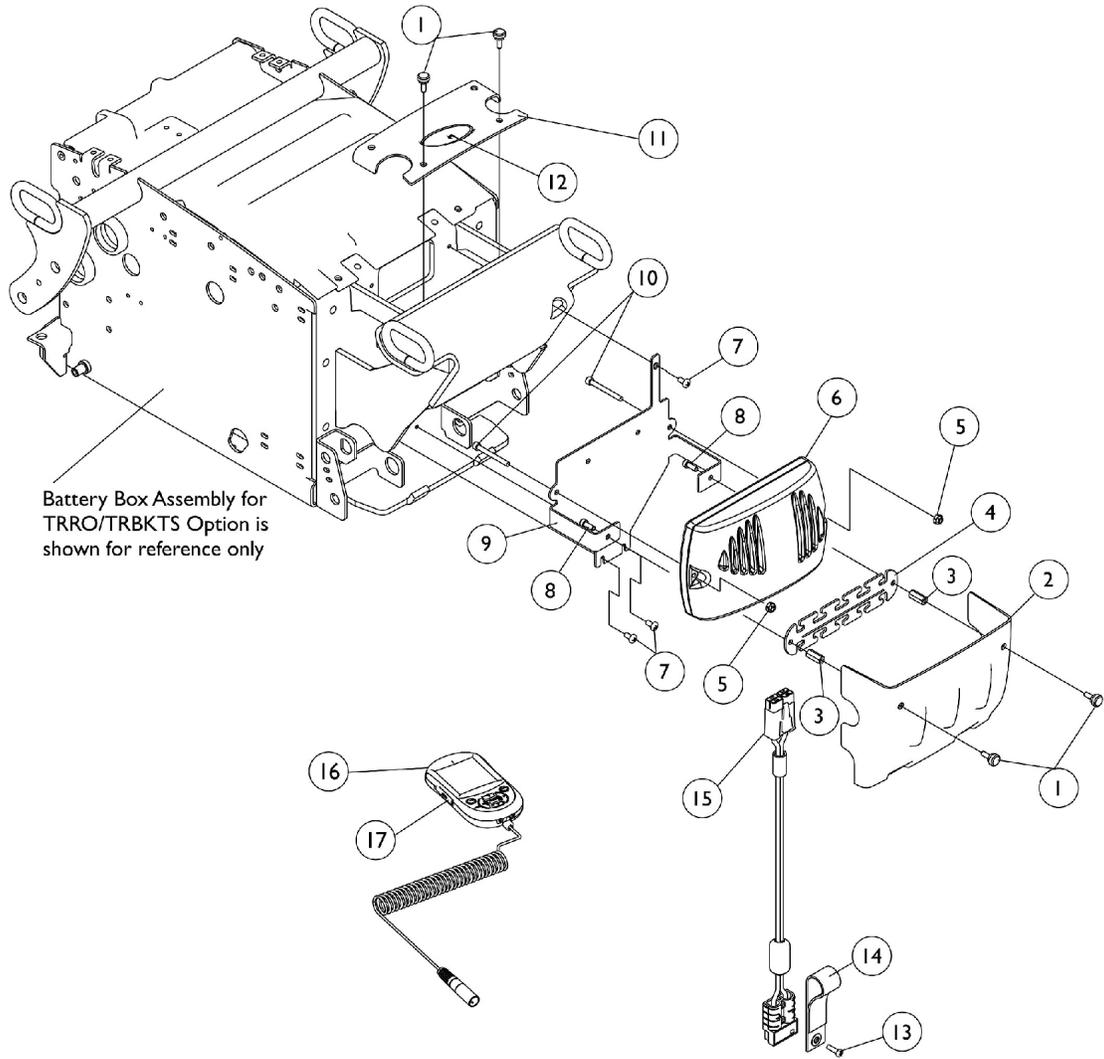

Controller, Programmer, Shroud Cover & Mounting Hardware - TDX 3, 3 SE, 4 & 5 with TRRO/TRBKTS Option

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1117240	Screw, Thumb (#10-32 x 1/2")	1	4
2		1137243-NLA	NLA - Cover, Controller Rear Shroud TRRO/TRBKTS	1	1
3		1113342-NLA	NLA - Stand-Off (#10-32 x 3/4")	1	2
4		1117194-NLA	NLA - Strip, Wiring	1	1
5		1056098	Locknut (#8-32)	1	2
6	A	1136897	Controller, MK690	1	1
6	B,B1	1136898	Controller, MK690 w/ ACC	1	1
6	C	1148360-NLA	NLA - Controller, MK690 w/ ACC	1	1
7		1022597	Screw, Phillips Pan Head Tap (#10-32 x 3/8")	1	3
8		1113438-NLA	NLA - 3Screw, Socket Head (#10-32 x 1/2")	1	2
9		1133754-NLA	NLA - Bracket, Controller Mounting, Black	1	1
10		1127352	Screw, Phillips Pan Head (#8-32 x 1")	1	2
11		1139532-NLA	NLA - Cover, Controller Top Shroud	1	1
12		1095489	Decal, Invacare 3-D Logo (2-1/2" L)	1	1
13		1114276	Screw, Phillips Pan Head (#8-16 x 5/8")	1	1
14		1113331-NLA	NLA - Strap, Battery Plug Pull	1	1
15		60102824	Kit, Cable, Battery Extension	1	1
16	D	1139985	Programmer, MK6i Remote	1	1
16	D	1142215	Kit, MK6i Remote Programmer w/ SD Professional Card & Card Holder (1813M6)	1	1
17		1139997	Cover, SD Card Slot	1	1

NOTE: A - MK690 Controller used for All Non-Powered Seating except for the Formula TRE Powered Seating Systems.
B - MK690 w/ ACC used for Formula CG , Formula Invisible Super Low Tilt and Formula PTO Plus Powered Seating Systems.
B1 - part #1148360 is no longer required as a replacement for Base Only orders shipped before 1/19/06 because the software for part #1136898 was revised 9/12/07 to version 2.61a and now accommodates all Base Only orders that shipped before 1/19/07 as well.
C - Used for all Base Only orders. Discontinued 9/12/07, see part #1136898 as a direct replacement.
D - The MK6i Programmer Manual (1141471) is not included and is strongly recommended the guide be ordered or downloaded from Literature Fulfillment.