


One Piece Adjustable Angle Footboard

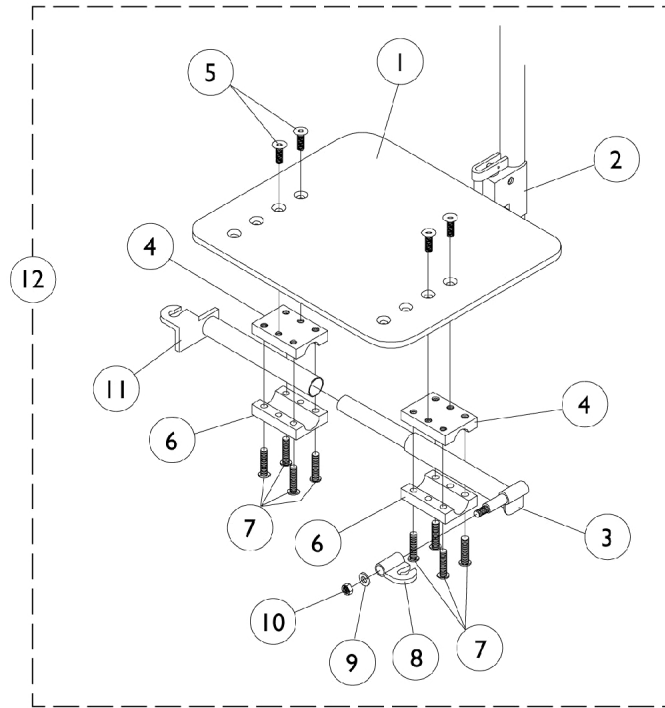

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1044076	Footboard	1	1
1	B	1044089-NLA	NLA - Footboard	1	1
2		1027657-NLA	NLA - Clamp, Footplate	1	2
3	A	1027676-NLA	NLA - Footrest Support Assembly, Black	1	1
3	B	1030271-NLA	NLA - Footrest Support Assembly, Black	1	1
4		1027626-NLA	NLA - Half Clamp, Threaded	1	2
5		1029545	Screw, Socket Flat Head (1/4-20 x 3/4")	1	4
6		1027627-NLA	NLA - Half Clamp, Non-Threaded	1	2
7		1009837	Screw, Socket Button Head (1/4-20 x 1")	1	8
8		1027630-NLA	NLA - Tab, Weldment, Black	1	1
9		1026033	Washer, Nylon (1/4 x 1/2 x 1/16")	1	1
10		7930323-NLA	NLA - Package, Locknut (1/4-20)	25	1
11	A	1027638-NLA	NLA - Footrest Support Assembly, Black	1	1
11	B	1030199-NLA	NLA - Footrest Support Assembly, Black	1	1
12	A	1027640-NLA	NLA - Footboard Assembly, One Piece Adjustable Angle	1	1
12	B	1030200-NLA	NLA - Footboard Assembly, One Piece Adjustable Angle	1	1

NOTE: A - 12-13" Wide Chairs
 B - 14-16" Wide Chairs