


Minor Bathlift

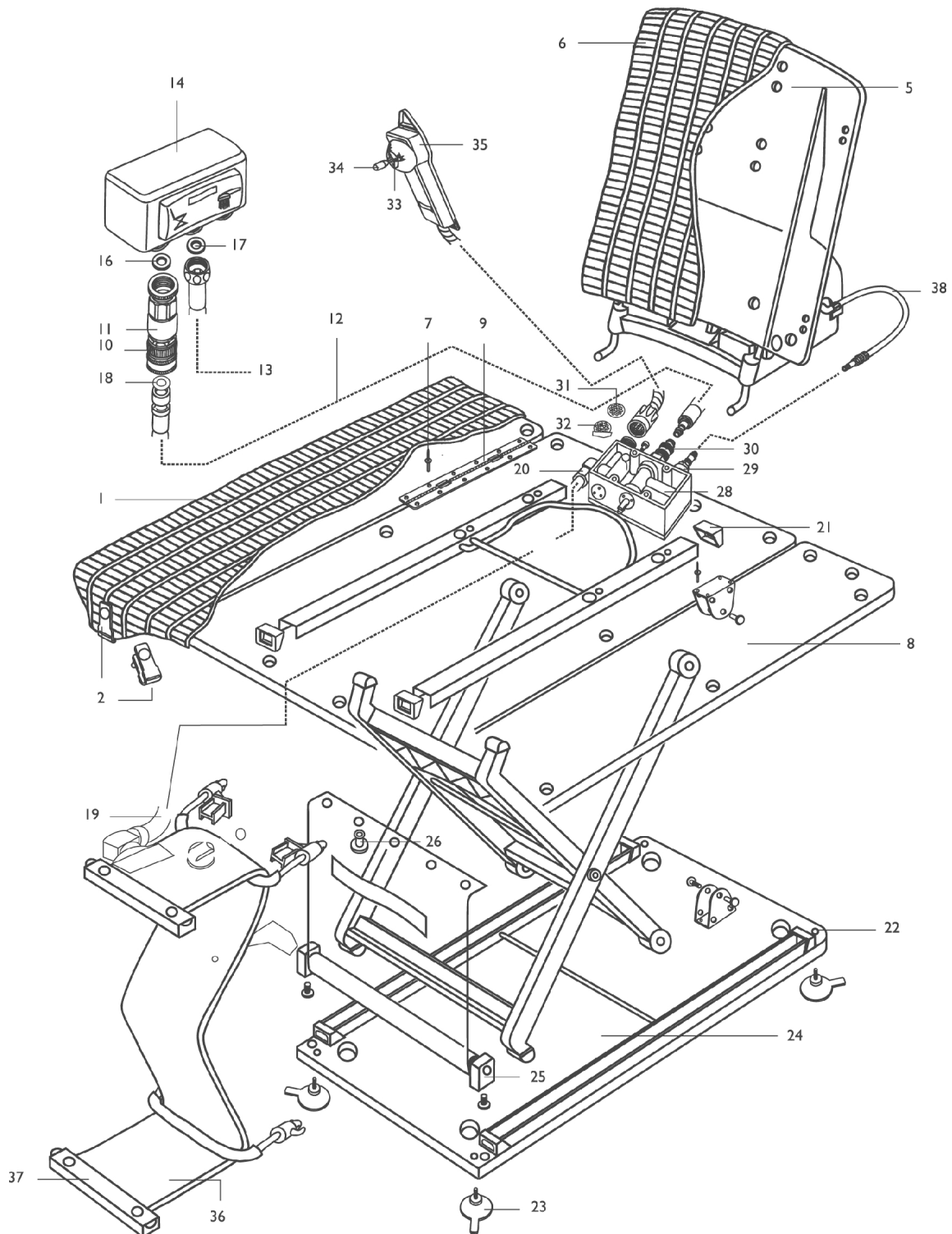

Minor Bathlift

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		11142-NLA	NLA - Cover, Seat, Blue (5 mm)	--	--
2		12423-NLA	NLA - Fixation Set for Cover, Blue	--	--
5		11517	Backrest, Reclining without Cover	--	--
6		11535	Cover, Reclining Backrest, Blue (5 mm)	--	--
7		14041	Rivet (3.2 x 9.5L)	--	--
8		10929	Flap, Side	--	--
9		10176	Hinge	--	--
10		11491	Quick Disconnect (3/8")	--	--
11	A	11023-NLA	NLA - Insert, Sieve	--	--
12		12404	Hose, Supply (2 m)	--	--
13		11302	Hose, Shower (1 m)	--	--
14		12585	Switch-Over	--	--
15	A	11551-NLA	NLA - Adhesive Tape for Switch-Over	--	--
16		10860-NLA	NLA - Washer	--	--
17		11385-NLA	NLA - Washer, Shower Hose (19mm x 8mm x 1mm)	--	--
18		11661	O-Ring (18mm x 2mm)	--	--
19		11020-NLA	NLA - Hose, PVC, Blue (1 m)	--	--
20		10356-NLA	NLA - Nut, Aluminum	--	--
21		10933	Cover, C-Rail	--	--
22		10701	Screw	--	--
23		12496	Cups, Suction, Set	--	--
24		12478	Footguard, Complete	--	--
25	B	10171	Mountings, Footguard	--	--
26		11034	Clip, Footguard	--	--
28		14149	Control Unit, Stationary Backrest	--	--
28		11456	Control Unit, Reclining Backrest	--	--
29		11003	Coupling, Miniature, Reclining Backrest	--	--
30		10112DOL	Disconnect, Quick (3/8")	--	--
31		11376	Washer, Round	--	--
32		12608	Washer, Flat	--	--
33		14403	Joystick without Knob Assembly, Stationary Backrest	--	--
33		12413	Joystick without Knob Assembly, Reclining Backrest	--	--
34		12331	Knob, Reclining Backrest	--	--
34		14098	Knob, Stationary Backrest	--	--
35		14060	Hand Control, Stationary Backrest	--	--
36		13942	Bladder, Stationary Backrest	--	--
36		12851	Bladder, Reclining Backrest	--	--
37		11027-NLA	NLA - Cover, Hose, Blue	--	--
38		12843	Hose, Reclining Backrest	--	--
	A	13954	Cover, Seat Special, Blue (5 mm)	--	--
	A	11172	Cover, Seat Special for Minor M2/M3, Blue (5 mm)	--	--
	A	11161	Backrest, Stationary without Cover	--	--
	A	11136-NLA	NLA - Cover, Backrest, Blue (5 mm)	--	--
	A	11154-NLA	NLA - Cover, Stationary Backrest Special, Blue (5 mm)	--	--
	A	11148	Cover, Reclining Backrest Special, Blue (5 mm)	--	--

Minor Bathlift

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
	A	11082	Adapters, Height (2 cm)	--	--
	A	11081	Adapters, Height (4 cm)	--	--
	A	11080	Adapters, Height (6 cm)	--	--
	A	10123	Adapters, Height, Special	--	--
	A	15317-NLA	NLA - Adapters, Special Height (20 mm)	--	--
	A	15318-NLA	NLA - Adapters, Special Height (40 mm)	--	--
	A	15319	Adapters, Special Height (60 mm)	--	--
	A	12404	Hose, Supply (2 m)	--	--
	A	12405	Hose, Supply Set (3 m)	--	--
	A	11302	Hose, Shower (1 m)	--	--
	A	11364-NLA	NLA - Hose, Shower (1.5 m)	--	--
	A	10084-NLA	NLA - Holder, Supply Hose	--	--
	A	12704	Extension, Vaillant Switch (1/2")	--	--
	A	11568	Junction, Vaillant	--	--
	A	10447-NLA	NLA - Fixation Piece, L-Shaped for Vaillant Switch (New Style)	--	--
	A	14147	Connector, Y-Style	--	--
	A	12550	Connector, Vaillant Switch Reduction (3/4" - 1/2" Regular Thread)	--	--
	A	10001	Connector, Shower (1/2")	--	--
	A	11123	Fixation Set for Switch Cover	--	--
	A	10128	Connection, Waterstop	--	--
	A	12101	Kit, Installation	--	--
	A	11127	Sleeve, Nut for Suction Cup	--	--
NOTE: A - Not Shown. B - Includes (2) Fixation Screws and (1) Mounting Screw.					