


Ocean Commode

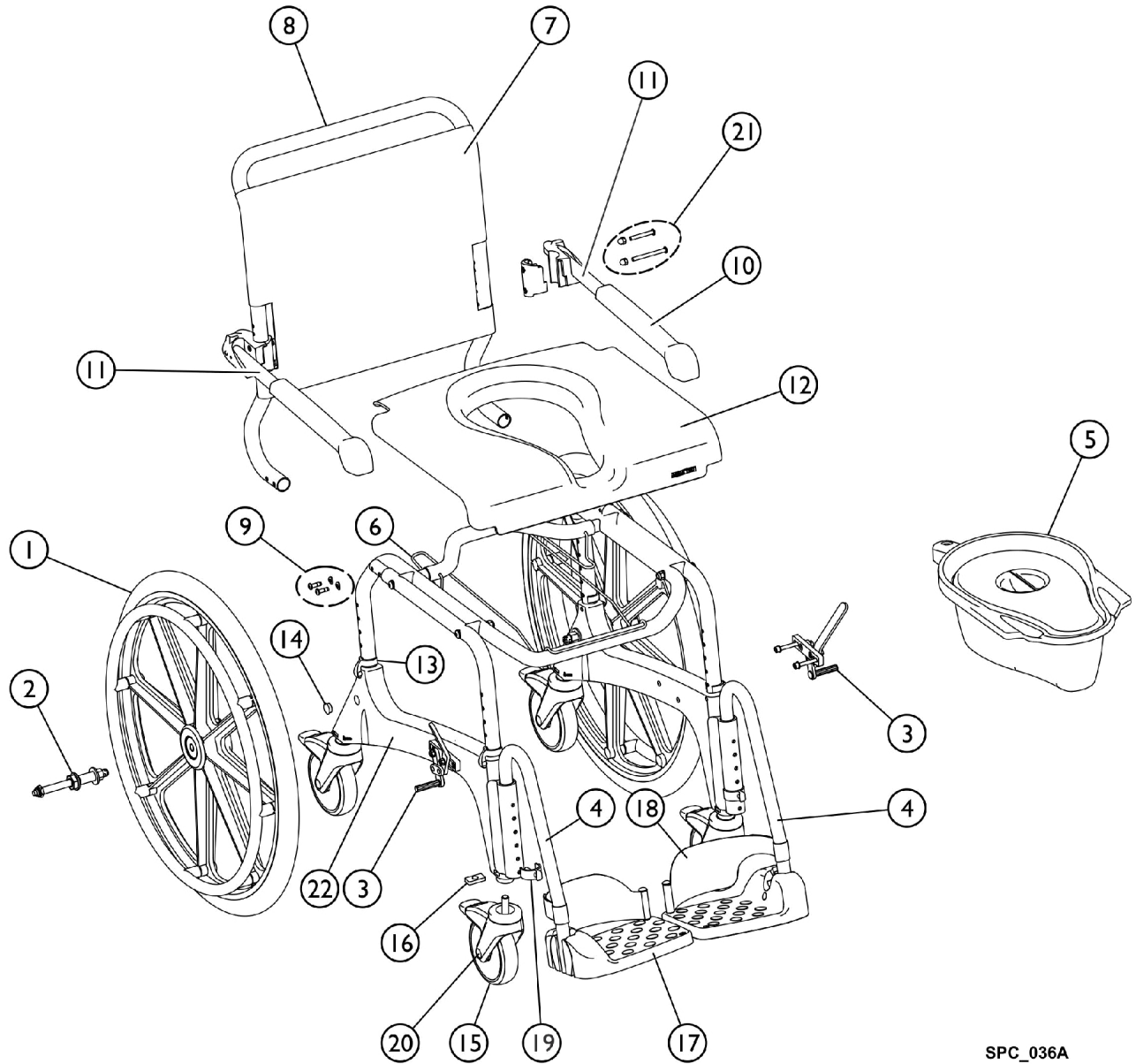

SPC_036A

Ocean Commode

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		16336	24 In Wheel W / Handrim for Ocean/Ergo SP	1	--
2		16418	Cpl Set Hub and QR Axle (16338 and 16337)	1	--
3		16339	Lt. Brk. Lever - Ocean / Ergo Commodes	1	--
3		16340	Rt. Brk. Lever Ocean / Ergo Commodes	1	--
	A	16379	Self Propelling Wheels Kit complete including brakes	1	--
4	C	1470696-NLA	NLA - Footrest, Complete - Right	1	--
4		SP1643542	Footrest cpl., Right - Ocean (XL) / VIP / Ocean Ergo Series	1	--
4	D	1470695-NLA	NLA - Footrest, Complete - Left	1	--
4		SP1643541	Footrest cpl., Left - Ocean (XL) / (Dual-)VIP / Ocean Ergo Series	1	--
5		10230	Collection Pan with Lid	1	--
	B	1541289	Lid, Pan	1	--
6		1527985	Holder, Toilet Pan	1	--
7		1528289	Cover, Backrest, Ocean Facelift (After July 2011)	1	--
7		1527988	Cover, Backrest, Ocean XL Facelift (After July 2011)	1	--
7		16257	Cover, Backrest, Adjustable, Ocean (Before July 2011)	1	--
8		1528025	Backrest w/o Armrest and Cover	1	--
9	E	1542900	Bkrst Hdwr. - Ergo / Ocean (Dual-)VIP / XL	1	--
10		16472	Armrest Pads Incl. Screws Kit	1	--
11		1470693	Armrest assembly, Right	1	--
11		1470694	Armrest Assembly Left	1	--
12		16262	Seat	1	--
13		16256	Side Frame Adj. Clip	1	--
14		16258	Package, Plug	1	--
15		16259	Caster, Locking, Red Tab	1	--
16		1470688	Plate, Threaded	1	--
17		1470689	Footrest Plate	1	--
18		16334	Heel Loop (Pair)	1	--
19		16333	Clip, Footrest	1	--
20		1513809	Cover Cap for 5" Wheel	1	--
21		1471052	Ocean Armrest Kit Parts	1	--
22		16737	Side Frame w/ Wheels	1	--

NOTE: A - Includes Left and Right Brakes, Hub and Axle, and the Wheel.
 B - Includes Pan.
 C - Part number 1470696 in No Longer Available, use part number SP1643542 as a replacement.
 D - Part number 1470695 in No Longer Available, use part number SP1643541 as a replacement.
 E - After SN14403200/14406200