


Lower Frame & Seat Frame Assembly (After 06/2001)

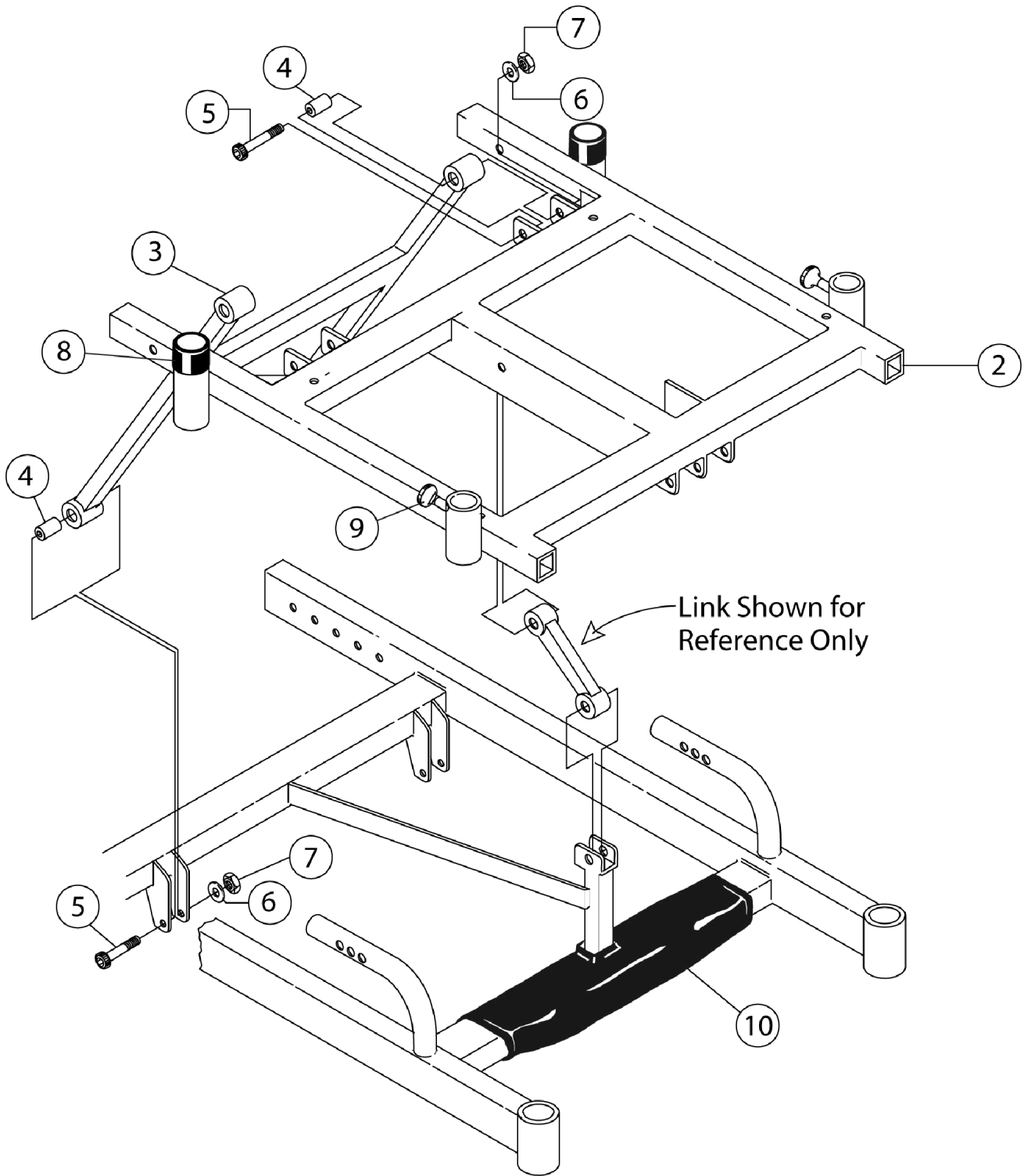

Lower Frame & Seat Frame Assembly (After 06/2001)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		440079-5	Frame, Lower, Black Metallic (15" W)	--	1
1		440080-5	Frame, Lower, Black Metallic (16" W)	--	1
1		440081-5	Frame, Lower, Black Metallic (17" W)	--	1
1		440082-5	Frame, Lower, Black Metallic (18" W)	--	1
1		440083-5	Frame, Lower, Black Metallic (19" W)	--	1
1		440084-5	Frame, Lower, Black Metallic (20" W)	--	1
2		1077170	Frame, Seat (15" W)	--	1
2		1077172	Frame, Seat (16" W)	--	1
2		1077174	Frame, Seat (17" W)	--	1
2		1077176	Frame, Seat (18" W)	--	1
2		1077178	Frame, Seat (19" W)	--	1
2		1077180	Frame, Seat (20" W)	--	1
3		440100	Support Link - Rear	--	1
4		440101	Bushing	--	4
5		1042484-NLA	NLA - Screw, Hex Head (5/16-18 x 2")	4	--
6		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	4	--
7		1026158	Locknut (5/16-18)	4	--
8		XB-55	Silencer, Tube, Arm Socket - Rear	--	2
9		ALK	Kit, Arm Lock	--	--
			NLA - Knob, Black	1	1
			Spring, Lift Pin	1	1
			NLA - Pin, Lift	1	1
	A	1108134-NLA	NLA - Knob, Black	1	1
	A	1108188-NLA	Spring, Lift Pin	1	1
	A	1132724-NLA	NLA - Pin, Lift	1	1
10		1153543	Impact, Lower (15 - 17" Frame)	--	1
10		1153542	Impact, Lower (18 - 20" Frame)	--	1

NOTE: A - Included in ALK kit.