


Adjustable Angle Back

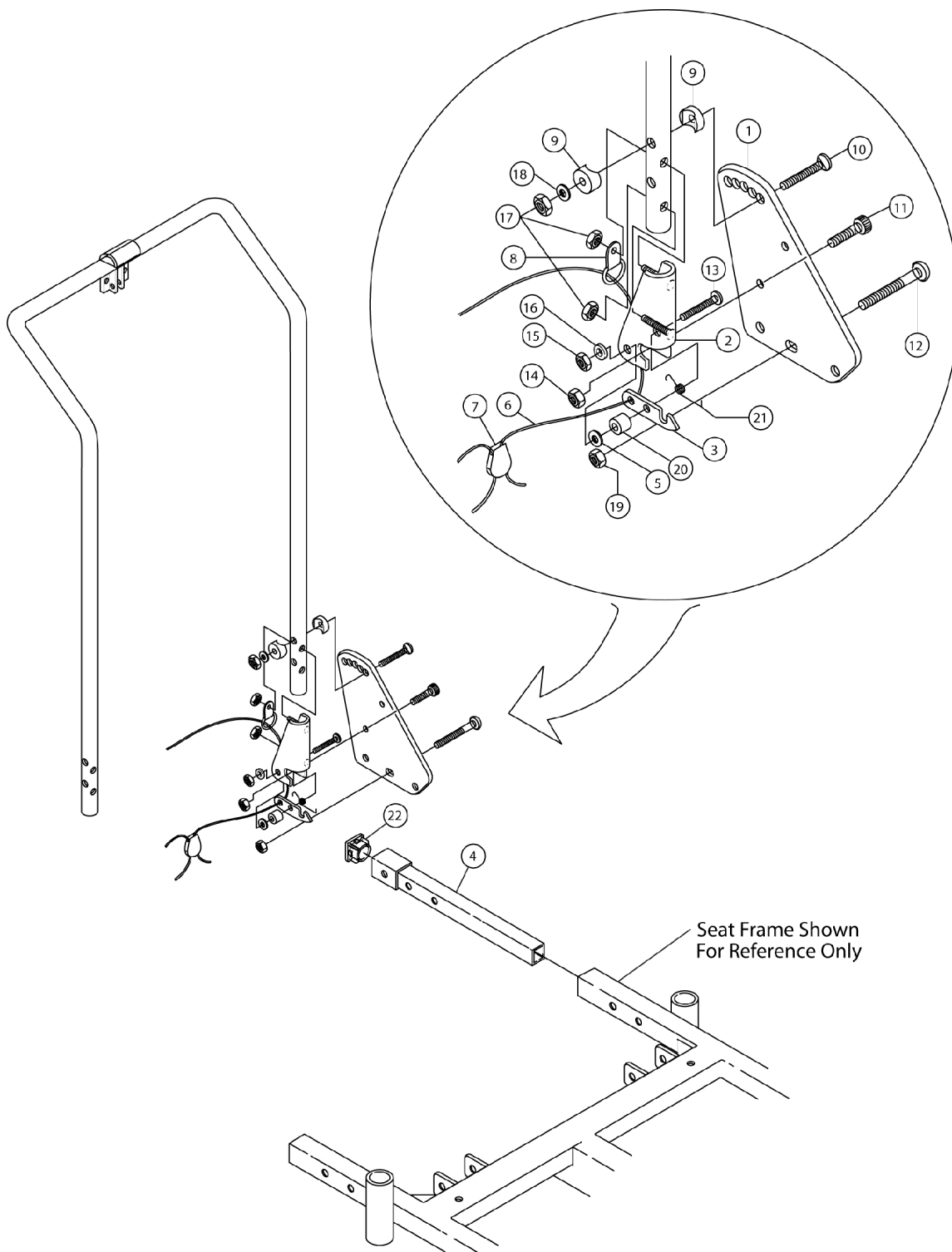

Adjustable Angle Back

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1-22		440589	Adjustable Angle Kit	--	1
1		1027143-NLA	NLA - Plate, Back Mounting, Left	--	1
		1027142	Plate, Back Mounting, Right	--	1
2		1025335-NLA	NLA - Bracket, Latch, Black - Left	--	1
		1025334-NLA	NLA - Bracket, Latch, Black - Right	--	1
3		1025337-NLA	NLA - Latch	--	2
4		440165	Tube, Extension	--	2
5		1002102-NLA	NLA - Washer, Wave (3/16 x 7/16 x 1/16")	--	2
6		1029437-NLA	NLA - Cord (5-1/2')	--	1
7		1029412	Lock, Cord, Black	--	1
8		1048817-NLA	NLA - Ring, "D"	--	2
9		40157x019	Coved Spacer	--	4
10		1001762	Screw, Socket Button Head (1/4-20 x 1-3/4")	--	2
11		360114	Screw, Shoulder Hex Socket (5/16 x 1-1/4")	--	2
12		1008495	Screw, Socket Button Head (1/4-20 x 1-1/2")	--	6
13		1021343	Screw, Socket Head (5/16-18 x 1-1/4")	--	2
14		1025195	Locknut (1/4-20)	--	8
15		1026031-NLA	NLA - Nut, Hex with Nylon Cap (#10-32)	--	2
16		1026034-NLA	NLA - Washer, Nylon (3/16 x 7/16 x 1/8")	--	2
18		1026060-NLA	NLA - Washer, Flat (9/32 x 1 x 5/64)	--	4
19		1025736	Locknut (1/4-20)	--	6
20		1020363-NLA	NLA - Spacer (1/4 x 1/2 x 1/2")	--	2
21		1025326-NLA	NLA - Spring, Torsion - Left	--	1
		1025325-NLA	NLA - Spring, Torsion - Right	--	1
22		B445	Plug	--	2
	A	1027095	Plug Button, Black (7/8")	--	--
NOTE: A - Not shown.					