


# Recline Back Assembly

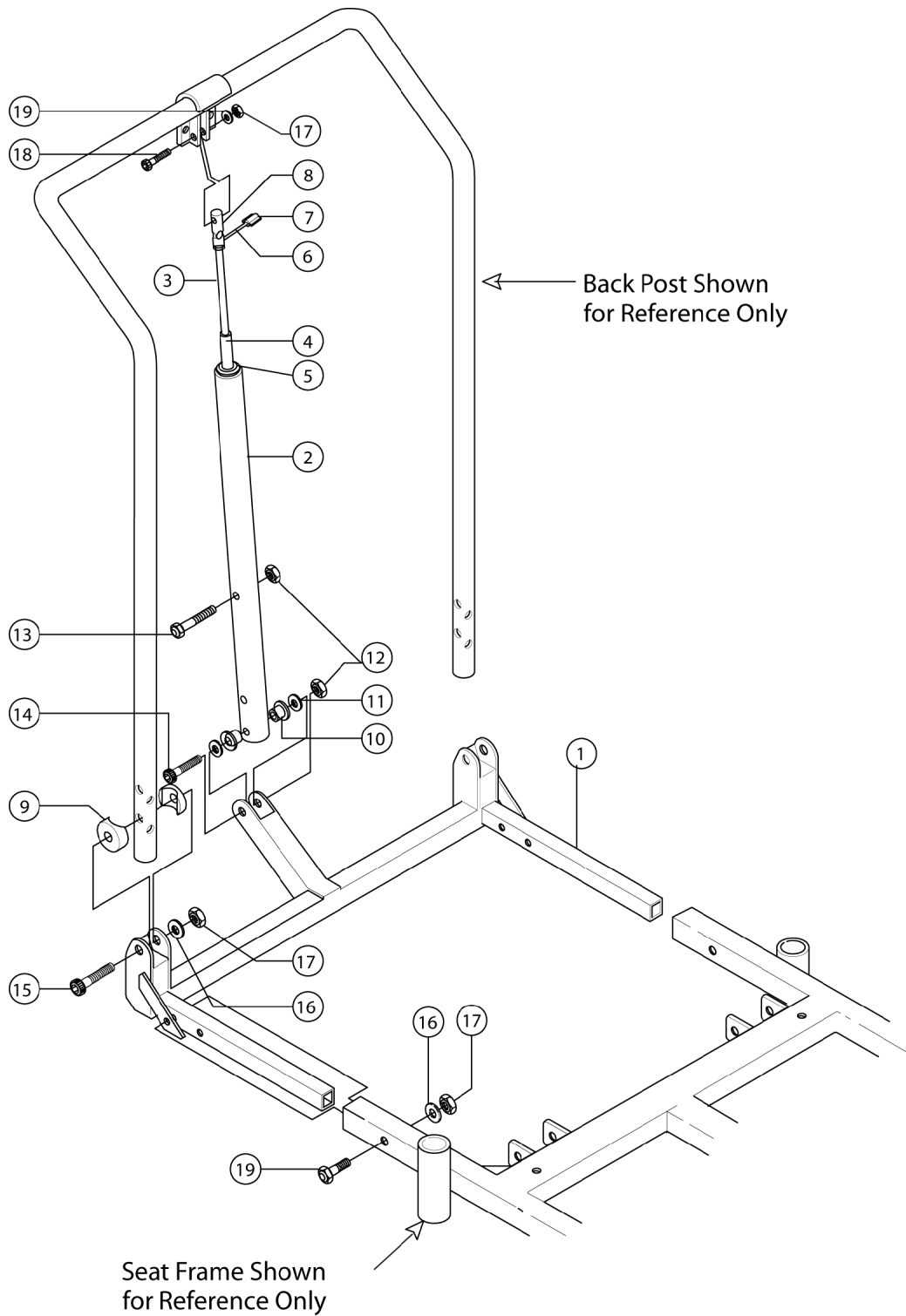

## Recline Back Assembly

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		440347-15	Frame, Recline, Back, Black (15" W)	--	1
1		440347-16	Frame, Recline, Back, Black (16" W)	--	1
1		440347-17	Frame, Recline, Back, Black (17" W)	--	1
1		440347-18	Frame, Recline, Back, Black (18" W)	--	1
1		440347-19	Frame, Recline, Back, Black (19" W)	--	1
1		440347-20	Frame, Recline, Back, Black (20" W)	--	1
2		440462	Tube, Extender (20" and 24" Back Height)	--	1
3-19		440342	Kit, Recline Back Hardware	--	1
3		1050154	Cylinder, Lock, Recline	--	1
4		440464	Spacer Bushing	--	1
5		361500	Washer (10.5mm I.D. x 25mm O.D. x 4mm)	--	1
6		361495	Handle, Clevis	--	1
7		361496	Handle, Grip	--	1
8		361494	End, Clevis	--	1
9		361502	Spacer, Coved (5/16 x 3/4 x 7/16")	--	4
10		361550	Bearing, Nylon	--	2
11		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	--	2
12		1026158	Locknut (5/16-18)	--	2
13		319156	Bolt, Hex Head (5/16 - 18-1/34")	--	1
14		1042484-NLA	NLA - Screw, Hex Head (5/16-18 x 2")	--	1
15		360114	Screw, Shoulder Hex Socket (5/16 x 1-1/4")	--	2
16		319476	Washer	--	5
17		1025736	Locknut (1/4-20)	--	5
18		360111	Screw, Shoulder Hex Socket (5/16 x 3/4")	--	1
19		1030949	Screw, Hex Head (1/4-20 x 1-1/2")	--	2