



Rear Wheels & Related Parts

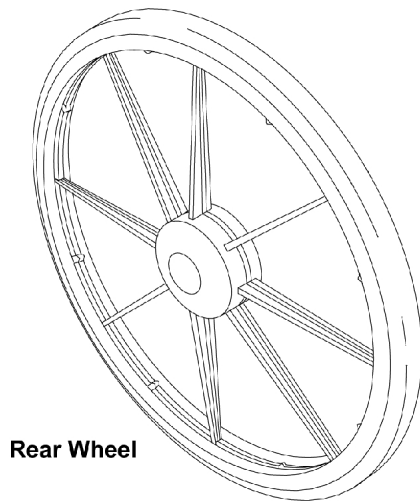

Rear Wheel

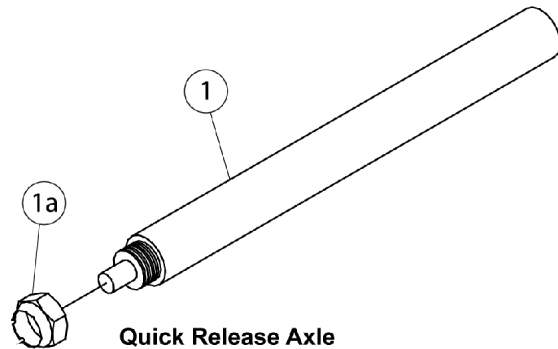

Quick Release Axle

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	56000X651	Axle, Quick Release (7/16" x 5-7/16")	1	--
1a		1045608	Locknut (7/16-20)	1	--
		1085454	Bolt, Hex Head Cap (7/16-20 x 5-1/8")	1	--
		See Chart	Rear Wheels and Related Parts	--	--
NOTE: A - Not shown.					

Rear Wheels & Related Parts

Wheel Size	Tire Type	Wheel Single W/O Handrim	Tire	Tube Insert	Bearing	Hub Spacer
20"	Pneumatic 20" x 1/38"	1034126	1017519	1017520	40117X024	1015756
20"	Pneumatic W/Flat Free 20" x 1/38"	1034127	1017519	1023322	40117X024	1015756
20"	Urethane	1038555	1037774	N/A	40117X024	1015756
22"	Pneumatic 22" x 1/38"	1027149	1018544	1018545	40117X024	1015756
22"	Pneumatic W/Flat Free 22" x 1/38"	1027150	1018544	1023323	40117X024	1015756
22"	Urethane	1032115	1028250	N/A	40117X024	1015756
24"	Pneumatic 24" x 1/38"	1025123	40176X060	40176X061	40117X024	1015756
24"	Pneumatic W/Flat Free 24" x 1/38"	1025124	40176X060	1010906	40117X024	1015756
24"	Urethane	1032117	1026904	N/A	40117X024	1015756