
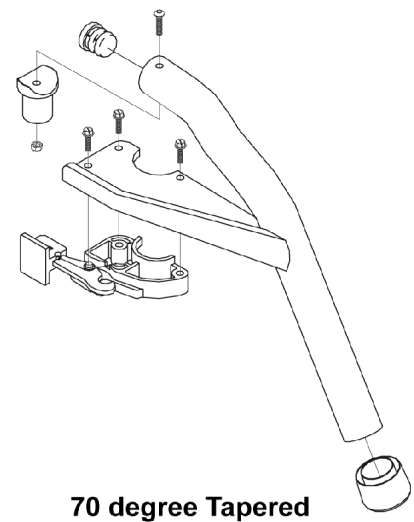
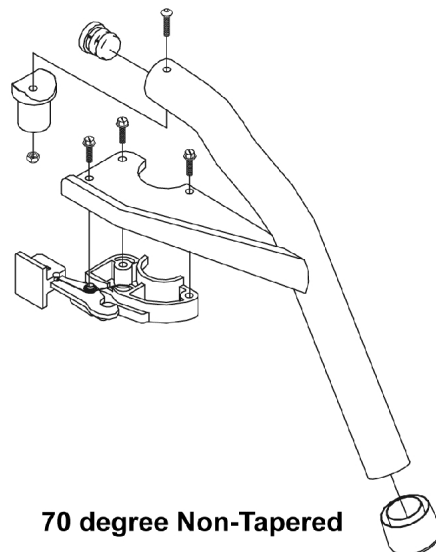
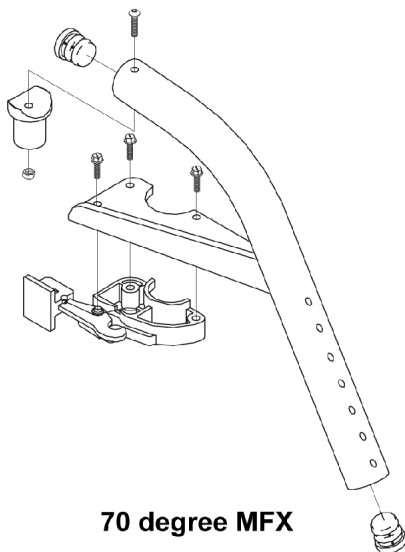
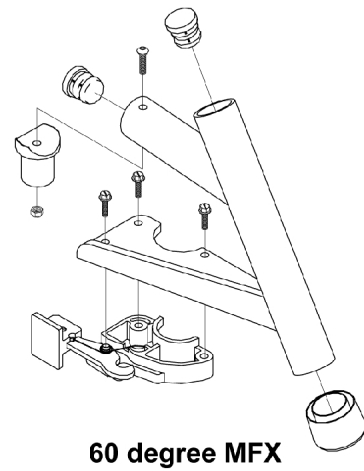


Front Riggings - Swingaway Supports 60 and 70 Degree

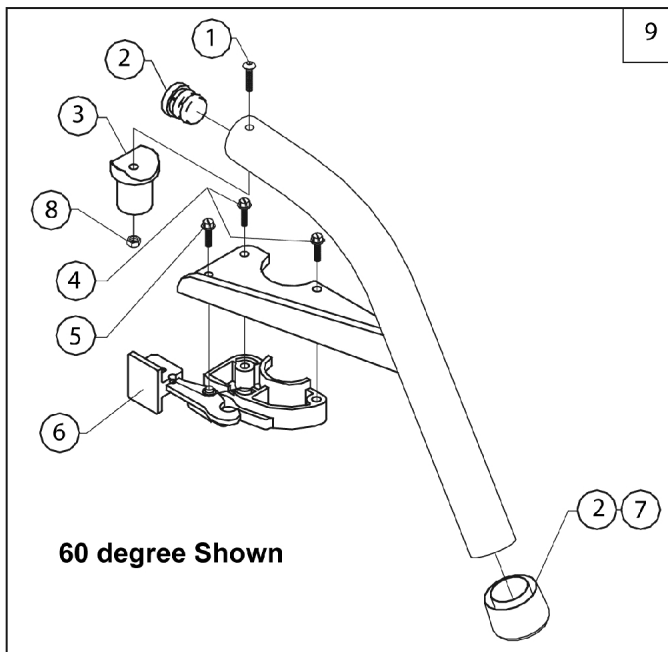

Front Riggings - Swingaway Supports 60 and 70 Degree

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1001762	Screw, Socket Button Head (1/4-20 x 1-3/4")	1	2
2		1056328	Plug Button, Black (1")	2	4
3		1017148	Pivot, Footrest	1	2
4		1025451	Screw, Slotted Hex Washer Head (#10-32 x 5/8")	2	4
5		1017423	Screw, Slotted Hex Washer Head w/ Patch (#10-32 x 1/2")	1	2
6		1025454	Housing, Latch - Right	1	--
		1025455	Housing, Latch - Left	1	--
7		1026124	Collar, Anti-Rattle	1	--
8		1025736	Locknut (1/4-20)	1	--
9		1073011	Footrest Support Assembly, 60 Degree, Sector, Black Metallic - Right	--	1
		1073010	Footrest Support Assembly, 60 Degree, Sector, Black Metallic - Left	--	1
		1073027	Footrest Support Assembly, 70 Degree, Sector, Black Metallic - Right	--	1
		1073026	Footrest Support Assembly, 70 Degree, Sector, Black Metallic - Left	--	1
		1073003	Footrest Support Assembly, 60 Degree MFX, Sector, Black Metallic - Right	--	1
		1073002	Footrest Support Assembly, 60 Degree MFX, Sector, Black Metallic - Left	--	1
		1073019	Footrest Support Assembly, 70 Degree MFX, Sector, Black Metallic - Right	--	1
		1073018	Footrest Support Assembly, 70 Degree MFX, Sector, Black Metallic - Left	--	1
		1073035	Footrest Support Assembly, 70 Degree Tapered, Sector, Black Metallic - Right	--	1
		1073034	Footrest Support Assembly, 70 Degree Tapered, Sector, Black Metallic - Left	--	1