




Pivot/Slide Tube w/ Footplate Assembly - Aluminum

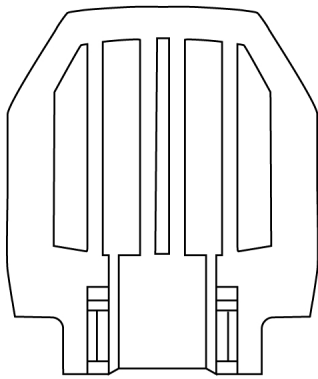

Small / Medium

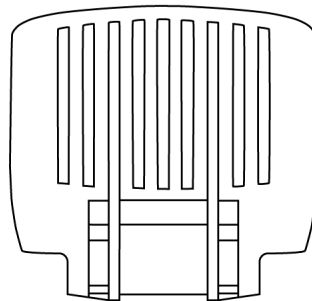

Extra Small

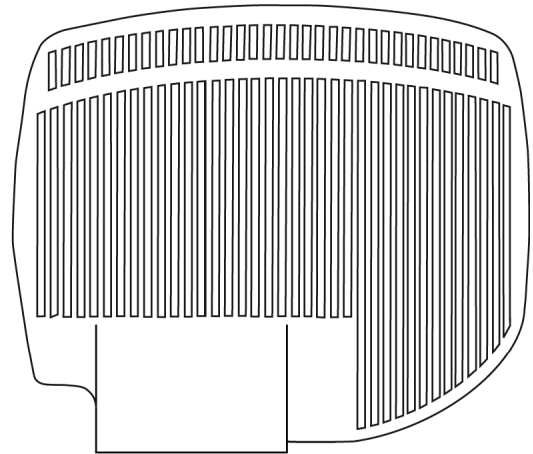

Extra Large

Pivot/Slide Tube w/ Footplate Assembly - Aluminum

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		44200X032-NLA	NLA - Footplate, Aluminum, Extra Small, Black (5-3/4" W x 6" L)	1	1
1		44200X059	Footplate, Aluminum, Small, Black (6-3/4" W x 6" L)	1	2
1		44200X027	Footplate, Aluminum, Medium, Black (7-3/4" W x 6" L)	1	2
1		44200X025	Footplate, Aluminum, Extra Large, Black - Right (8" W x 8-1/2" L)	1	2
1		44200X026	Footplate, Aluminum, Extra Large, Black - Left (8" W x 8-1/2" L)	1	2
2		1006981	Spring, Tension	1	2
2		1036123	Spring, Leaf	1	2
3		1025729	Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	4
4		1048834	Pivot/Slide Tube w/ Plug Button, Black	1	2
5		1027097	Plug Button, Black (3/4")	1	2
6		1041542-NLA	NLA - Pivot/ Slide Tube w/ Footplate Aluminum 9-1/2" L - Right	1	1
6		1041543-NLA	NLA - Pivot/ Slide Tube Assembly w/ Aluminum Footplate, Black - Left (9-1/2" L)	1	1
6		1026401	Footplate Assembly, Aluminum, Black - Right	1	1
6		1026402	Footplate Assembly, Aluminum, Black - Left	1	1
6		1026395	Pivot/Slide Tube Assembly w/ Extra Large Footplate, Black - Right (9-1/2" L) (16"-20" W)	1	1
6		1026396	Pivot/Slide Tube Assembly w/ Extra Large Footplate, Black -Left (9-1/2" L) (16"-20" W)	1	1