
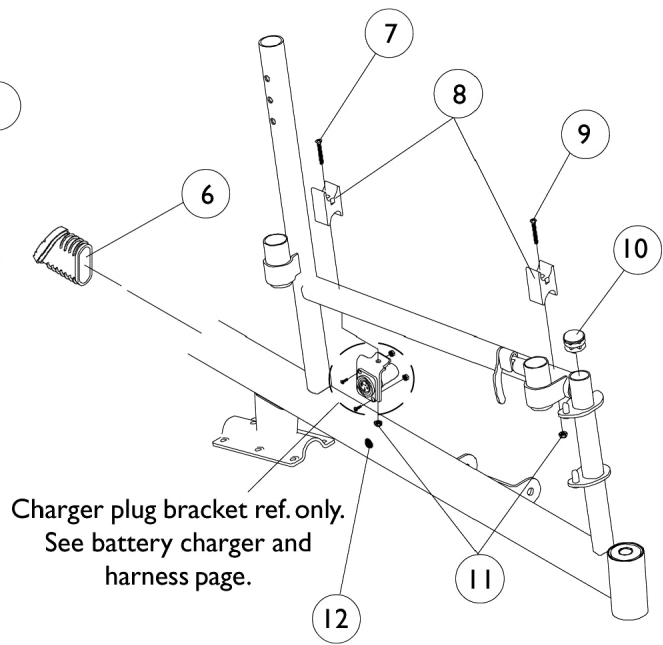


# Side Frames and Hardware

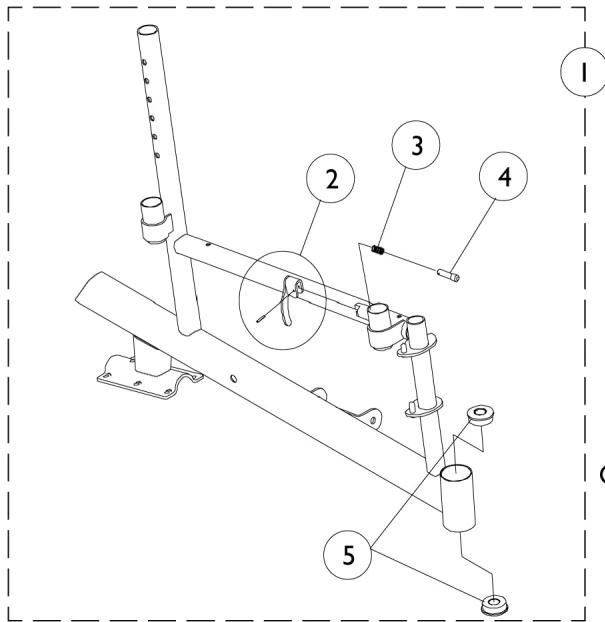

# Side Frames and Hardware

## Item #1 Side Frames

#1 Side Frames	Part Number
Adult 14", 16", 18" 20" Wide - Right	8881057898-NLA See Note(s) B, A
Adult 14", 16", 18" 20" Wide - Left	8881057899-NLA See Note(s) A, B
Hemi 16", 18" 20" Wide - Right	8881057931-NLA See Note(s) B, A
Hemi 16", 18" 20" Wide - Left	8881057932-NLA See Note(s) B, A

NOTE: A - Specify frame color when ordering, see page titled "Frame Finishes, Touch-Up Paints and Upholstery Options."  
B - Decals not included with frame, see page titled "Packaging, Literature and Decals".

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		See Chart	#1 Side Frames and Hardware	1	1
2		1090768	Kit, Lock Handle w/Rivets (Pair)	2	1
3		47471X074	Spring, Arm Lock	1	1
4		1028594	Pin, Arm Lock, Black	1	1
5		40117X006	Bearing (5/8")	1	2
6		1059371-NLA	NLA - Cap, End, Oval	1	1
7	A	1085124-NLA	NLA - Screw, Socket Flat Head (#10-32 x 1-3/4")	1	1
8		1007541	Clamp, Seat	1	2
9	A	1085134-NLA	NLA - Screw, Socket Flat Head (#10-32 x 1-5/8")	1	1
10		1027095	Plug Button, Black (7/8")	1	1
11		1025731	Locknut (#10-32)	1	2
12		1058520-NLA	NLA - Fastener, Panel Snap-In	1	1

NOTE: A - Screws changed 9/16/99 and serves as a direct replacement for chairs manufactured Before 9/16/99.