
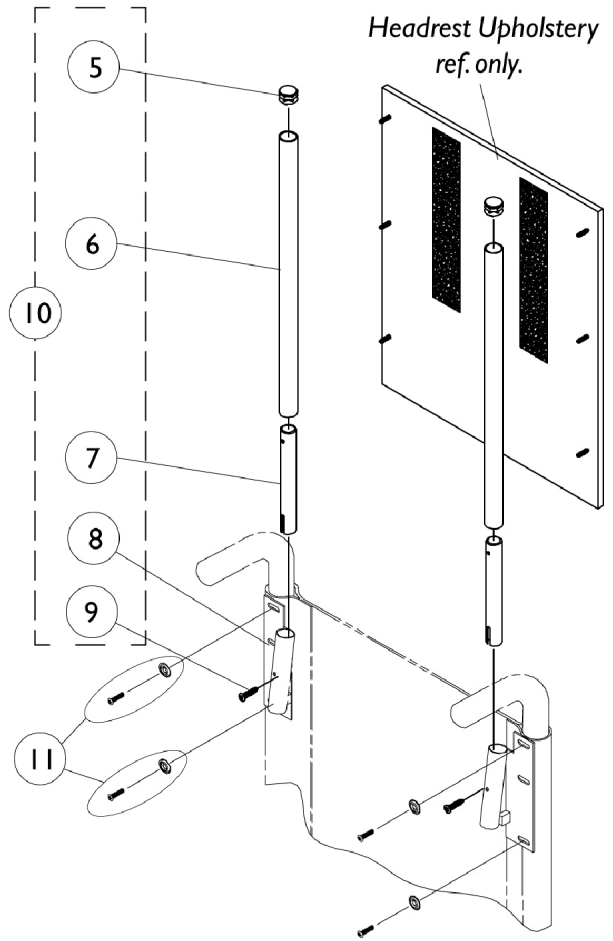


Back Canes and Headrest Extension (9110L)

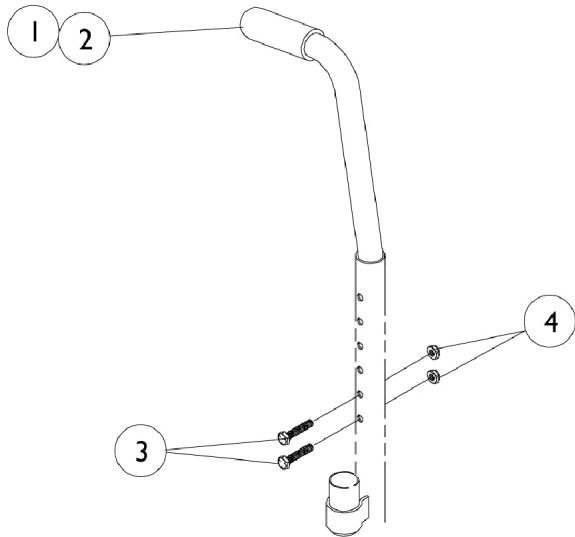

Back Canes and Headrest Extension (9110L)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1028372	Back Cane Assembly, 10 Degree Bend	1	2
2		1006229	Grip, Foam (8")	1	2
3		1057955	Screw, Hex Head (1/4-20 x 1-1/4")	1	4
4		1025195	Locknut (1/4-20)	1	4
5		1027095	Plug Button, Black (7/8")	1	2
6		1054605-NLA	NLA - Tube, Headrest Extension	1	2
7		1028756-NLA	NLA - Tube, Insert	1	2
8		1028752-NLA	NLA - Bracket, Headrest, Black - Right	1	2
8		1028753	Bracket, Headrest, Black - Left	1	2
9		1028484	Screw, Phillips Pan Head Tap (#10-32 x 1/2")	1	2
10	A	1054612-NLA	NLA - Bracketry, Headrest, Black	2	1
11		1031095	Screws with Washers (#10-16 x 7/8")	1	4
11		1076284	Screw w/ Washer (#10 x 7/8")	4	1

NOTE: A - Includes items 5-9