


Elevate Actuator and Mounting Hardware

After 1/3/08 (CGE/CGTE/CGTRE) - TDX SP/SP-GT/SR

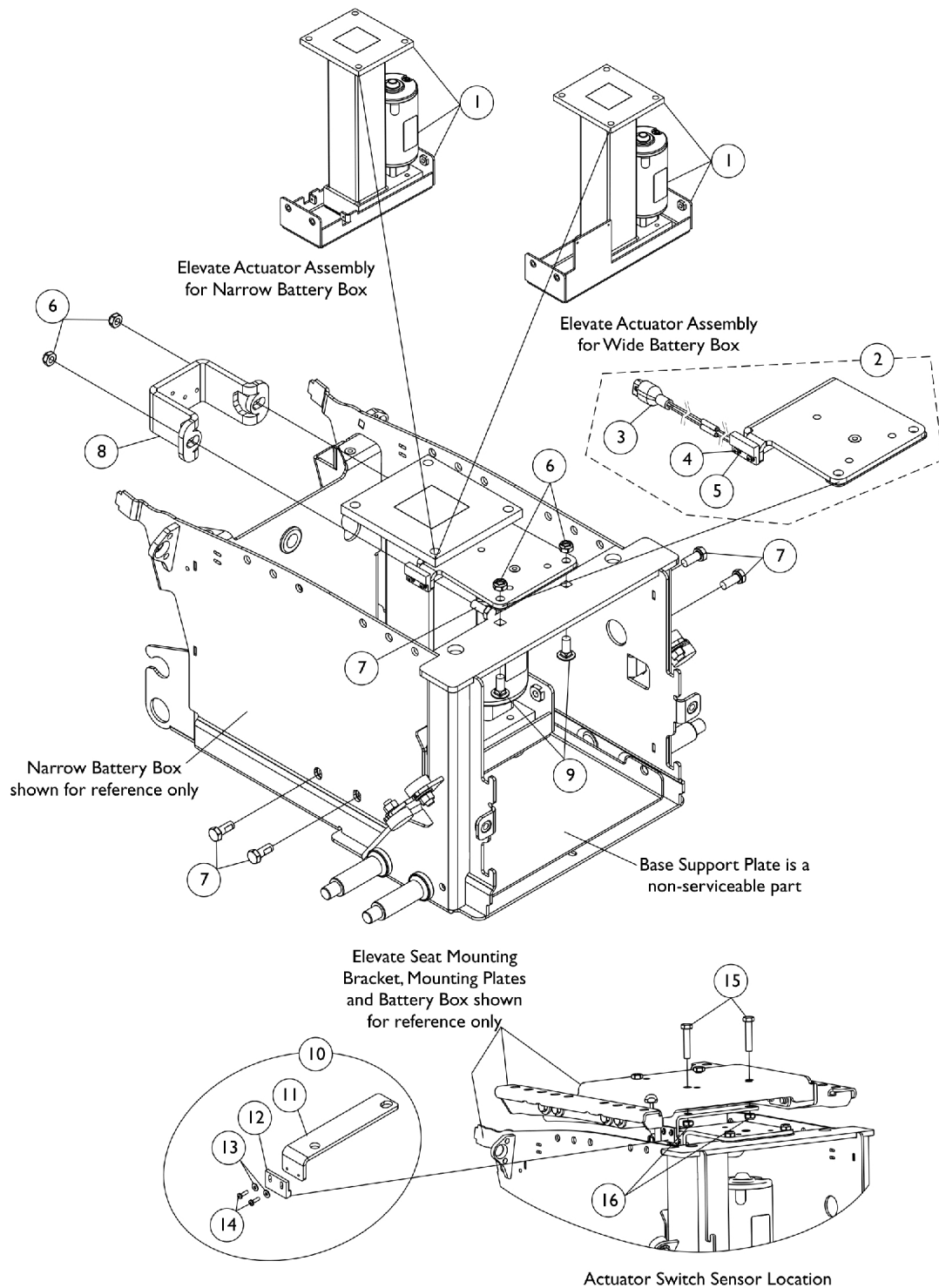

Elevate Actuator and Mounting Hardware

After 1/3/08 (CGE/CGTE/CGTRE) - TDX SP/SP-GT/SR

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1149779	Actuator Motor Assembly, Elevate, For Narrow Battery Box	1	1
1	C	1149780	Actuator Motor Assembly, Elevate, For Wide Battery Box	1	1
2	A	1149760	Elevate Support Bracket Assembly w/ Proximity Switch Sensor	1	1
3		1122586	Sensor, Proximity Switch (30 1/4" L)	1	1
4		1027704-NLA	NLA - Screw, Phillips Pan Head (#4-40 x 3/8")	1	2
5		1011011-NLA	NLA - Washer, Plain (1/8 x 5/16 x 1/32")	1	2
6		1028769	Locknut (5/16-18)	1	4
7		1039860	Screw, Hex Head w/ Patch (5/16-18 x 3/4")	1	6
8		1149761	Bracket, Elevate Actuator Mounting	1	1
9		1146551-NLA	NLA - Screw, Round Head Square Neck (5/16-18 x 3/4")	1	2
10	B	1149781	Kit, Actuator Switch Sensor and Hardware	1	1
11			NLA - Bracket, Actuator Switch Sensor Mounting	1	1
12			Sensor, Actuator Switch	1	1
13			NLA - Washer, Plain (1/8 x 5/16 x 1/32")	1	2
14			NLA - Screw, Phillips Pan Head (#4-40 x 3/8")	1	2
15		1070398-NLA	NLA - Screw, Hex Head (5/16-18 x 1-1/2")	1	2
16		1028769	Locknut (5/16-18)	1	2
NOTE: A - Includes the Elevate Support Bracket and items 3-5 B - Includes items 11-14 C - Part Number 1149780 is used for TDX SR and TDX SP-MCG with 24TRY.					