



# Seat Release Hardware and Adapter Frame


## ADJASBA/ADJRECL Seat Frame TDX SC

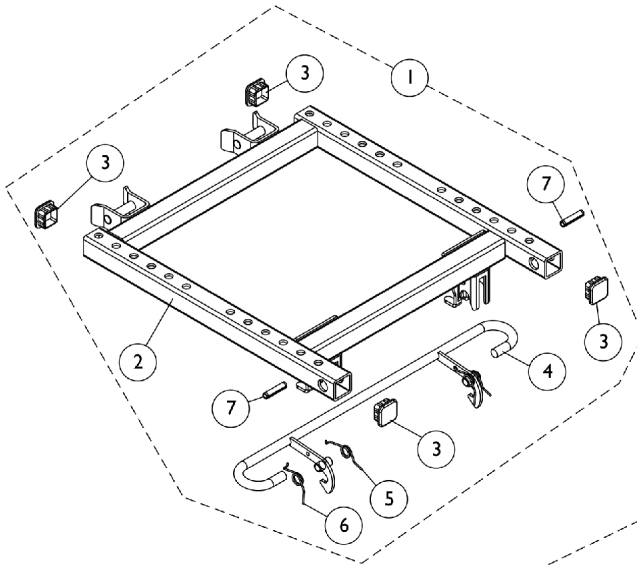

**Seat Frame Adapter with  
Seat Release Hardware**

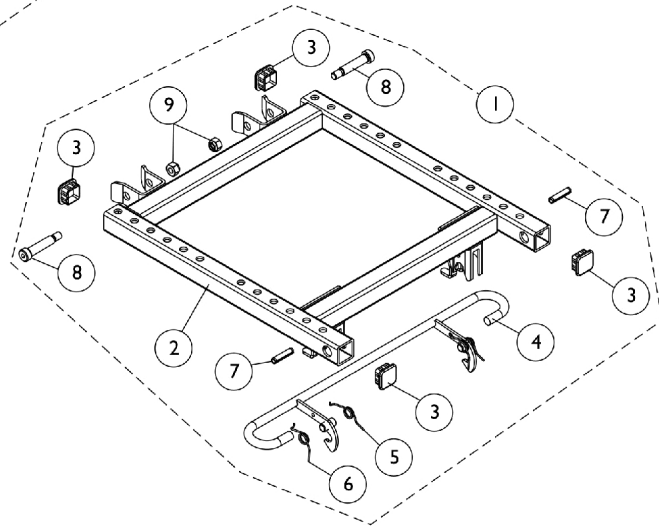

**Seat Frame Adapter with  
Seat Release Hardware  
TRBKTS/TRRO Option only**

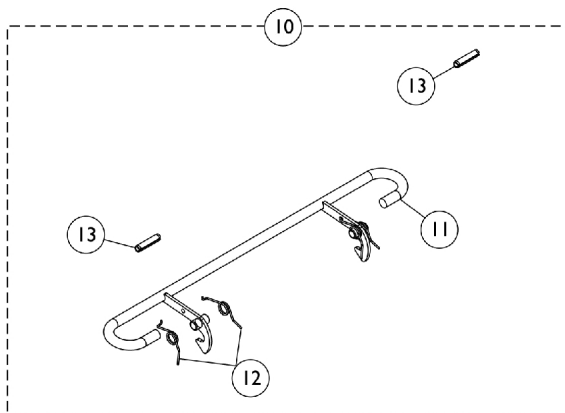

**Seat Release Hardware**

## Seat Release Hardware and Adapter Frame ADJASBA/ADJRECL Seat Frame TDX SC

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1147960-NLA	NLA - Kit, Seat Frame Adapter and Release Hardware	1	1
2			Frame, ADJASBA Seat Adapter, Black	1	1
3			Plug Button, Black (1" Square)	1	4
4			Latch, Seat, Black	1	1
5			NLA - Spring, Torsion - Right	1	2
6			NLA - Spring, Torsion - Left	1	2
7			NLA - Pin, Slotted (1/4 x 1-1/8")	1	2
1	B	1147982-NLA	NLA - Kit, Seat Frame Adapter and Release Hardware - TRBKTS/TRRO Option Only	1	1
2			Frame, ADJASBA Seat Adapter, Black	1	1
3			Plug Button, Black (1" Square)	1	4
4			Latch, Seat, Black	1	1
5			NLA - Spring, Torsion - Right	1	1
6			NLA - Spring, Torsion - Left	1	1
7			NLA - Pin, Slotted (1/4 x 1-1/8")	1	2
8			Screw, Shoulder Hex Socket Head (3/8 x 1-1/2", 5/16-18)	1	2
9			Locknut (5/16-18)	1	2
10		1154109-NLA	NLA - Kit, Seat Release Hardware	1	1
11			Latch, Seat, Black	1	1
12			NLA - Spring, Torsion - Right	1	1
13			NLA - Spring, Torsion - Left	1	1
14			NLA - Pin, Slotted (1/4 x 1-1/8")	1	2

NOTE: A - Includes items 2-7 and used for NON-TRBKTS/TRRO option chairs  
B - Includes items 2-9 and used for TRBKTS/TRRO option chairs only