


Footrest Support Assembly - T93H Series

Veranda Only

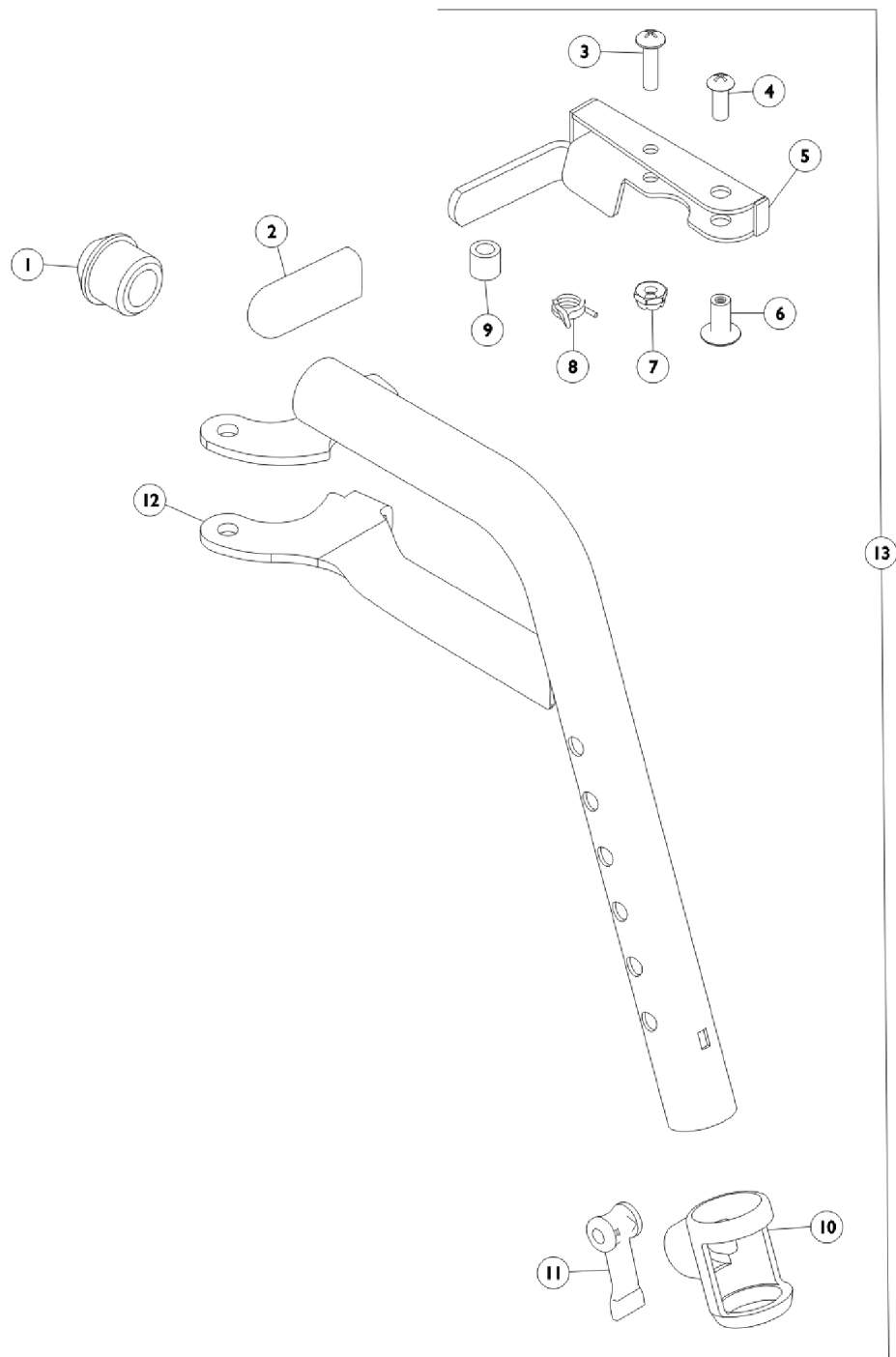

Footrest Support Assembly - T93H Series

Veranda Only

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1020719	Anti-Rattle, Black	1	1
2		1006490	Tip, Plastic, Black	1	1
3	D	40163X008	Screw, Phillips Pan Head (#10-32 x 11/16")	1	1
3	B,C	1025716	Screw, Phillips Pan Head (#10-32 x 11/16")	1	1
4	D	40163X002	Screw, Phillips Round Head (#10-32 x 1/2")	1	1
4	B,C	1026114	Screw, Phillips Pan Head (#10-32 x 1/2")	1	1
5		1110176-NLA	NLA - Handle, Release, Chrome - Right	1	1
5		1110177-NLA	NLA - Handle, Release, Chrome - Left	1	1
5	B,C	1110178	Handle, Release, Black - Right	1	1
5	B,C	1110179	Handle, Release, Black - Left	1	1
6	D	67005X000	Rivnut (#10-32)	1	1
6	B,C	1028846-NLA	NLA - Rivnut (#10-32)	1	1
7	B,C	1025735	Locknut (#10-32)	1	1
8	D	1008935	Spring, Release - Right	1	1
8	D	1008936	Spring, Release - Left	1	1
8	B,C	1028707	Spring, Release - Right	1	1
8	B,C	1028708	Spring, Release - Left	1	1
9		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
10		1048085	Housing, Cam	1	1
11		1048086	Lever, Cam, Black	1	1
12		1111268	Tube, Footrest, Support, Hemi, Black - Right	1	1
12	F	1111269-NLA	NLA - Tube, Footrest, Support, Hemi, Black - Left	1	1
12		1111270-NLA	NLA - Tube, Footrest Support, Hemi, Chrome - Right	1	1
12		1111271-NLA	NLA - Tube, Footrest Support, Hemi, Chrome - Left	1	1
13	A	See Chart	Footrest Support Assembly - T93H Series	1	1

NOTE: A - Includes items 1-12
 B - For use on black wheelchairs.
 C - For use on color wheelchairs.
 D - For use on chrome wheelchairs Before 10/15/14 and black wheelchairs After 10/14/14 .
 F - No longer available individually. See Item number 13 for replacement.

Item #12 Footrest Support Tube is not available in Color. Order item #13 Footrest Support Assembly.

Footrest Support Assembly - T93H Series

Current Model Number	Chair Width	Right Side	Left Side
T93H A/C Color	16-22"	8881118241	8881118242
T93H A/C Black	16-22"	1111272	1111273
T93H A/C Black	16-22"	1111274 See Note(s) B	1111275 See Note(s) B

NOTE: B - After 10/14/14 Color changed from Chrome to Black. It is recommended to order both the right and the left side to match.