


Rear Wheels, Anti-Tippers and Motor Attaching Hardware

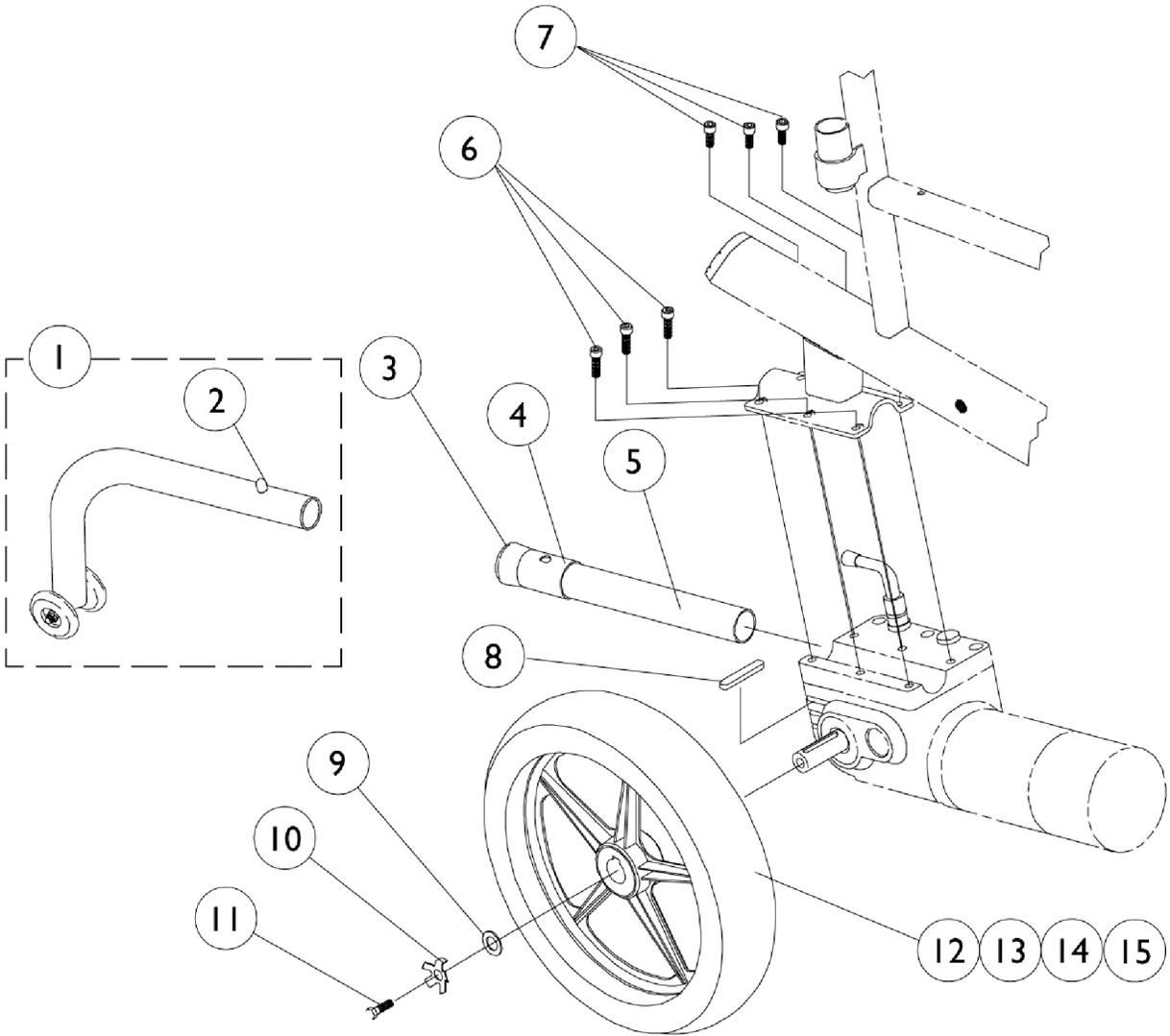

Rear Wheels, Anti-Tippers and Motor Attaching Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1058824-NLA	NLA - Anti-Tipper	1	1
2		1025763	Spring, Torsion with Double Button	1	1
3		15680X013	Anti-Rattle (1")	1	1
4		1037900-NLA	NLA - Bushing, Anti-Wear	1	1
5		1057423-NLA	NLA - Tube, Support, Anti-Tipper, Black	1	1
6		1057823-NLA	NLA - Screw, Socket Head (M6 x 30mm)	1	3
7		1057824-NLA	NLA - Screw, Socket Head (M6 x 16)	1	3
8		1023264	Key (6mm x 6mm x 40mm)	1	1
9		1079247	Washer, Flat (21/64 x 53/64 x 7/64")	1	1
10		1064179	Kit, Locking Tab Washer w/ Instruction Sheet (Pair)	2	1
11		7930322-NLA	NLA - Package, Screw, Hex Head (M8 x 1.25 x 20mm)	25	1
12		1064178-NLA	NLA - Drive Wheel Assembly, 5 Spoke, Pneumatic (12-1/2 x 2-1/4") Includes: Locking Tab Washer with Instruction Sheet	1	1
12		1064177	Drive Wheel Assembly, 5 Spoke w/ Foam Fill Insert (12-1/2 x 2-1/4") Includes: Locking Tab Washer with Instruction Sheet	1	1
13		1011524	Tire, Pneumatic, Grey (12-1/2" x 2-1/4")	1	1
14		1011525	Tube, Inner, Black (12-1/2 x 2-1/4")	1	1
14	A	1073936-NLA	NLA - Insert, Flat Free	1	1
15		1064176-NLA	NLA - Rim with Locking Tab	1	1

NOTE: A - As of 10/20/99 the "solid" flat free insert was replaced by a "foam fill" process. When a Flat Free Insert is requested, a complete wheel must be ordered.