


# Wheel Lock

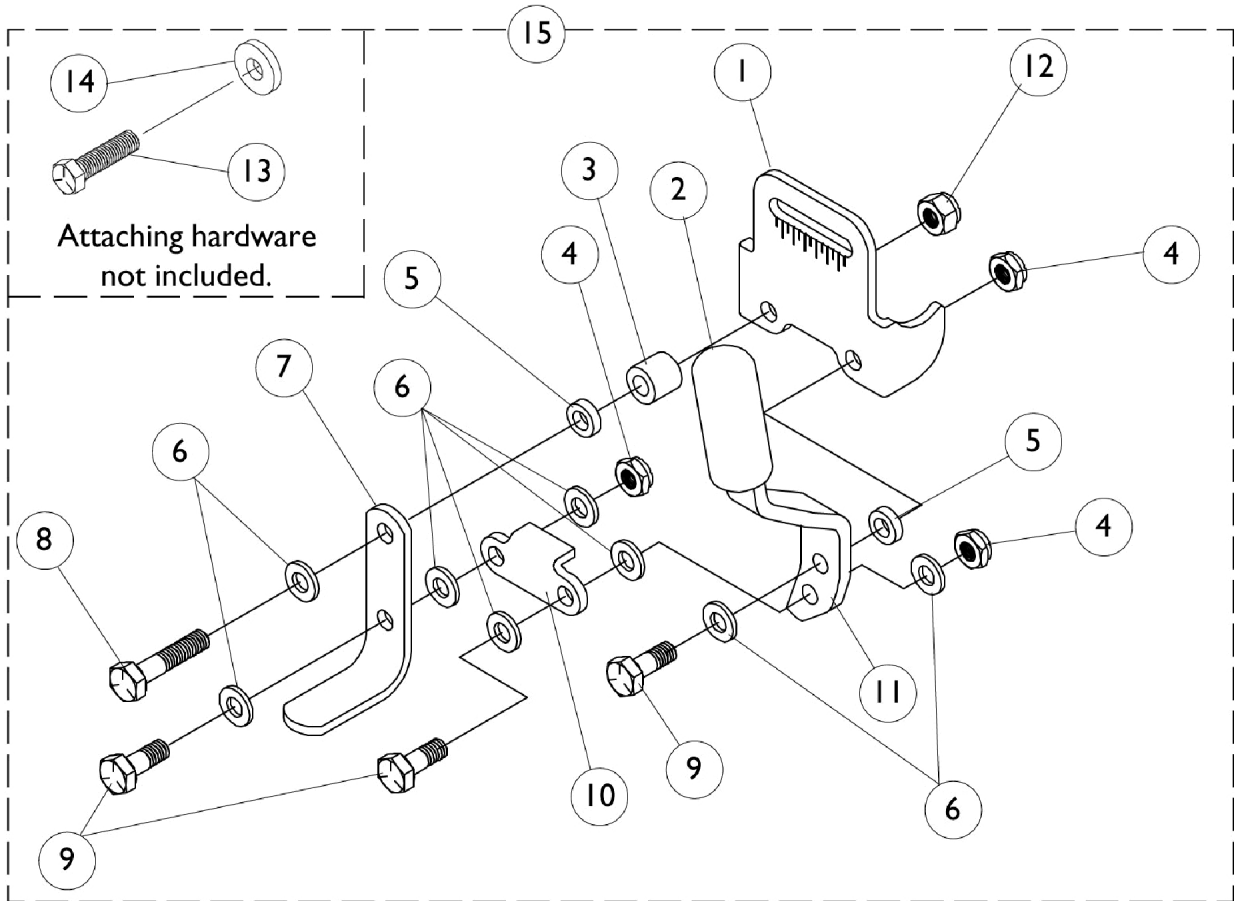

## Wheel Lock

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1057912-NLA	NLA - Bracket, Black	1	1
2		47376X024	Tip, Rubber Wheel Lock Handle, Black	1	1
3		1057946-NLA	NLA - Spacer (17/64 x 31/64 x 7/16")	1	1
4		7930334-NLA	NLA - Locknut (1/4-28)	25	3
5		1026105-NLA	NLA - Spacer, Nylon (1/4 x 7/16 x 1/8")	1	2
6		1026033	Washer, Nylon (1/4 x 1/2 x 1/16")	1	8
7		1031833-NLA	NLA - Shoe, Wheel Lock	1	1
8		1021406	Screw, Hex Head (1/4-20 x 1-1/4")	1	1
9		1028687	Screw, Hex Head (1/4-28 x 3/4")	1	3
10		1018733-NLA	NLA - Link, Stop, Black	1	1
11		1021408-NLA	NLA - Handle, Wheel Lock, Black - Right	1	1
11		1021409-NLA	NLA - Handle, Wheel Lock, Black - Left	1	1
12		7930333	Locknut (1/4-20)	25	1
13		7930326-NLA	NLA - Package, Screw, Hex Head (1/4-20 x 3/4")	25	1
14		1018739	Washer (1/4 x 5/8 x 3/32")	1	1
15		1057947-NLA	NLA - Wheel Lock Assembly - Right	1	1
15		1057948-NLA	NLA - Wheel Lock Assembly - Left	1	1
	A	1058899	Wheel Lock Extension Handle Assembly, Black (6")	1	1
	A	1058900	Wheel Lock Extension Handle Assembly, Black (8")	1	1

NOTE: A - Not Shown.