

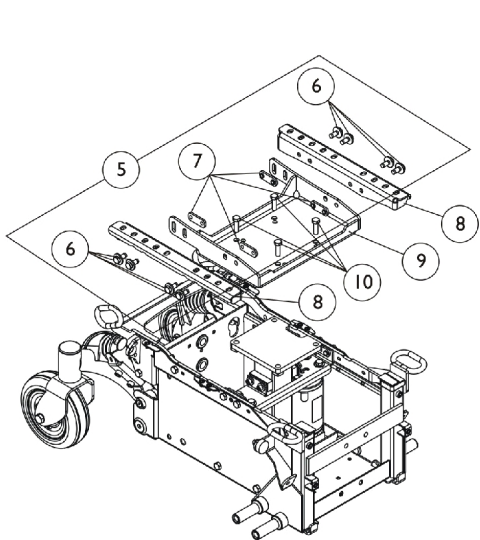


# ADJASBA Seat Frame Mounting Hardware

## (16"-20" Wide) Elevate TDX SC

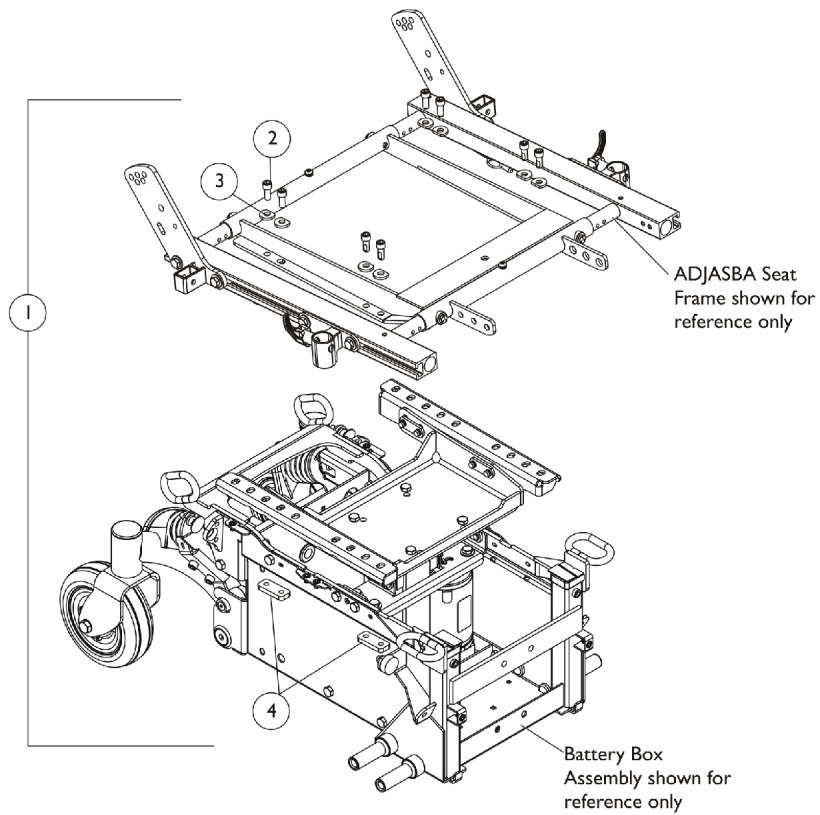

**LOW Seat To Floor Hardware**

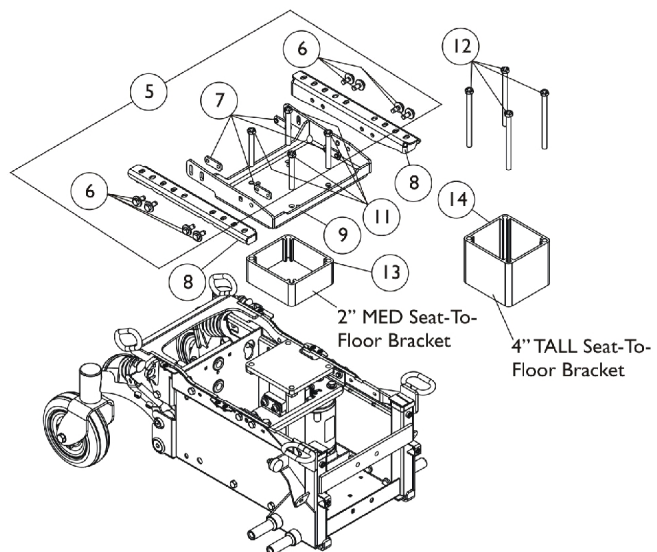

**MED and TALL Seat To Floor Hardware**

## ADJASBA Seat Frame Mounting Hardware (16"-20" Wide) Elevate TDX SC

| Item Number                                                                                                                                                                               | Note | Part Number | Description                                    | Quantity Per                               |          |
|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------|-------------|------------------------------------------------|--------------------------------------------|----------|
|                                                                                                                                                                                           |      |             |                                                | Package                                    | Assembly |
| 1                                                                                                                                                                                         | A    | 1144686     | Kit, Seat Frame Mounting Hardware              | 1                                          | 1        |
| 2                                                                                                                                                                                         |      |             | NLA - Screw, Socket Head (5/16-18 x 3/4")      | 1                                          | 8        |
| 3                                                                                                                                                                                         |      |             | NLA - Washer, Ground (11/32 x 13/16 x 1/8")    | 1                                          | 8        |
| 4                                                                                                                                                                                         |      |             | Plate, Nut (5/16-18)                           | 1                                          | 4        |
| 5                                                                                                                                                                                         | B    | 1154118-NLA | NLA - Kit, Mounting Plate Hardware             | 1                                          | 1        |
| 6                                                                                                                                                                                         |      |             | Screw, Hex Head SEMS w/ Patch (5/16-18 x 3/4") | 1                                          | 8        |
| 7                                                                                                                                                                                         |      |             | NLA - Nut, Weld (5/16-18 x 1-1/8" Spacing)     | 1                                          | 4        |
| 8                                                                                                                                                                                         |      |             | 1121710                                        | NLA - Plate, Elevate Seat Mounting - Right | 1        |
| 8                                                                                                                                                                                         |      | 1121711-NLA | NLA - Plate, Elevate Seat Mounting - Left      | 1                                          | 1        |
| 9                                                                                                                                                                                         |      | 1147098-NLA | NLA - Bracket, Seat Mounting, Elevate, Black   | 1                                          | 1        |
| 10                                                                                                                                                                                        | C    | 1065939-NLA | NLA - Screw, Hex Head (3/8-16 x 1")            | 1                                          | 4        |
| 11                                                                                                                                                                                        | D    | 1151872-NLA | NLA - Screw, Hex Head (3/8-16 x 3")            | 1                                          | 4        |
| 12                                                                                                                                                                                        | E    | 1151873-NLA | NLA - Screw, Hex Head (3/8-16 x 5")            | 1                                          | 4        |
| 13                                                                                                                                                                                        |      | 1151870-NLA | NLA - Bracket, 2" Seat-To-Floor                | 1                                          | 1        |
| 14                                                                                                                                                                                        |      | 1151871-NLA | NLA - Bracket, 4" Seat-To-Floor                | 1                                          | 1        |
| NOTE: A - Includes items 2-4<br>B - Includes items 6 and 7<br>C - Item 10 used for Low Seat-To-Floor<br>D - Item 11 used for Med Seat-To-Floor<br>E - Item 12 used for Tall Seat-To-Floor |      |             |                                                |                                            |          |