



Seat Height Adjustment Slide Tubes and Mounting Hardware

TDX SC

LOW Seat-To-Floor = 17-3/4", MED Seat-To-Floor = 18-1/2", TALL Seat-To-Floor = 19-1/4"

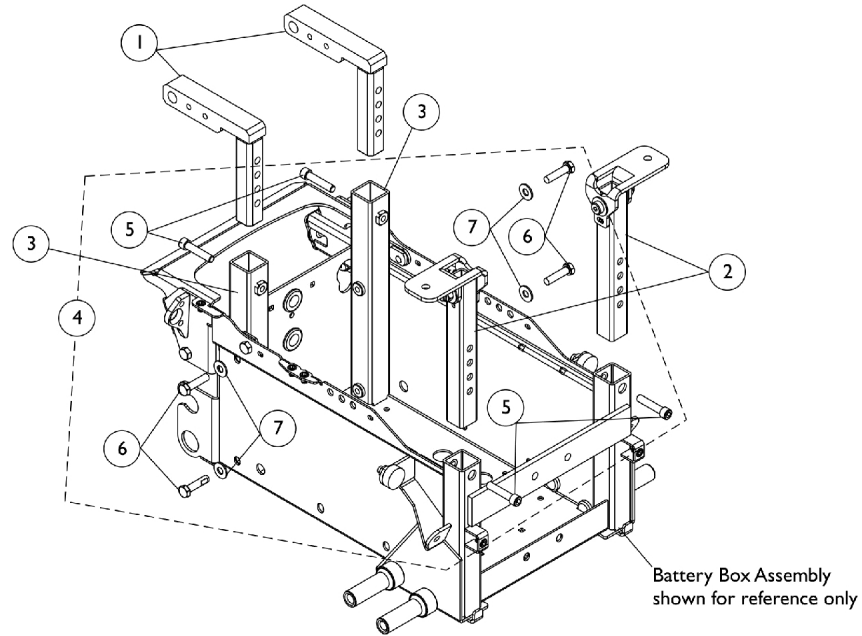

**Seat Height Adjustment Slide
Tubes and Mounting Hardware
without TRBKTS Option**

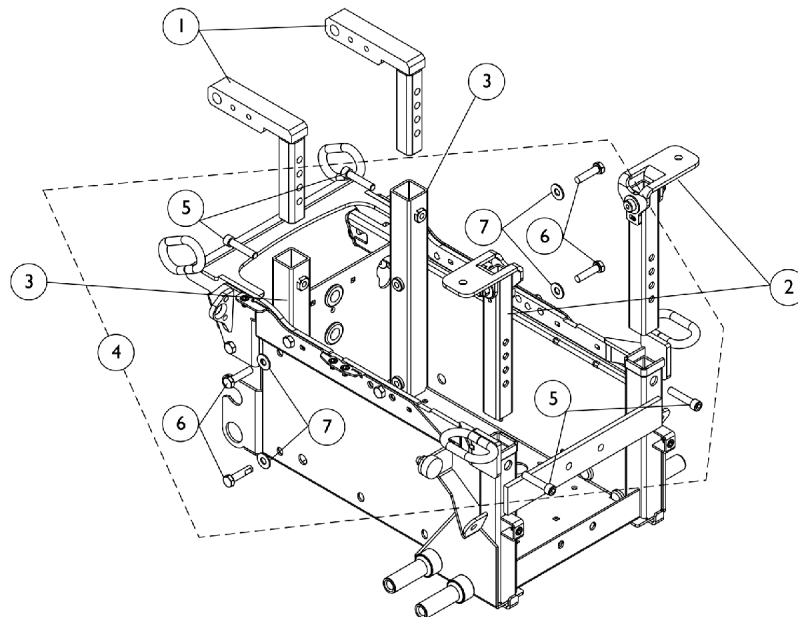

**Seat Height Adjustment Slide
Tubes and Mounting Hardware
with TRBKTS Option**

Seat Height Adjustment Slide Tubes and Mounting Hardware

TDX SC

LOW Seat-To-Floor = 17-3/4", MED Seat-To-Floor = 18-1/2", TALL Seat-To-Floor = 19-1/4"

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1150900-NLA	NLA - Tube, Slide, Seat Height Adjustment, Rear - CGT	1	2
2		1147977-NLA	NLA - Tube, Slide, Seat Height Adjustment, Front - CGT	1	2
3		1149556-NLA	NLA - Tube, Rear Upright, Black - Right	1	1
3		1149557-NLA	NLA - Tube, Rear Upright, Black - Left	1	1
4	A	1154114	Kit, Seat Height Adjustment Slide Tube Mounting Hardware	1	1
5			NLA - Screw, Socket Head w/ Patch (5/16-18 x 1-1/2")	1	4
6			Screw, Hex Head w/ Patch (5/16-18 x 1-1/4")	1	4
7			Washer, Steel (5/16 x 3/4 x 1/16")	1	4

NOTE: A - Includes items 5-7