


P93 Footrest Supports

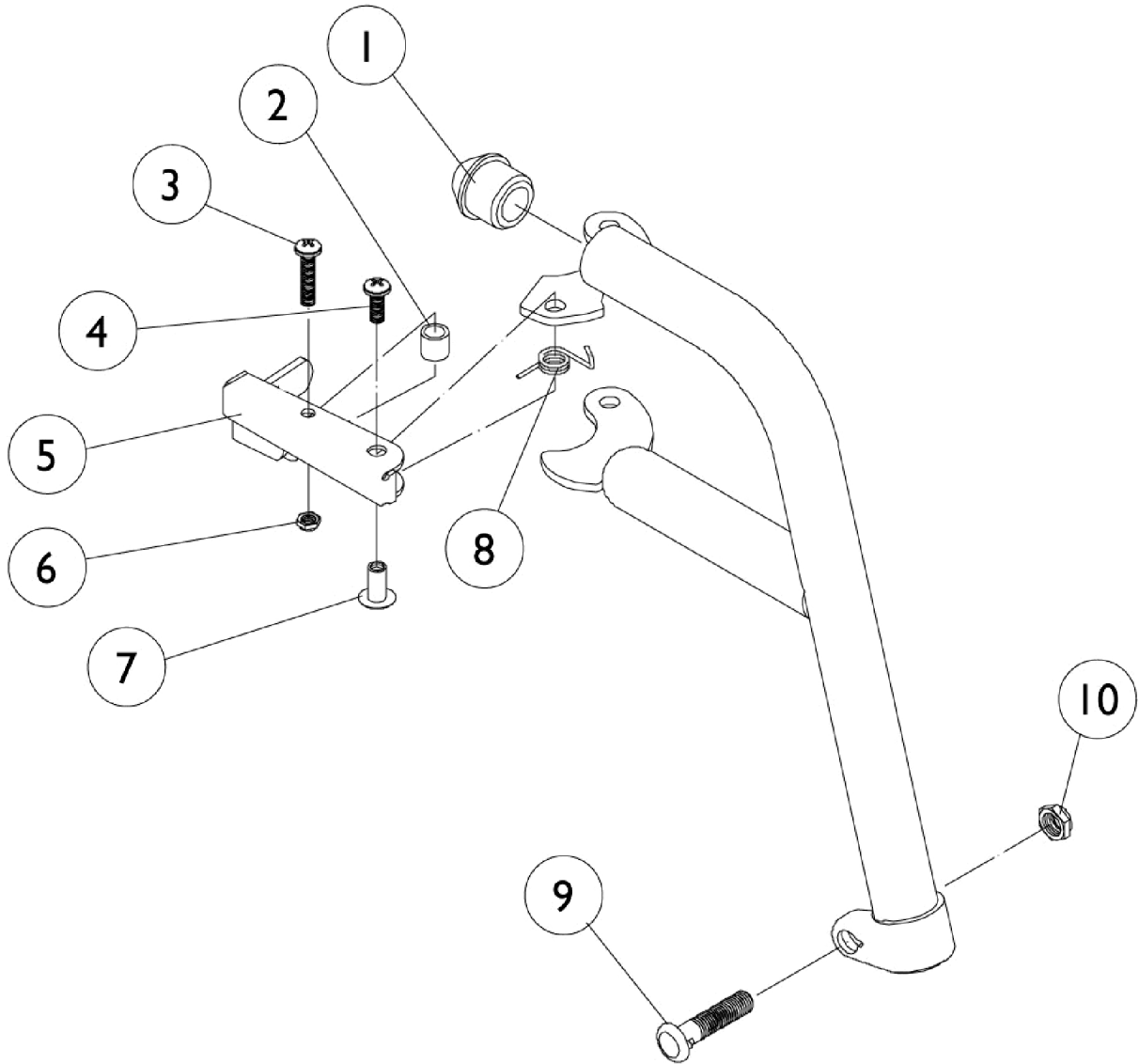

P93 Footrest Supports

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1020719	Anti-Rattle, Black	1	1
2		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
3		7930328-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 11/16")	25	1
4		7930335-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 1/2")	25	1
5		1035539	Handle, Release, Black - Right	1	1
5		1035540	Handle, Release, Black - Left	1	1
6		7930332-NLA	NLA - Locknut, High Profile (#10-32)	25	1
7		1028846-NLA	NLA - Rivnut (#10-32)	1	1
8		1028707	Spring, Release - Right	1	1
8		1028708	Spring, Release - Left	1	1
9		1026030	Bolt, Lug (5/16-24 x 1-3/8")	1	1
10		1107321	Nut, Low Cap (5/16"-24)	1	1
	A,B	1006490	Tip, Plastic, Black	1	1
	A,C	8881035779	Footrest Support Assembly, Adult, Color - Right	1	1
	A,C	8881035780	Footrest Support Assembly, Adult, Color - Left	1	1
	A	1030708	Footrest Support Assembly, Adult, Silver Vein - Right	1	1
	A	1030709	Footrest Support Assembly, Adult, Silver Vein - Left	1	1

NOTE: A - Not Shown
 B - Before 10/1/00 only
 C - Specify frame color when ordering, see page titled "Frame Finishes, Touch-Up Paints and Upholstery Options."