


# Luggage Carrier

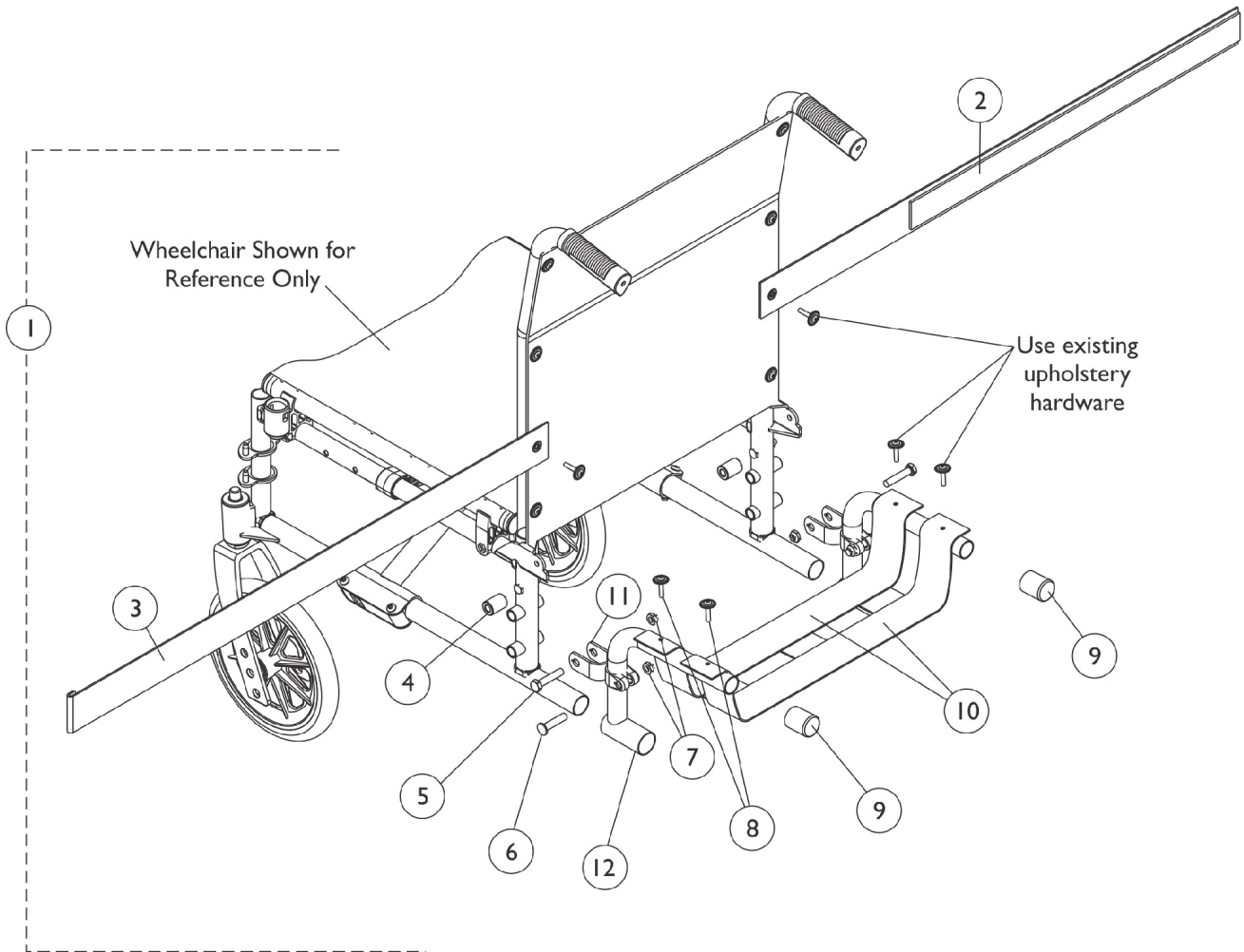

# Luggage Carrier

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	8881154504	Luggage Carrier Assembly w/ Hardware, Color (18" Wide)	1	1
2			Strap Securing Luggage Hook Assembly	1	1
3			Strap Securing Luggage Loop Assembly	1	1
4			NLA - Spacer, Chrome (5/16 x 11/16 x 15/16")	1	2
5			Screw, Hex Head (5/16-24 x 1-1/2")	1	2
6			Bolt, Lug (5/16-24 x 1-3/8")	1	2
7			Locknut (5/16-24)	1	4
8			Screw, Phillips Oval Head w/ Washer (#10-16 x 7/8")	1	4
9			Cap, End, Tube, Black (7/8")	1	2
10			Strap, Luggage Carrier (18" Wide)	1	1
11			Bracket, Mounting, Color	1	2
12			Bracket, Support, Color	1	2
			Instruction Sheet, Luggage Carrier	1	1
1	A	8881154352	Luggage Carrier Assembly w/ Hardware, Color (20" Wide)	1	1
2			Strap Securing Luggage Hook Assembly	1	1
3			Strap Securing Luggage Loop Assembly	1	1
4			NLA - Spacer, Chrome (5/16 x 11/16 x 15/16")	1	2
5			Screw, Hex Head (5/16-24 x 1-1/2")	1	2
6			Bolt, Lug (5/16-24 x 1-3/8")	1	2
7			Locknut (5/16-24)	1	4
8			Screw, Phillips Oval Head w/ Washer (#10-16 x 7/8")	1	4
9			Cap, End, Tube, Black (7/8")	1	2
10			Strap, Luggage Carrier (20" Wide)	1	1
11			Bracket, Mounting, Color	1	2
12			Bracket, Support, Color	1	2
			Instruction Sheet, Luggage Carrier	1	1

NOTE: A - Includes items 1-12. Specify 34P Flat Black when ordering.