


M51 Van Seat Frame and Actuator Hardware w/ Solid Base after 4/29/08

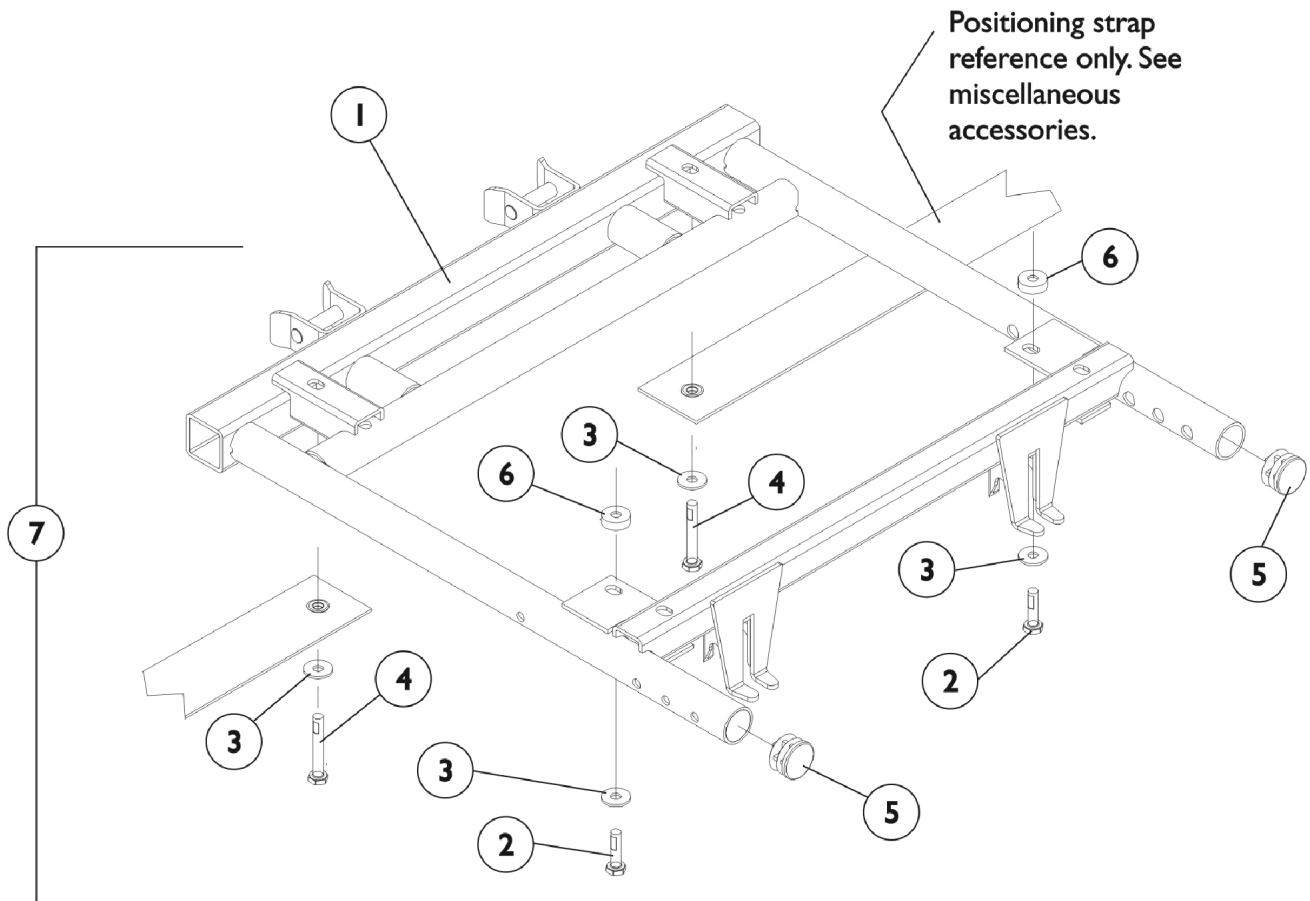

M51 Van Seat Frame and Actuator Hardware w/ Solid Base after 4/29/08

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1123423-NLA	NLA - Frame, Van Seat, Black (16" W)	1	1
1		1115343	Frame, Van Seat, Black (18" W)	1	1
1		1115344	Frame, Van Seat, Black (20" W)	1	1
2		1088897-NLA	NLA - Screw, Hex Head w/ Patch (1/4-20 x 1-1/4")	1	2
3		1026900	Washer (1/4 x 23/32 x 1/16")	1	2
4		1052957	Screw, Hex Head w/ Patch (1/4-20 x 2")	1	2
5		1029501	Plug Button, Black (1")	1	2
6		1108410	Spacer (1/4 x 5/8 x 1/4")	1	2
7	A	1116421	Hardware, Van Seat Mounting	1	1

NOTE: A - Includes items 2-6