

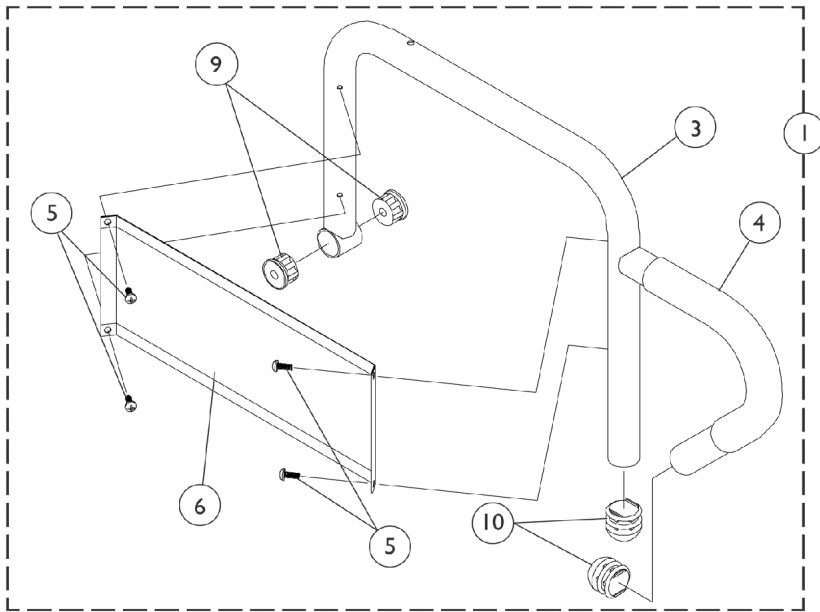




# Arms - ASBA & AdjASBA Seat Fixed Height

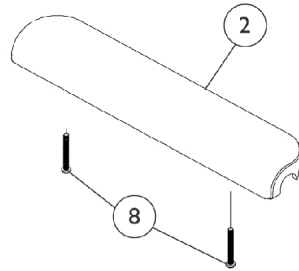

**Desk Length Fixed Height Arm**

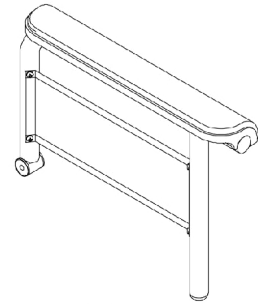

**Full Length  
Fixed Height Arm**

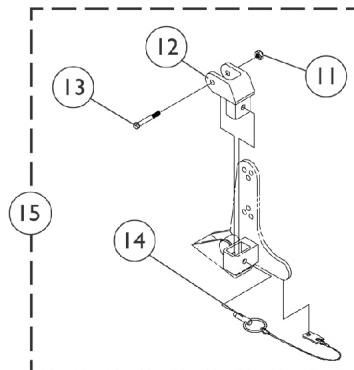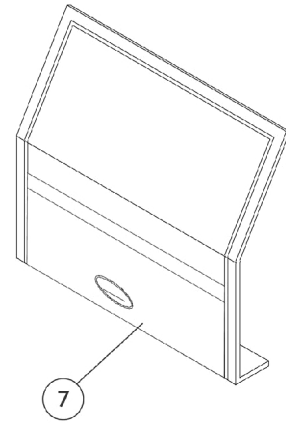

## Arms - ASBA & AdjASBA Seat Fixed Height

| Item Number | Note | Part Number | Description  | Quantity Per |          |
|-------------|------|-------------|--|--------------|----------|
|             |      |             |  | Package      | Assembly |
| 1           |      | See Chart   | #1 Fixed Height Arms w/o Pad   | 1            | 1        |
| 2           | A    | 1102585-NLA | NLA - Pad, Arm, Full Length, Urethane Waterfall, Black (13" L x 2-5/8" W)                | 1            | 1        |
| 2           |      | 1110818     | Kit, Arm Pad w/ Screws, Full Length, Urethane Waterfall, Black (13" L x 2-5/8" W) (Pair) | 2            | 1        |
| 2           | B    | 1102586-NLA | NLA - Pad, Arm, Desk Length, Urethane Waterfall, Black                                   | 1            | 1        |
| 2           |      | 1110817     | Kit, Arm Pad w/ Screws, Desk Length, Urethane Waterfall, Black (9" L x 2-5/8" W) (Pair)  | 2            | 1        |
| 3           |      | See Chart   | #3 Arm Tube - Fixed Height   | 1            | 1        |
| 4           |      | 1032244     | Grip, Foam Hand (9" L)   | 1            | 1        |
| 5           |      | 1025729     | Screw, Phillips Pan Head Tap (#8-32 x 3/8")  | 1            | 4        |
| 6           |      | 1035472-NLA | NLA - Guard, Clothing, Desk Length   | 1            | 1        |
| 6           |      | 1044572-NLA | NLA - Guard, Clothing, Full Length (16 - 19" D)  | 1            | 1        |
| 6           |      | 1044005-NLA | NLA - Guard, Clothing, Full Length (20" and Deeper)                                      | 1            | 1        |
| 7           |      | 1035065     | Cover, Clothing Guard w/ Pocket, FXDESK & ADJFULL/ADJDESK                                | 1            | 1        |
| 7           |      | 1044588     | Cover, Clothing Guard with Pocket Fixed Height Full Length (20" and Deeper)              | 1            | 1        |
| 8           |      | 1055949     | Package, Screw, Phillips Pan Head (#10-32 x 1-3/8")                                      | 2            | 1        |
| 8           |      | 1094270-NLA | NLA - Package, Screw, Phillips Pan Head (#10-32 x 1-1/4")                                | 2            | 1        |
| 9           |      | 1065936     | Plug, Pivot  | 1            | 2        |
| 10          |      | 1067734     | Plug Button, Black (5/8")  | 1            | 2        |
| 11          |      | 1025195     | Locknut (1/4-20)   | 1            | 1        |
| 12          |      | 1068025     | Clevis, Pivot  | 1            | 1        |
| 13          |      | 1019590     | Screw, Socket Button Head (1/4-20 x 1-5/8")  | 1            | 1        |
| 14          |      | 1068039     | Pin, Pull  | 1            | 1        |
| 15          |      | 1068488     | Kit, Swingaway Armrest Mounting Hardware   | 1            | 1        |

NOTE: A - No longer available. Use 1110818 as its replacement.  
B - No longer available. Use 1110817 as its replacement.

### #1 Fixed Height Arms w/o Pad

| Chair Seat Depth | Arm Style   | Part Number |
|------------------|-------------|-------------|
| 16-19"           | Desk -Right | 1068566-NLA |
| 16-19"           | Desk -Left  | 1068567-NLA |
| 16-19"           | Full -Right | 1068559-NLA |
| 16-19"           | Full -Left  | 1068560-NLA |
| 20"              | Desk -Right | 1068540-NLA |
| 20"              | Desk -Left  | 1068541-NLA |
| 20"              | Full -Right | 1068528-NLA |
| 20"              | Full -Left  | 1068529-NLA |

## Arms - ASBA & AdjASBA Seat Fixed Height

---

### #3 Arm Tubes - Fixed Height

| Chair Seat Depth | Arm Style   | Part Number |
|------------------|-------------|-------------|
| 16-19"           | Desk -Right | 1068564     |
| 16-19"           | Desk -Left  | 1068565-NLA |
| 16-19"           | Full -Right | 1068490     |
| 16-19"           | Full -Left  | 1068491-NLA |
| 20"              | Desk -Right | 1068414-NLA |
| 20"              | Desk -Left  | 1068415-NLA |
| 20"              | Full -Right | 1068473-NLA |
| 20"              | Full -Left  | 1068474-NLA |