


Casters and Forks

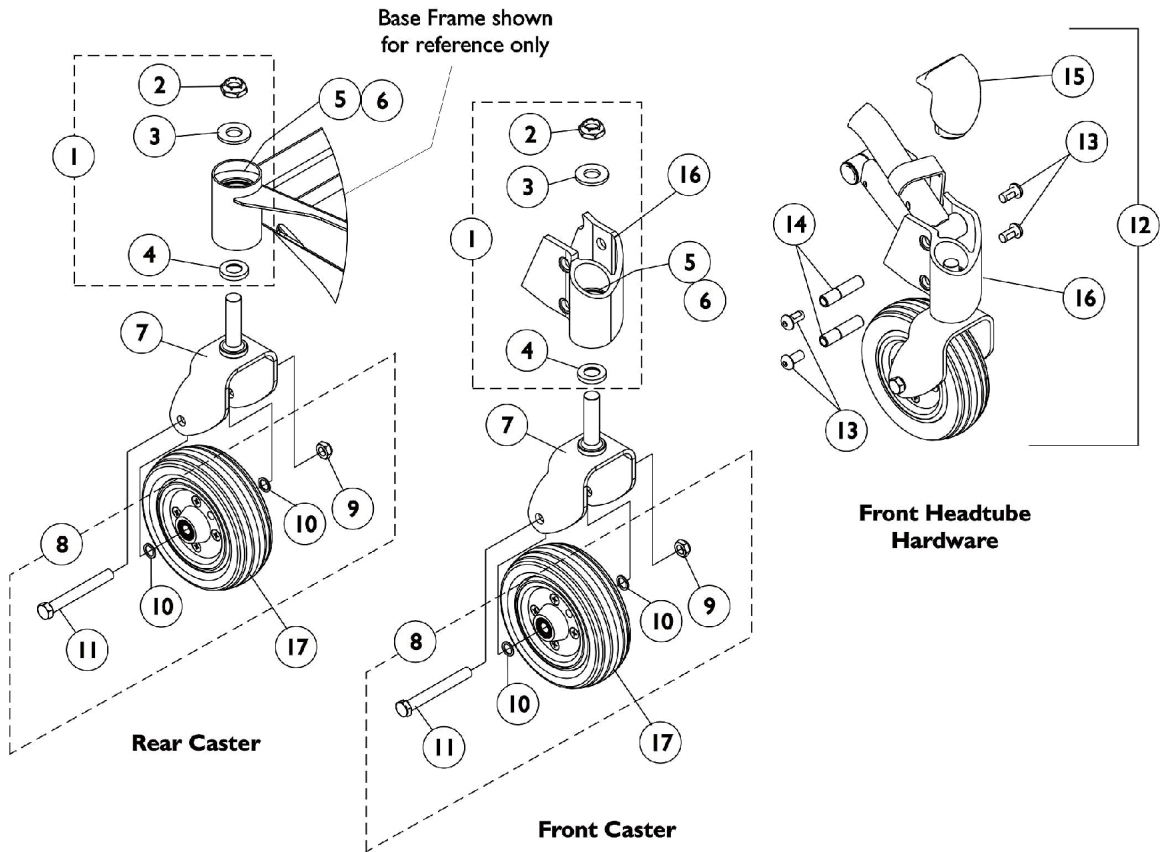

Casters and Forks

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1106226	Kit, Fork Mounting Hardware	1	1
2		1028550	Locknut (5/8-18)	1	1
3		1041444	Washer, Nylon (5/8 x 1-5/16 x 1/8")	1	1
4		1106068	Spacer, Stem (5/8 x 1-1/8 x 3/16")	1	1
5		40117X006	Bearing (5/8")	1	2
6		1110717	Ring, Tolerance	1	2
7		1110683	Fork and Stem, 2-Sided - Right/Left	1	1
8	B	1110708	Kit, Caster Mounting Hardware	1	1
9		1025734	Locknut (7/16-20)	1	1
10		1025724	Washer, Steel (15/32 x 5/8 x 3/64")	1	2
11		1025726	Screw, Hex Head (7/16-20 x 3-1/4")	1	1
12	C	1110688	Kit, Caster Headtube with Hardware - Right	1	1
13			Screw, Socket Button Head Cap (3/8-24 x 3/4")	1	4
14			Bushing, Pivot Head Tube	1	2
15			Cap, Head Tube - Right	1	1
16			Headtube Assembly with Bearings, Black - Right	1	1
12	C	1110689	Kit, Caster Headtube with Hardware - Left	1	1
13			Screw, Socket Button Head Cap (3/8-24 x 3/4")	1	4
14			Bushing, Pivot Head Tube	1	2
15			Cap, Head Tube - Left	1	1
16			Headtube Assembly with Bearings, Black - Left	1	1
12	E,F	1140293	Kit, Caster Headtube with Hardware - Right (TRRO/TRBKTS)	1	1
13			Screw, Socket Button Head Cap (3/8-24 x 3/4")	1	4
14			Bushing, Pivot Head Tube	1	2
15			Cap, Head Tube - Right	1	1
16			Headtube Assembly with Bearings, TRRO/TRBKTS, Black - Right	1	1
			Spacer, Stem (5/8 x 1-1/8 x 3/16")	1	1
12	E,F	1140294	Kit, Caster Headtube with Hardware - Left (TRRO/TRBKTS)	1	1
13			Screw, Socket Button Head Cap (3/8-24 x 3/4")	1	4
14			Bushing, Pivot Head Tube	1	2
15			Cap, Head Tube - Left	1	1
16			Headtube Assembly with Bearings, TRRO/TRBKTS, Black - Left	1	1
			Spacer, Stem (5/8 x 1-1/8 x 3/16")	1	1
13		1109222	Screw, Socket Button Head Cap (3/8-24 x 3/4")	1	4
14		1112633	Bushing, Pivot Head Tube	1	2
15		1113012	Cap, Head Tube - Right	1	1
15		1113013	Cap, Head Tube - Left	1	1
16	D	1110694	Headtube Assembly with Bearings, Black - Right	1	1
16	D	1110695	Headtube Assembly with Bearings, Black - Left	1	1
16	D,F	1140289	Headtube Assembly with Bearings, TRRO/TRBKTS, Black - Right	1	1
16	D,F	1140290	Headtube Assembly with Bearings, TRRO/TRBKTS, Black - Left	1	1

Casters and Forks

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
17		1115179	Wheel Assembly, Semi-Pneumatic, Urethane, Deep Profile, Grey Tire/Black Hub (6" x 2")	1	1
17		1122066	Wheel Assembly, Semi-Pneumatic, Urethane, Deep Profile, Grey Tire/Black Hub (6" x 2") (Pair)	2	1
NOTE: A - Includes Items 2-4 B - Includes Items 9-11 C - Includes Items 13-16 D - Includes Items 5 and 6 E - Includes Items 13-16 & Stem Spacer (not shown) F - Chairs w/ TRRO/TRBKTS option. These headtubes will have a dimple stamped on the outside for identification purposes.					