


Front Rigging - Center Mount Rigging CTMT-FJ (Before 12/10/08) (CTMT-FJ = 9-1/4" - 12" Range)

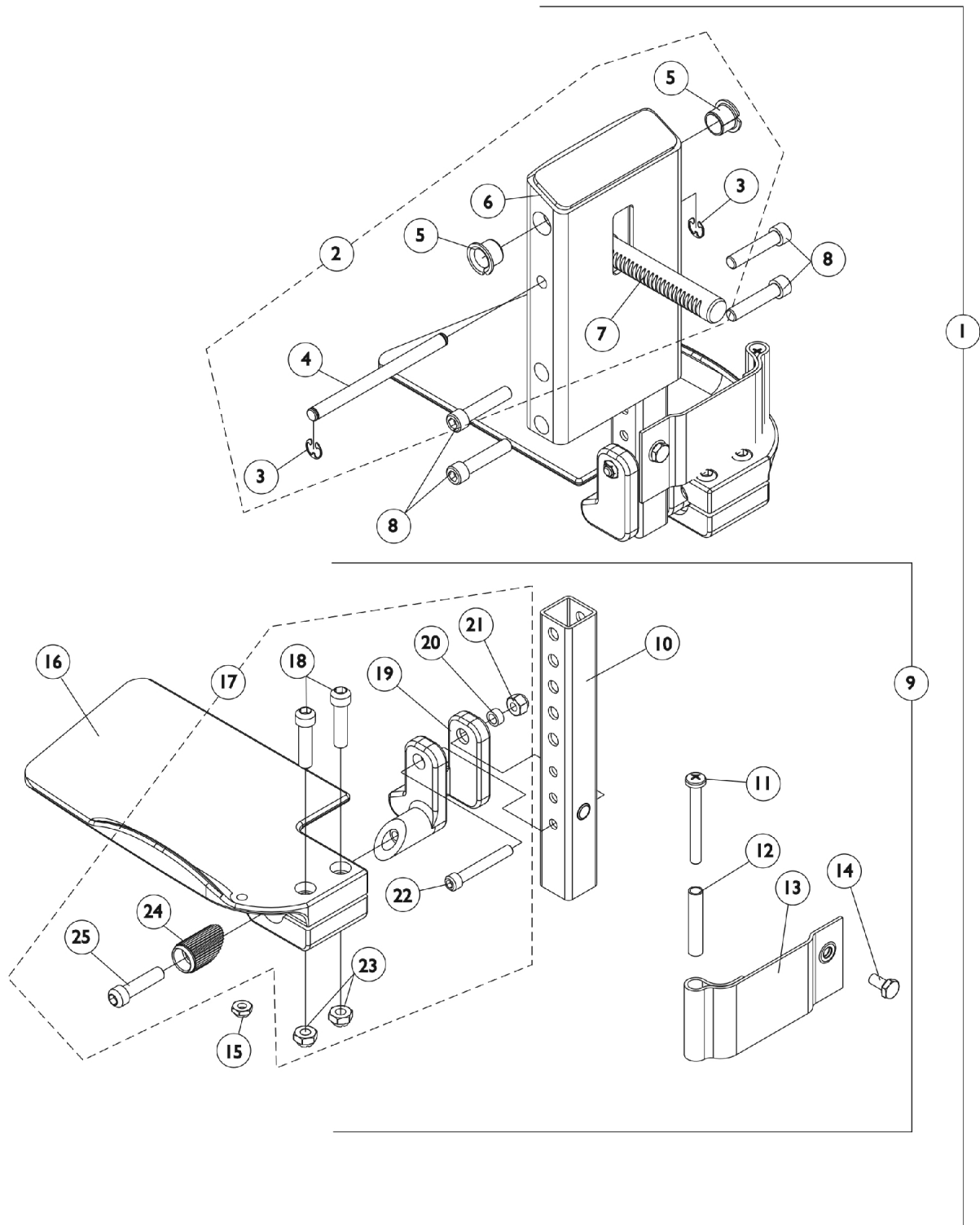

Front Rigging - Center Mount Rigging CTMT-FJ (Before 12/10/08) (CTMT-FJ = 9-1/4" - 12" Range)

| Item Number | Note | Part Number | Description | Quantity Per | |
|-------------|------|-------------|---|--------------|----------|
| | | | | Package | Assembly |
| 1 | A | See Note | See Footnote | 1 | 1 |
| 2 | B | 1123021-NLA | NLA - Legrest Support Assembly, Center Mount, Junior, Black | 1 | 1 |
| 3 | | 1113376-NLA | NLA - Ring, Retainer (1/4 x 1/2 x 1/64") | 1 | 2 |
| 4 | | 1113373-NLA | NLA - Pin, Angle, Adjustable, ASBA | 1 | 1 |
| 5 | | 1110669 | Bearing, Flange (1/2 x 13/16 x 1/2") | 1 | 2 |
| 6 | | 1121984-NLA | NLA - Support, Junior Legrest (CTMTJ/CTMT-FJ) | 1 | 1 |
| 7 | | 1113372 | Rack, Center Mount Rigging Angle Adjustable | 1 | 1 |
| 8 | | 1021344 | Screw, Socket Head (5/16-18 x 1-1/2") | 1 | 4 |
| 9 | C | 1123042-NLA | NLA - Extension Tube and Footplate Assembly w/ Heel Loop, Junior - Right | 1 | 1 |
| 9 | C | 1123043-NLA | NLA - Extension Tube and Footplate Assembly w/ Heel Loop, Junior - Left | 1 | 1 |
| 10 | | 1123044 | Tube, Footrest Extension, Short (CTMTJ/CTMT-FJ = 9-1/4 - 12") (CTMT/CTMT-F = 12-1/4 - 15") (PCTMT = 10 - 13") | 1 | 1 |
| 11 | | 1021421-NLA | NLA - Screw, Phillips Pan Head (1/4-20 x 2-1/4") | 1 | 2 |
| 12 | | 1006477-NLA | NLA - Spacer (1/4 x 11/32 x 1-3/4") | 1 | 2 |
| 13 | | 1121996-NLA | NLA - Heel Loop, Center Mount Rigging | 1 | 2 |
| 14 | | 1078214 | Screw, Hex Head w/ Patch (1/4-20 x 1/2") | 1 | 2 |
| 15 | | 1025195 | Locknut (1/4-20) | 1 | 2 |
| 16 | | 1121988-NLA | NLA - Footplate, CTMTJ/CTMT-FJ - Right (5-1/4" W x 8-13/16" L) | 1 | 1 |
| 16 | | 1121989-NLA | NLA - Footplate, CTMTJ/CTMT-FJ - Left (5-1/4" W x 8-13/16" L) | 1 | 1 |
| 17 | D | 1120345-NLA | NLA - Center Mount Rigging Footplate Mounting Hardware - Right | 1 | 1 |
| 17 | D | 1121123-NLA | NLA - Center Mount Rigging Footplate Mounting Hardware - Left | 1 | 1 |
| 18 | | 1021343 | Screw, Socket Head (5/16-18 x 1-1/4") | 1 | 2 |
| 19 | | 1113384-NLA | NLA - Hinge, Adjustable Angle - Right | 1 | 1 |
| 19 | | 1113385-NLA | NLA - Hinge, Adjustable Angle - Left | 1 | 1 |
| 20 | | 1047449-NLA | NLA - Spacer (1/4 x 3/8 x 3/16") | 1 | 1 |
| 21 | | 1065362 | Locknut (1/4-28) | 1 | 1 |
| 22 | | 1096810 | Screw, Socket Head (1/4-28 x 1-3/4") | 1 | 1 |
| 23 | | 1108122-NLA | NLA - Locknut (5/16-18) | 1 | 2 |
| 24 | | 1113530 | Attachment, Hinge | 1 | 1 |
| 25 | | 1021343 | Screw, Socket Head (5/16-18 x 1-1/4") | 1 | 1 |

NOTE: A - As of 12/10/08, item 1 is no longer available. Individual parts may be ordered separately.
B - Includes items 3-7
C - Includes items 10-16 & 17
D - Includes items 18-25