



Axles & Axle Mounting Plate


Permanent Axle

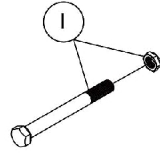



Quick Release Axle
for composite wheels

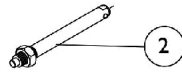

Quick Release Axle
for spoke wheels

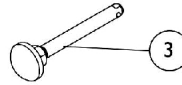

Quad Release Axle

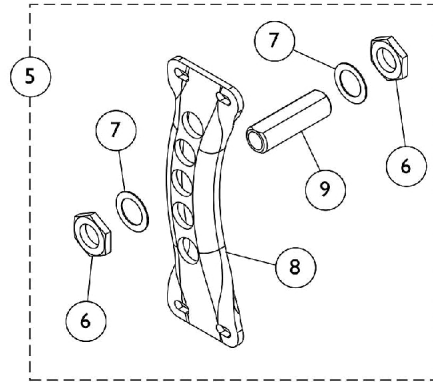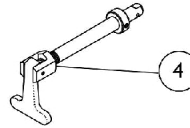

Axle Plate and Bushing Hardware

Axles & Axle Mounting Plate

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1047714	Axle, Permanent Hardware Assembly (1/2" x 5-1/4")	1	1
2	A	1040361	Axle, Quick Release w/ Locknut (1/2" x 5")	1	1
3	B	1113049	Axle, Quick Release w/ Dome Button, Black	1	1
3	C	1065303	Axle, Quick Release w/ Flat Button, Black	1	1
4		1069340	Axle, Quad Release w/ Clip	1	1
5	D	1130545	Kit, Axle Plate and Bushing Hardware, Silver Anodized	1	1
6			Nut, Jam (3/4"-16)	1	4
7			Washer, Stainless Steel (3/4 x 1-1/8 x 1/32")	1	4
8			Plate, Axle - Right	1	1
8			Plate, Axle - Left	1	1
9			Bushing, Threaded Axle	1	2
5	D	1154370	Kit, Axle Plate and Bushing Hardware, Black Anodized	1	1
6			Nut, Jam (3/4"-16)	1	4
7			Washer, Stainless Steel (3/4 x 1-1/8 x 1/32")	1	4
8			Bracket, Axle, Rear, Black Anodized - Right	1	1
8			Bracket, Axle, Rear, Black Anodized - Left	1	1
9			Bushing, Threaded Axle	1	2

NOTE: A - Used with Composite Rear Wheels
 B - Used with all Spoke Rear Wheels except 24SPIN, 25SPIN, and 26SPIN
 C - Used with 24SPIN, 25SPIN, and 26SPIN Spoke Rear Wheels
 D - Includes items 6-9