


Contoura Back without Mounting Hardware

ADJASBA Only - Before 5/15/09

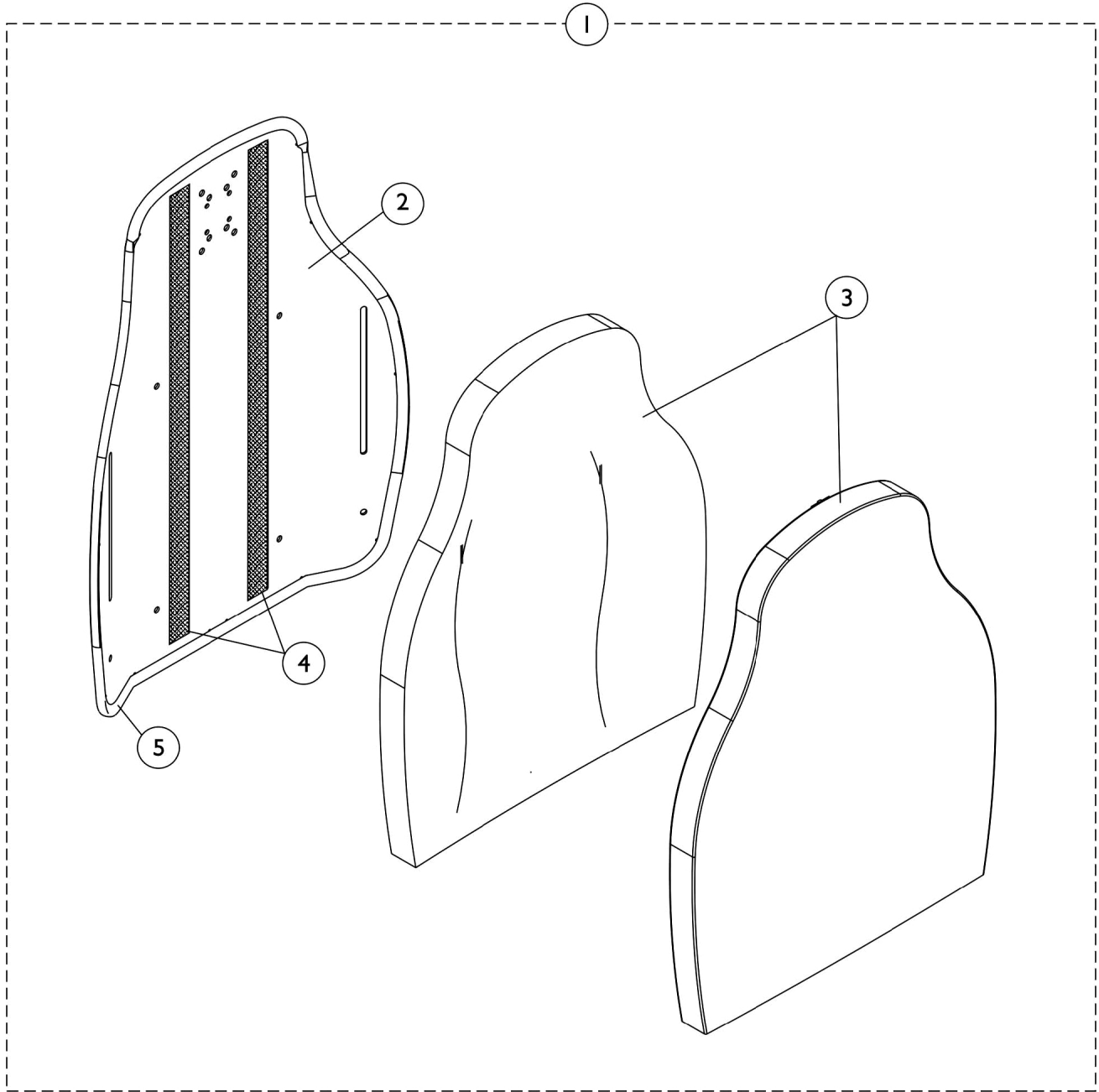

Contoura Back without Mounting Hardware ADJASBA Only - Before 5/15/09

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		See Chart	#1 Back Assembly without Hardware 16", 18", 20", 22" Wide	1	1
2		See Chart	#2 Back Pan with Vinyl Edge Trim 16", 18", 20", 22" Wide	1	1
3		See Chart	#3 Back Cushion Assembly 16", 18", 20", 22" Wide	1	1
4		1125725-NLA	NLA - Hook Fastener w/ Adhesive Back (1" W x 48" L)	1	1
5		1125724-NLA	NLA - Trim, Edge, Cut-To-Length, Vinyl Black (96" L)	1	1

Contoura Back without Mounting Hardware

ADJASBA Only - Before 5/15/09

#1 Back Assembly without Hardware 16", 18", 20", 22" Wide

Overall Back Height	Material Back Height	16" Wide	18" Wide	20" Wide	22" Wide
25"	20"	1132828-NLA	1132829-NLA	1132830-NLA	1132831-NLA
29.5"	24"	1132825-NLA	1124733-NLA	1124734-NLA	1132826-NLA

#2 Back Pan with Vinyl Edge Trim 16", 18", 20", 22" Wide

Overall Back Height	Material Back Height	16" Wide	18" Wide	20" Wide	22" Wide
25"	20"	1131405-NLA	1131406-NLA	1131407-NLA	1131408-NLA See Note(s) A
29.5"	24"	1131410-NLA	1122454-NLA	1122453-NLA	1131411-NLA

NOTE: A - Item is No Longer Available. Back Assembly, part # 1132831 must be purchased for replacement.

#3 Back Cushion Assembly 16", 18", 20", 22" Wide

Overall Back Height	Material Back Height	16" Wide	18" Wide	20" Wide	22" Wide
25"	20"	1132820-NLA	1132821-NLA	1132822-NLA	1132823-NLA
29.5"	24"	1132817-NLA	1125641-NLA	1125642-NLA	1132818-NLA