


Crossbrace Hardware

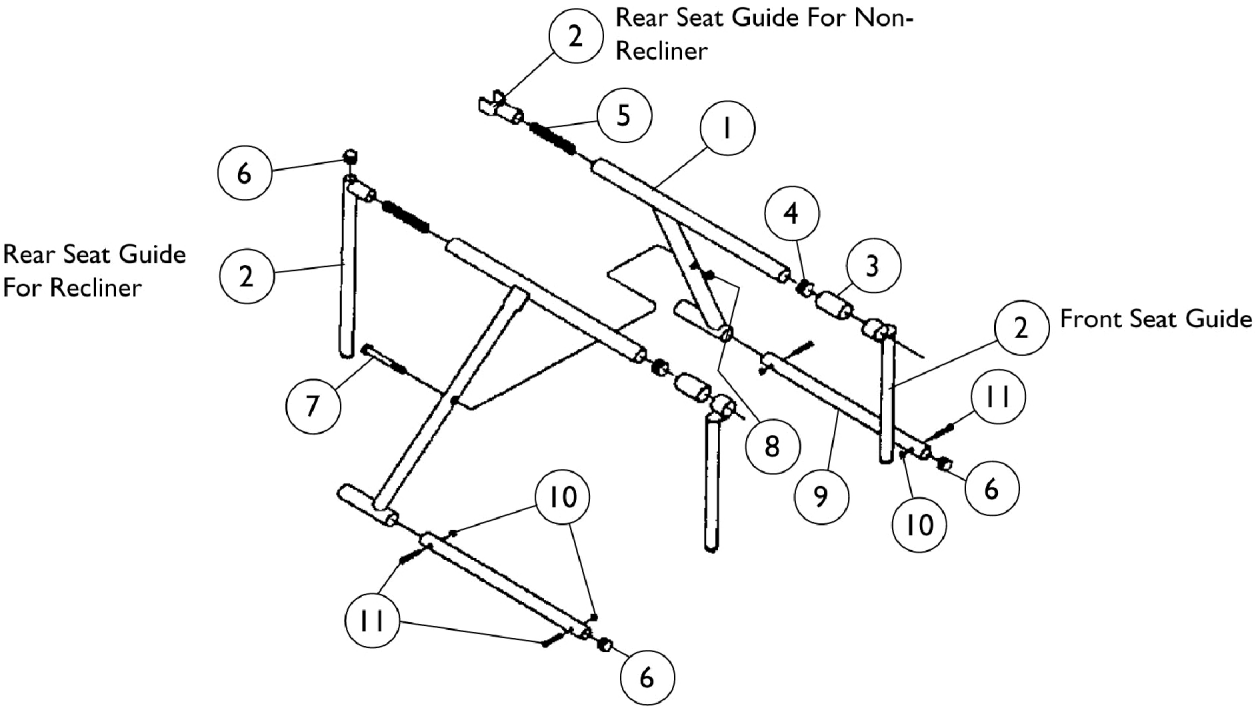

Crossbrace Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	B	See Page	See Footnote	1	1
2		1028598-NLA	NLA - Guide, Front Seat, All Models, Black	1	2
2	C	1028599-NLA	NLA - Guide, Rear Seat, Non-Recliner, Std., No Modifications, Black	1	2
2	C	1034907-NLA	NLA - Seat Guides, Black, Rear Non-Recliners, 2" Lower	1	2
2	D	1031090-NLA	NLA - Guide, Rear Seat, Non-Recliner, Black	1	2
2	E	Special	See Footnote	1	2
2	F	1029148-NLA	NLA - Guide, Rear Seat, Recliner, Black	1	2
2	G	Special	See Footnote	1	2
2	H	1029254-NLA	NLA - Seat Guides, Black Rear, Recliners	1	2
3		1028601-NLA	NLA - Spacer, Non Recliners, Black	1	2
3		1029149-NLA	NLA - Spacer, Tube (29/32 x 1 x 1-3/8")	1	2
4		1027095	Plug Button, Black (7/8")	1	2
5		47236X001	Spring, Tension	1	2
6		1027097	Plug Button, Black (3/4")	1	2
7	A	1028600-NLA	NLA - Screw, Hex Head (5/16-24 x 3")	1	2
8	A	7930330-NLA	NLA - Locknut (5/16-24)	25	1
9		1002088-NLA	NLA - Tube, Lower Insert	1	2
10		7930332-NLA	NLA - Locknut, High Profile (#10-32)	25	4
11		7930327-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 1-1/8")	25	4
NOTE: A - Chairs that have dual crossbraces will use 2 each of the bolt and nut. B - See Crossbraces without Hardware pages. C - 14-16" or 18" Wide D - (14-16" or 18" Wide - Talls) (20 or 22" Wide - Large) (All Widths Modified) E - 20 or 22" Wide - Large, 2" Lower, Modified F - All Chairs - Except 2" Lowers G - 2" Lower H - With Dual Crossbraces					